


# Ribbon layout (administration)

This admin page allows you to setup default Ribbon layout that consists of different tabs and controls. Each tab consists of different controls associated to it and you can configure appearance of tabs and controls using this page.

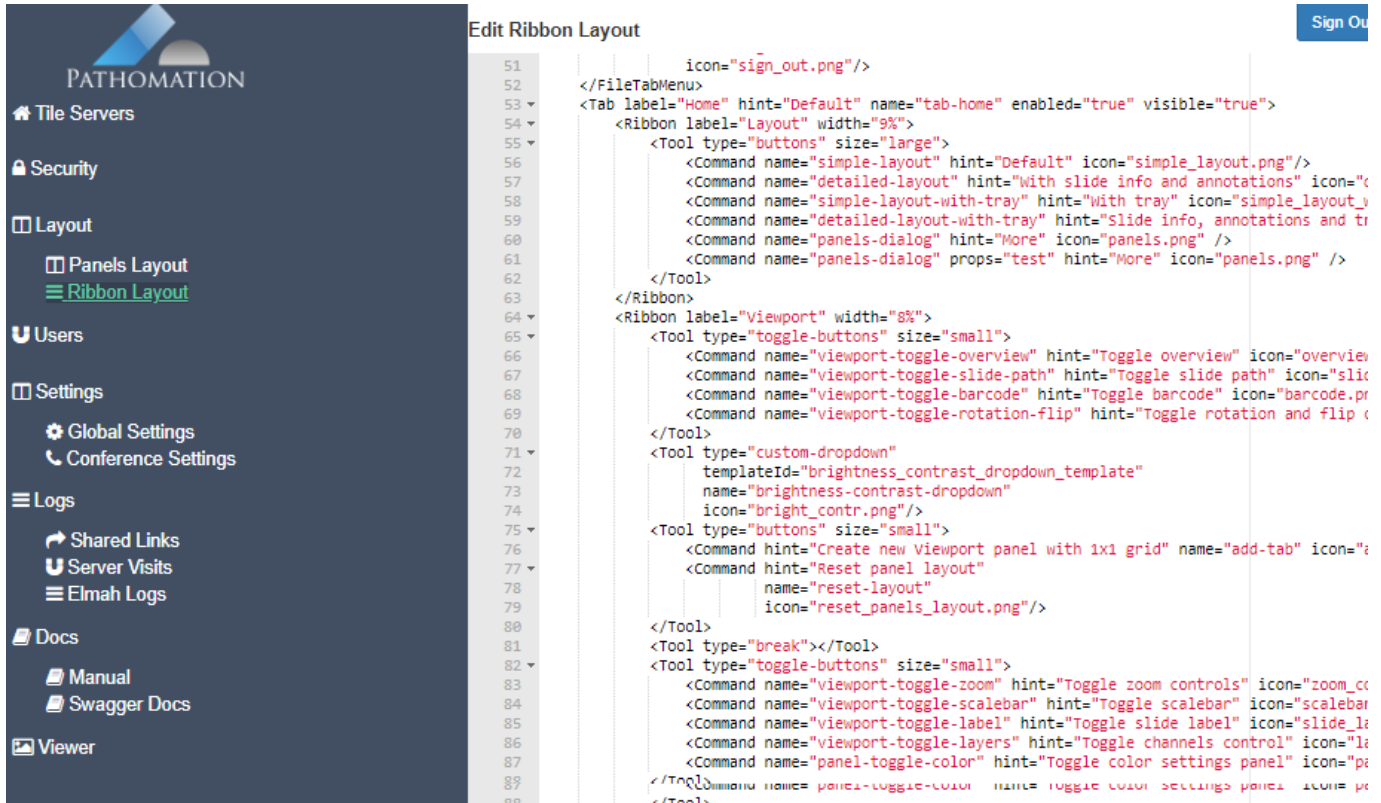

The screenshot shows the 'Edit Ribbon Layout' page in PMA.Studio. The left sidebar is dark blue with the Pathomation logo and navigation menu. The main area is white and displays XML code for configuring the ribbon layout. The code includes tabs like 'Home' and 'Layout', and various tools and commands. A 'Sign Out' button is visible in the top right corner.

Below are some of the tabs and corresponding controls used in PMA.Studio.

**Home tab** has 8 sections:

1. View
2. Grid
3. Snap
4. Magnify
5. Directory
6. Slide
7. Share
8. Search



**Grid tab** supplies instruments to manage grid and it has 6 sections:

1. ItemSize
2. Synchronize
3. Rows/Columns
4. Slide
5. Snapshot
6. Share




**Annotations tab (available in PMA.Studio)** supplies instruments for working with annotations and has 4 sections:

- 1. Annotations
- 2. Shapes
- 3. Export
- 4. Import



**Pre-set Annotations tab (available in PMA.Studio)** consists of 3 sections:

- 1. Annotations
- 2. Stamps
- 3. Protocols

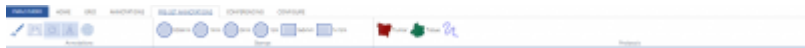


**Conferencing tab** has 2 sections:

- 1. Conferencing
- 2. Server



**Configure tab** has 4 sections:

- 1. Server
- 2. Panels
- 3. Pre-set layouts
- 4. Viewport

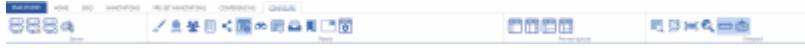

From: <https://docs.pathomation.com/pimsls/3.1.1/> - **PIMS LS 3.1.1**

Permanent link: [https://docs.pathomation.com/pimsls/3.1.1/doku.php?id=admin\\_ribbon&rev=1674470694](https://docs.pathomation.com/pimsls/3.1.1/doku.php?id=admin_ribbon&rev=1674470694)

Last update: **2023/01/23 13:44**

