


Annotation Import and Export and 3rd Party Annotations



PIMS LS allows you to import and export annotations in a variety of formats, including Well-known text (the format used by PMA.core internally), Visiopharm mld files, Indica Labs HALO annotations files, Definiens/Aperio xml files, and 'plain' csv files.

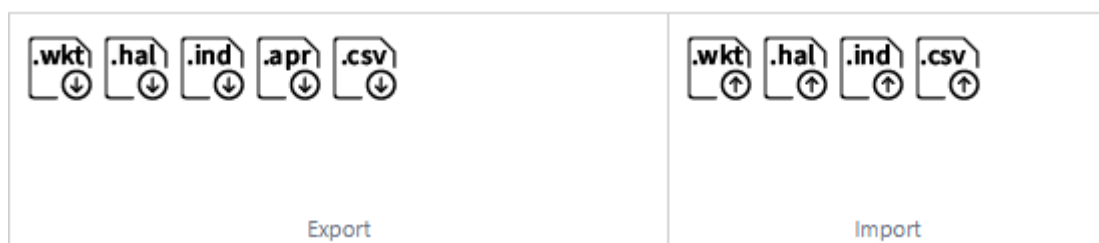

Export

You can export slide annotations in different formats including Well-known text (the format used by PMA.core internally), Visiopharm mld files, Indica Labs HALO annotations files, Definiens/Aperio xml files, and 'plain' csv files.

Import

User can also import/upload annotations through PIMS LS. Once you click on the import button for the format you want to import, a menu will appear prompting you to select the file that contains annotation data.

Once a valid file is uploaded annotations are added to the slide.

Upload new annotations:

Annotation type wkt

Annotation file No file chosen

From: <https://docs.pathomation.com/pimsls/3.1.1/> - PIMS LS 3.1.1

Permanent link: https://docs.pathomation.com/pimsls/3.1.1/doku.php?id=annotations_import_export&rev=1746910036

Last update: 2025/05/10 23:47

