


# Simple Shapes


Below are the set of simple shapes that can be added to a annotation.

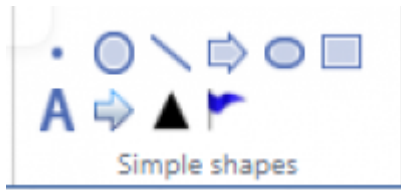

- Point
- Circle
- Line
- Arrow
- Ellipse
- Rectangle
- Text
- Landmark 1
- Landmark 2
- Landmark 3


## Point Shape

Lets start with drawing a point, once you select the shape and click on the slide; a point will be drawn and a popup dialog would appear that would allow you to add annotation title and classification. Once you press "Add Annotation" the annotation will be saved and will appear on the slide.

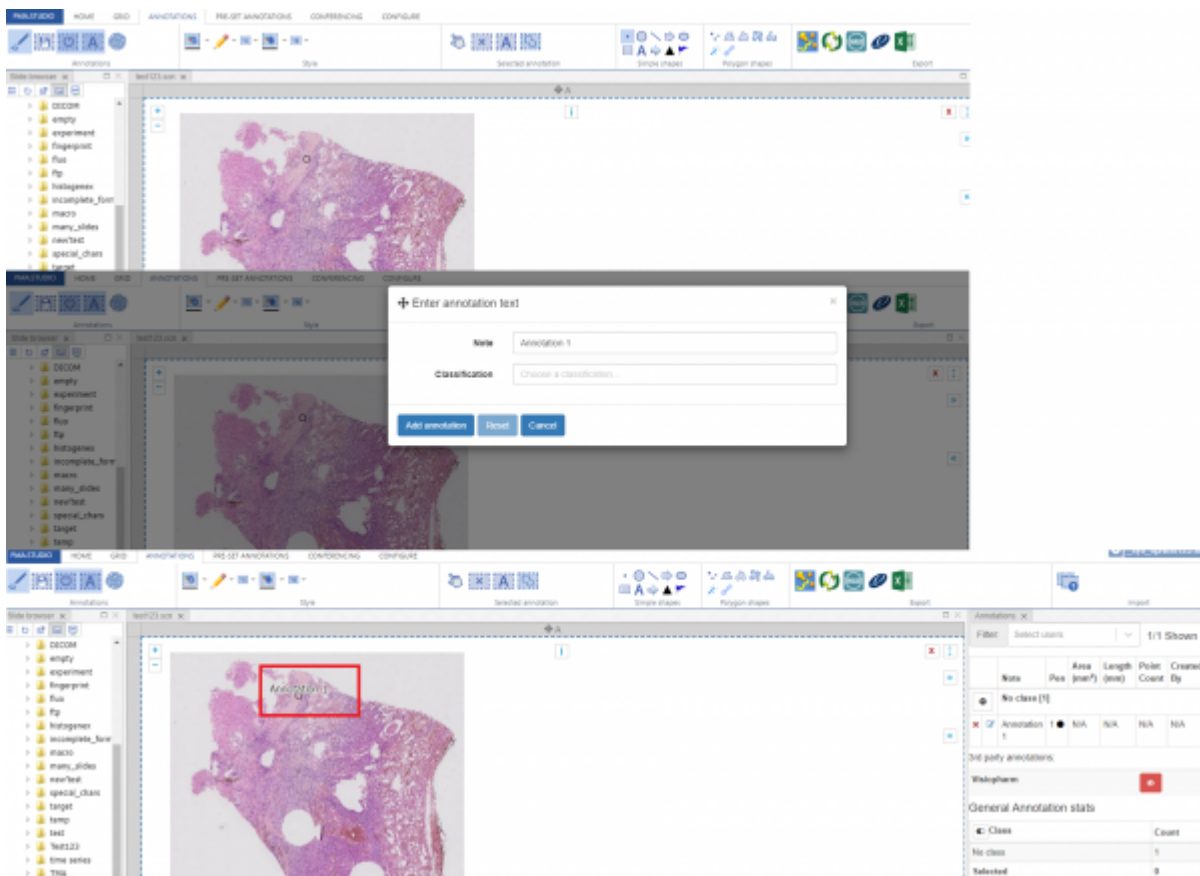


## Circle Shape

Next is “Circle” shape that allows you to draw circle annotation on a slide.

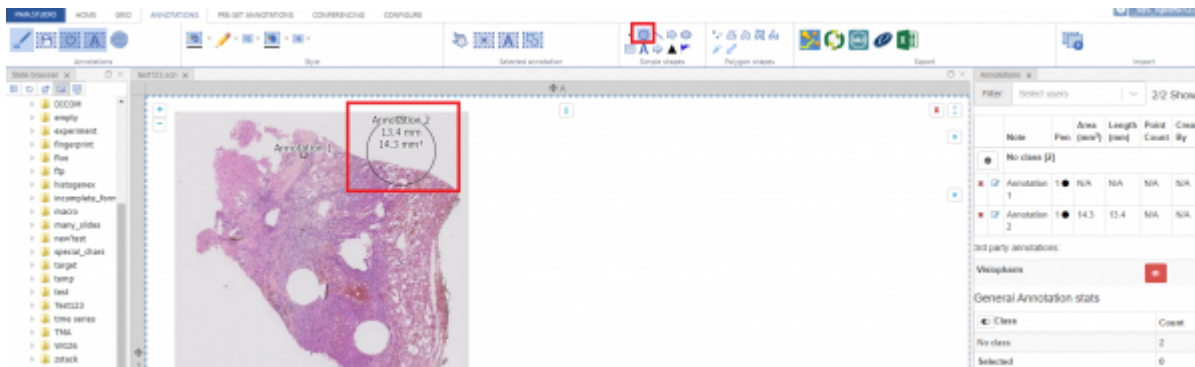


## Line Shape

Next is “Line” shape that allows you to draw line annotation on a slide.

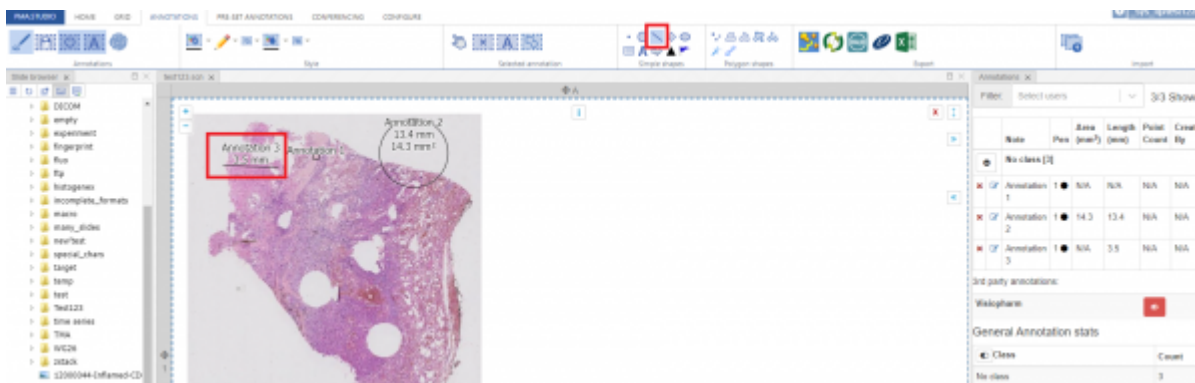


## Arrow Shape

Next is “Arrow” shape that allows you to draw arrow annotation on a slide.

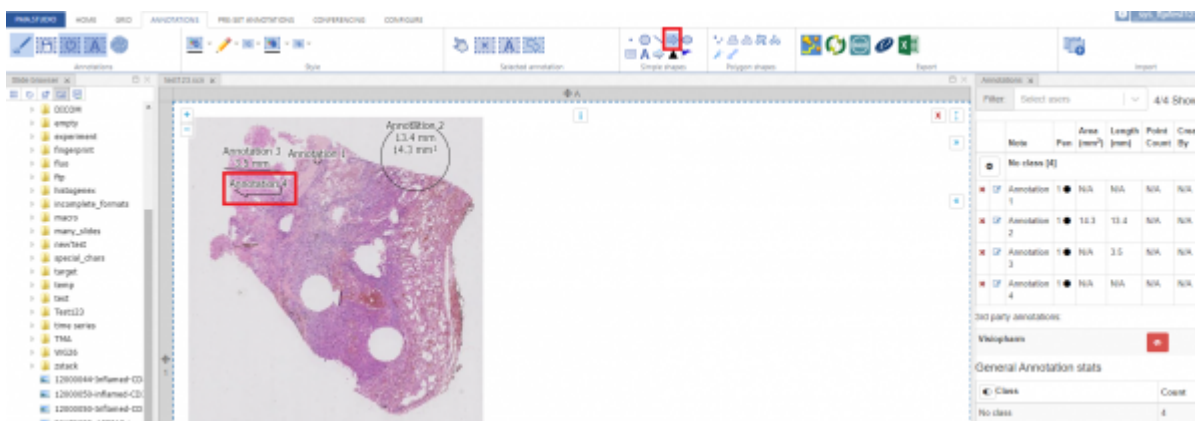

## Ellipse Shape

Next is "Ellipse " shape that allows you to draw ellipse annotation on a slide.

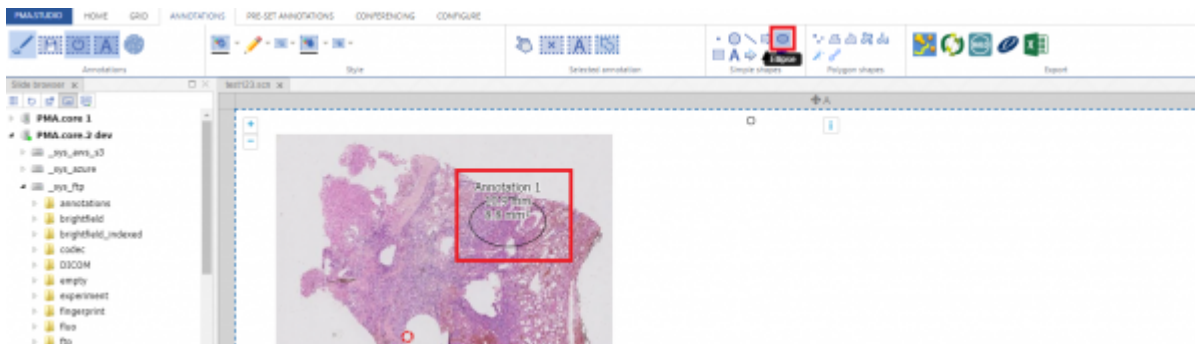


## Rectangular Shape

Next is "Rectangular" shape that allows you to draw rectangular annotation on a slide.




## Landmark 1 Shape

Next is "Landmark 1" shape that allows you to add first of the 3 landmark annotation on a slide.

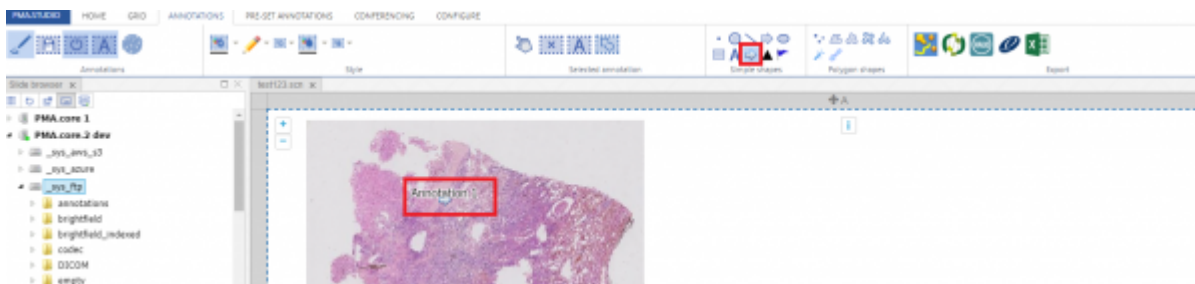


## Landmark 2 Shape

Next is "Landmark 2" shape that allows you to add 2nd of the 3 landmark annotation on a slide.

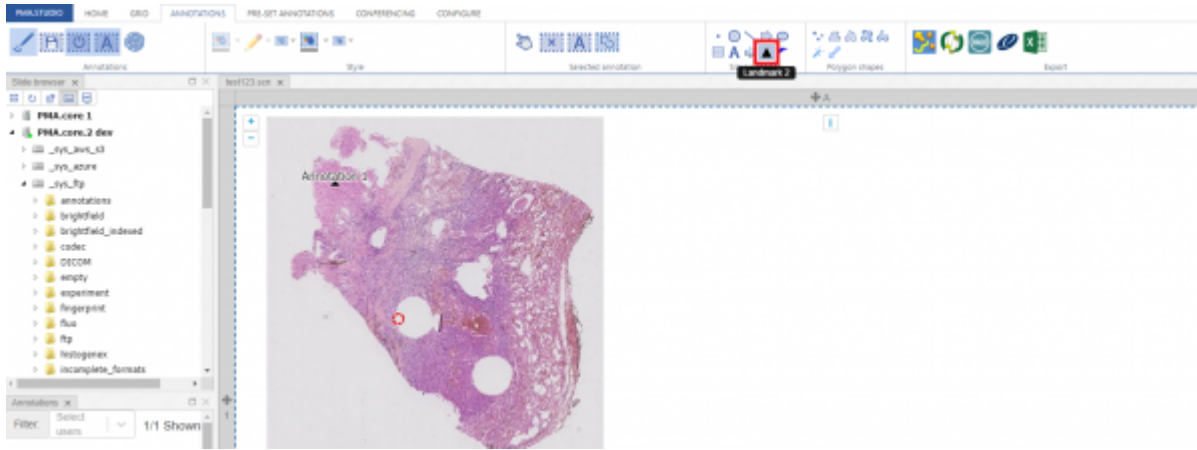

## Landmark 3 Shape

Next is "Landmark 3" shape that allows you to add last of the 3 landmark annotation on a slide.

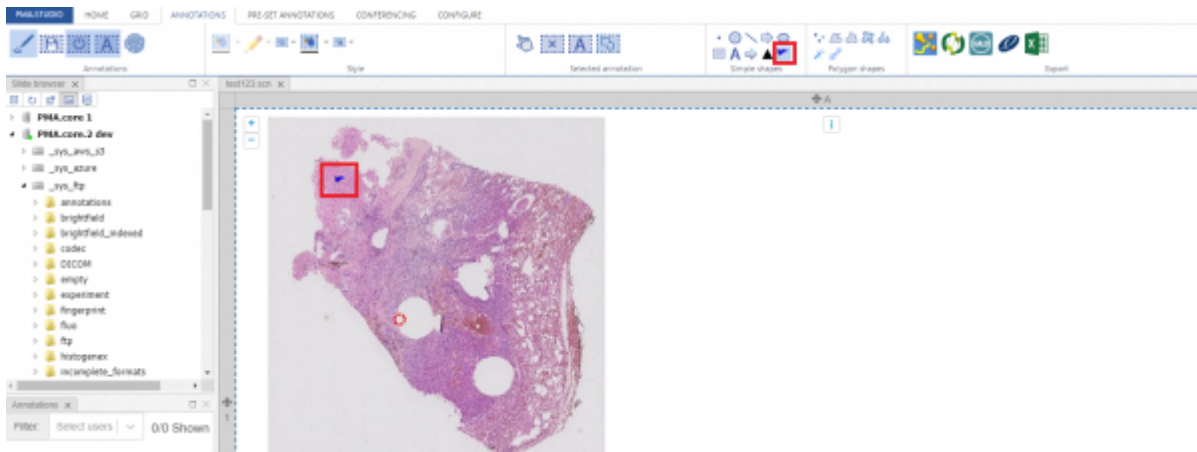

From: <https://docs.pathomation.com/pimsls/3.1.1/> - PIMS LS 3.1.1

Permanent link: [https://docs.pathomation.com/pimsls/3.1.1/doku.php?id=annotations\\_simple&rev=1650555446](https://docs.pathomation.com/pimsls/3.1.1/doku.php?id=annotations_simple&rev=1650555446)

Last update: 2022/04/21 18:37

