

# Context menu

The product indication label in the top-left corner of the PMA.studio and PMA.vue user interface serves as a drop-down menu.



The items in the menu are the same for PMA.studio and PMA.vue.

## About us

This menu option brings up the about dialog with general information about the system.


This menu option is helpful in determining whether you use PMA.vue or PMA.studio, and what version.



If connected to a PMA.core instance and looking at a slide, the information is included in the dialog (as in the example above).

## Show EULA

Pulls up the End-User License Agreement.

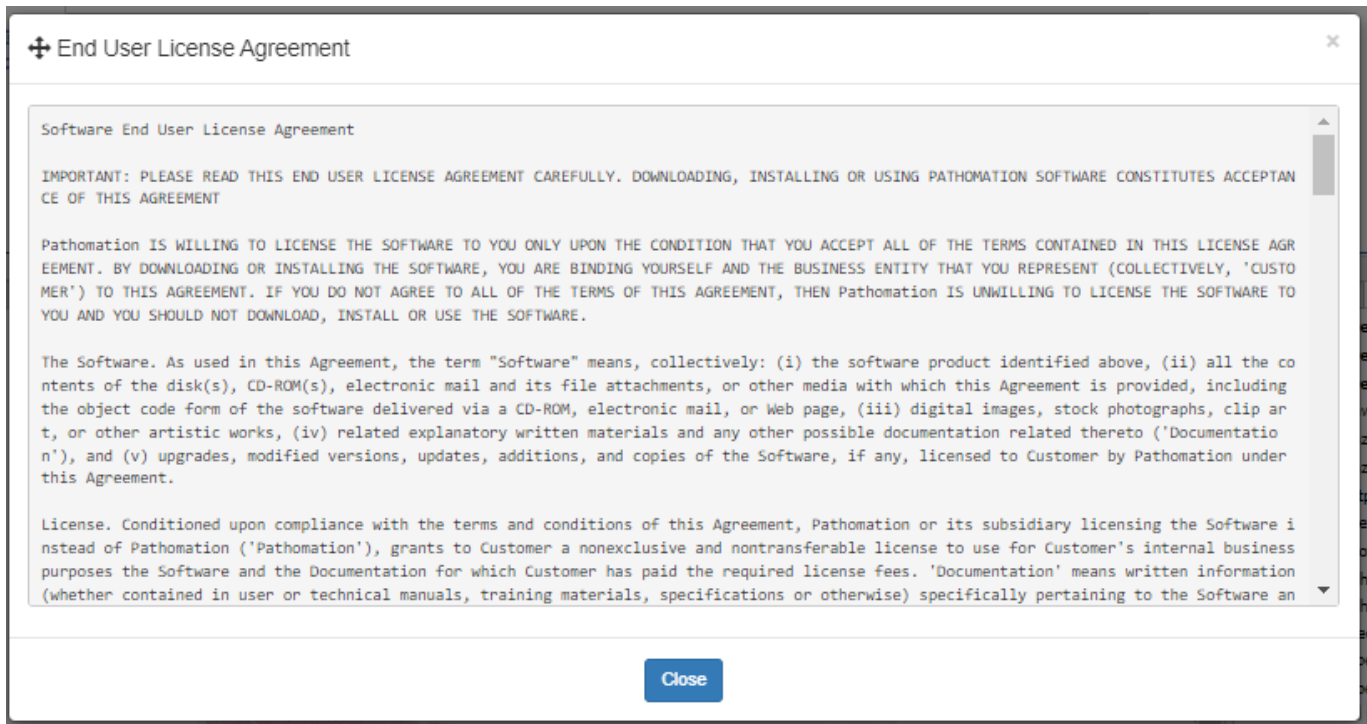


An organization can customize the EULA based on specific needs or use cases (e.g. [RUO](#) related).

### Manual

Takes you this [this website](#).

### Keyboard shortcuts

Brings up available keyboard shortcuts for navigating between slides and within a slide.

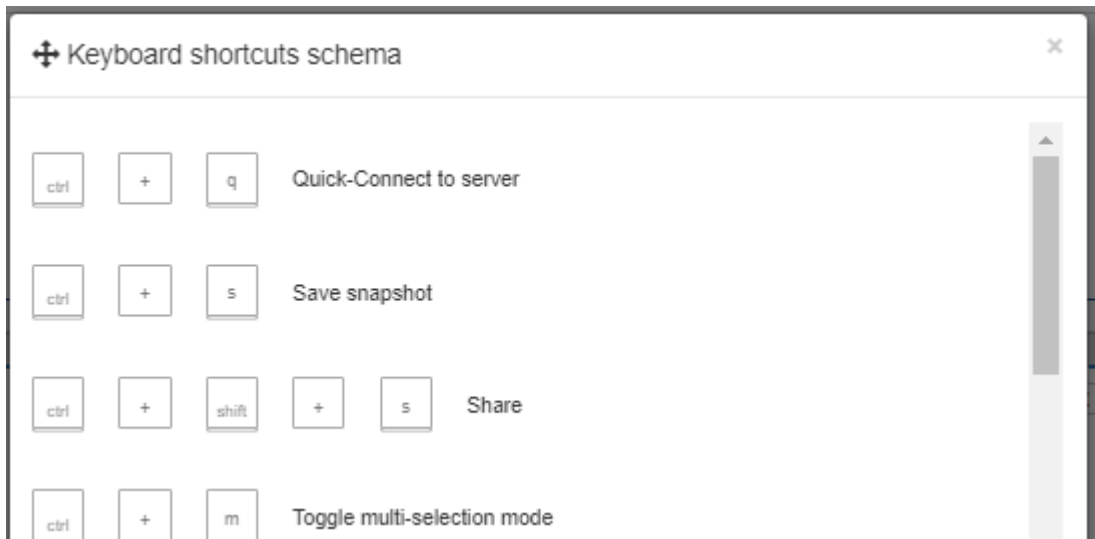

Keyboard shortcuts that were defined in addition to make certain ribbon-options more directly accessible are not included in this list.

## File formats

This menu option takes you to a live website where all currently supported file formats are listed.

### Supported WSI formats

- Number of brightfield formats: 39
- Number of fluorescent formats: 19
- Number of z-stacking formats: 19

| VENDOR / FILE FORMAT | EXTENSION   | COMPRESSION                            | BRIGHTFIELD | FLUORESCENCE | Z-STACKING |
|----------------------|-------------|--|-------------|--------------|------------|
| TIFF                 | .tif, .tiff | JPEG, JPEG2000, LZW, Deflate, Raw, RLE | ✓           |              |            |
| JPEG                 | .jpeg, .jpg | JPEG                                   | ✓           |              |            |
| JPEG 2000            | .jp2        | JPEG2000                               | ✓           |              |            |
| PNG                  | .png        | PNG                                    | ✓           |              |            |
| Olivinus VSI         | .vsi        | lossless JPEG, JPEG, ...               | ✓           | ✓            | ✓          |

The list may slightly differ from the actual file formats that *you* can read, because your [PMA.core tile server](#) may not be the most current version.

## Helpdesk

## Change password

## Edit settings

## Sign out

From: <https://docs.pathomation.com/pimsls/3.1.1/> - PIMS LS 3.1.1

Permanent link: [https://docs.pathomation.com/pimsls/3.1.1/doku.php?id=context\\_menu&rev=1671026184](https://docs.pathomation.com/pimsls/3.1.1/doku.php?id=context_menu&rev=1671026184)

Last update: 2022/12/14 16:56

