


Home: Grid Tool

The Grid dropdown contains tools for clearing the grid, opening new grid tabs, open current folder in grid and popping out selected slide in new window.

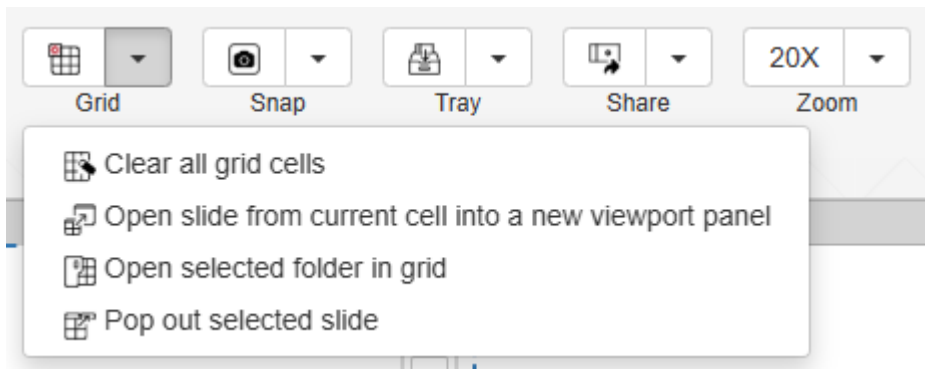

- Clear all grid cells - clears all images from current grid
- Opens current folder in grid - opens the menu below

+ Open in Grid ✕

You are about to open **4** slides in a grid

	Slides in current grid	1	
<input checked="" type="checkbox"/>	Slide		Thumbnail
<input checked="" type="checkbox"/>	_sys_ref/annotations CMU-1.mrxs		
<input checked="" type="checkbox"/>	_sys_ref/annotations CMU-1.ndpi		
<input checked="" type="checkbox"/>	_sys_ref/annotations CMU-1.svs		
<input checked="" type="checkbox"/>	_sys_ref/annotations CMU-1-JP2K-33005.svs		

How do you want to do it?

Clear current grid and open
Append to current grid
Open in new grid
Cancel

- Opens slide from current cell into a new viewport panel
- Pops out selected slide in a new window.
- Clear current cell - clears image from currently selected grid cell

From:

<https://docs.pathomation.com/pimsls/3.1.1/> - **PIMS LS 3.1.1**

Permanent link:

<https://docs.pathomation.com/pimsls/3.1.1/doku.php?id=grid&rev=1747058920>

Last update: **2025/05/12 17:08**

