


# The Grid Tab

The Grid tab is the second tab in the PIMS LS ribbon.

Here you will find tools to create grids of images i.e. view more than one image at a time. It's a very flexible system, allowing you to view anywhere from 2 to 36 images simultaneously - provided your screen is big enough!

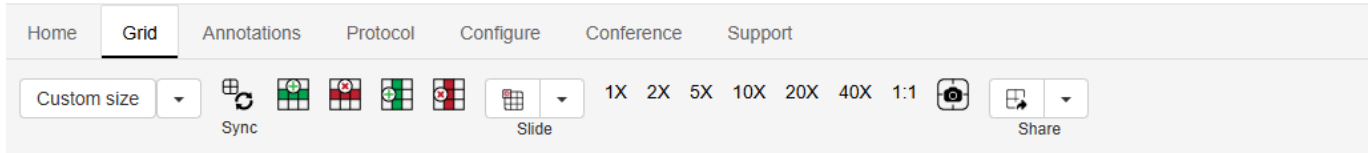

There are dedicated sub articles on the [Size](#), [Synchronize](#) and [Slide](#) tool groups.

The other sections are:

- Sync - [click for more details](#)
- Add row - Adds a new row to the grid on viewport
- Remove row - removes selected row from grid
- Add column - Adds new column to the grid
- Remove column - Removes selected column from the grid
- Slide - [click for more details](#)
- Zooms - Different zoom options for viewport
- Snapshot - Allows taking snapshot of the whole grid
- Share - [slides\\_sharing](#) } [click for more details](#)

From:

<https://docs.pathomation.com/pimsls/3.1.1/> - **PIMS LS 3.1.1**

Permanent link:

<https://docs.pathomation.com/pimsls/3.1.1/doku.php?id=gridtab&rev=1747062353>

Last update: **2025/05/12 18:05**

