

Keyboard Shortcuts

Below you can find a list of Keyboard Shortcuts.

Quick-Connect to server

- ctrl + q

Save snapshot

- ctrl + s

Share

- ctrl + shift + s

Toggle multi-selection mode

- ctrl + m

Decrease zoom

- [

Increase zoom

-]

Select next image

- ctrl + down arrow

Select previous image

- ctrl + up arrow

Select next directory

- ctrl + shift + down arrow

Select previous directory

- ctrl + shift + up arrow

Export Annotations

- ctrl + shift + a

Drag zoom

- shift + drag zoom

Rotate slide

- alt + rotate

From:

<https://docs.pathomation.com/pimsls/3.1.1/> - **PIMS LS 3.1.1**

Permanent link:

https://docs.pathomation.com/pimsls/3.1.1/doku.php?id=keyboard_shortcuts&rev=1734425188

Last update: **2024/12/17 11:46**

