

## Whats new?

### PIMS LS 3.1.1

#### Features

- Added auto navigate to particular folder using link sharing. Previously slide sharing was possible through the url by manually adding parameters to it. This version provides option to generate share link for both ready only and full access control to user slides or folders.
- Introduced vault-based secrets management system that delivers centralized security and configuration management capabilities, including secure configuration binding, encrypted secret storage, access control, secret rotation, and protected retrieval of sensitive application data.

#### Fixes

- Fixed search issues, previous there was message that popped up mentioning “indexing is performed” which did not give user enough information to wait, restart or stop it. Now changes are made to search functionality and user does not has to wait for indexing especially searching file paths.
- There was an issue with form data getting saved but not reflecting in the grid. Issue is fixed now.
- Gallery based navigation was broken in slide browser panel. That is fixed now.

From:

<https://docs.pathomation.com/pimsls/3.1.1/> - **PIMS LS 3.1.1**

Permanent link:

[https://docs.pathomation.com/pimsls/3.1.1/doku.php?id=release\\_notes&rev=1779742274](https://docs.pathomation.com/pimsls/3.1.1/doku.php?id=release_notes&rev=1779742274)

Last update: **2026/05/25 23:51**

