

# Ribbon > Annotations tab > Polygon shapes > Compound freehand

## Button identification




## Associated command

CompoundFreehand

## Function description

Drawing polygon is the typical run off the mill polygon annotation tool you find in many graphics tools like Photoshop, Paint.Net etcetera. The tool, as implemented, assumes there's an automatic connection between your first and last drawn point. You finish drawing your shape by double-clicking.

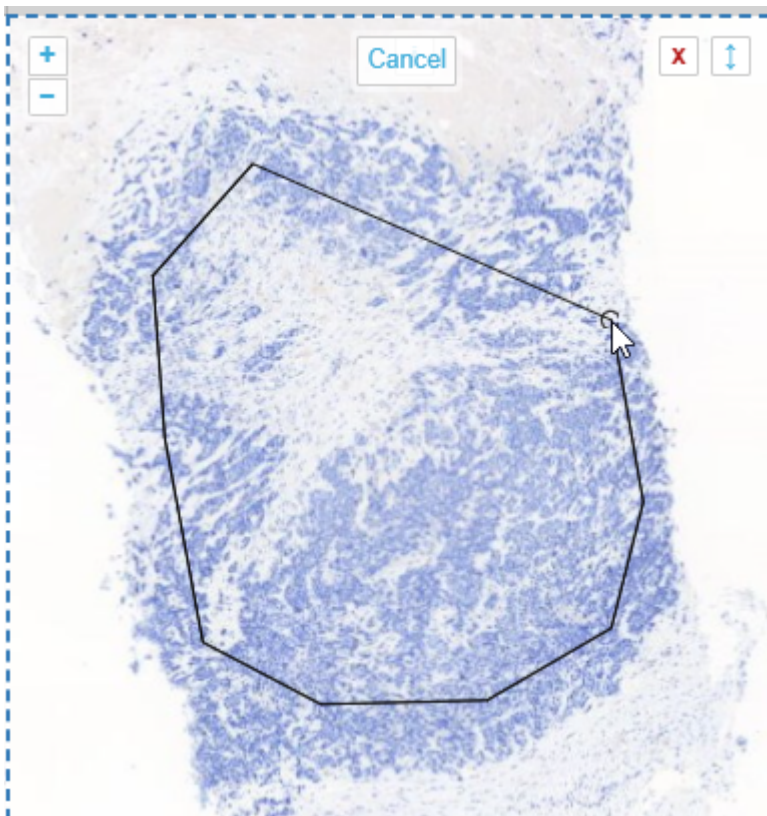

## Related functionality

[Ribbon > Annotations tab](#)

From:  
<https://docs.pathomation.com/pimsls/3.1.1/> - **PIMS LS 3.1.1**

Permanent link:  
[https://docs.pathomation.com/pimsls/3.1.1/doku.php?id=ribbon\\_annotations\\_compound\\_freehand&rev=1677580136](https://docs.pathomation.com/pimsls/3.1.1/doku.php?id=ribbon_annotations_compound_freehand&rev=1677580136)

Last update: **2023/02/28 13:28**

