

Ribbon > Home tab > Viewport group > Apply brightness and contrast

Button identification




Associated command

brightness-contrast-dropdown

Function description

This button brings down a dropdown menu with a number of default settings for brightness and contrast.

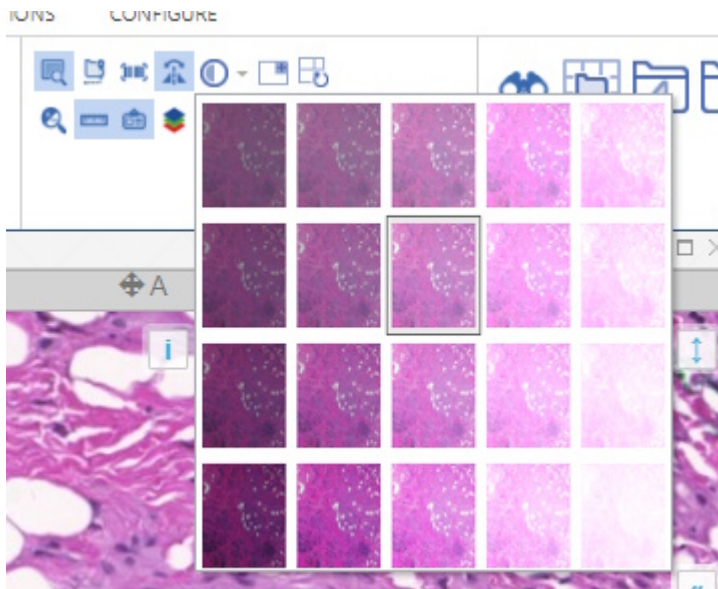

Click the thumbnail to apply the effect that you want. Thumbnails on the left show less brightness, and are brighter on the right. Thumbnails on the top show less contrast, and more contrast on the bottom.

Related functionality

[Ribbon > Home tab](#)

[Color adjustment panel](#)

Last update: 2023/02/10 17:05 ribbon_home_viewport_brightness_contrast https://docs.pathomation.com/pimsls/3.1.1/doku.php?id=ribbon_home_viewport_brightness_contrast

From: <https://docs.pathomation.com/pimsls/3.1.1/> - **PIMS LS 3.1.1**

Permanent link: https://docs.pathomation.com/pimsls/3.1.1/doku.php?id=ribbon_home_viewport_brightness_contrast

Last update: **2023/02/10 17:05**

