


# Grid: Synchronize Tool Group

The Synchronize toolgroup contains tools for turning on synchronised viewing and some grid management tools.

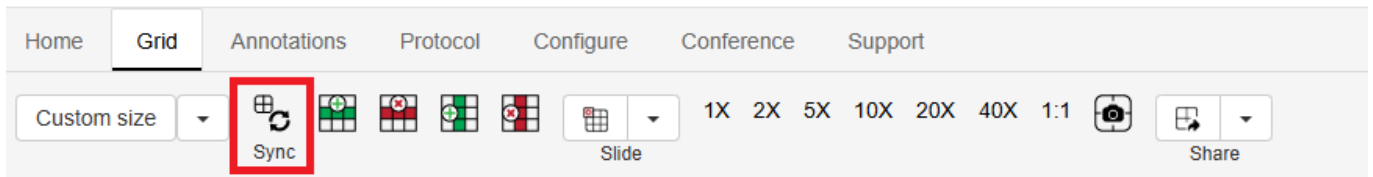

## Synchronized viewing

Turning on the Sync icon when you have 2 or more cells open in your grid will trigger synchronised viewing, you can see a quick video of this below:

The other functions are:

- 1. Clear all grid cells
- 2. Clear current cell
- 3. Open current slide/cell in a new viewport tab in a 1x1 grid

From:  
<https://docs.pathomation.com/pimsls/3.1.1/> - PIMS LS 3.1.1

Permanent link:  
<https://docs.pathomation.com/pimsls/3.1.1/doku.php?id=synchronize&rev=1746904692>

Last update: 2025/05/10 22:18

