


# Grid: Synchronize Tool

The Synchronize tool contains tools for turning on synchronized viewing and some grid management tools.

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
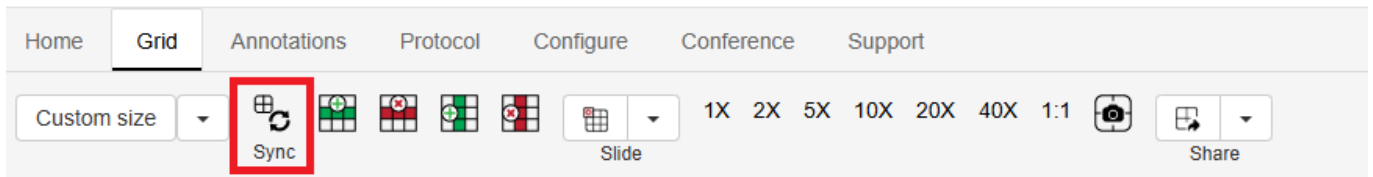


## Synchronized viewing

Turning on the Sync icon when you have 2 or more cells open in your grid will trigger synchronized viewing, you can see a quick video of this below:

## Brightness Contrast

This tool allows user to change brightness and contrast of the opened slide. When user clicks on the dropdown icon, number of different brightness contrast options are shown for selection.

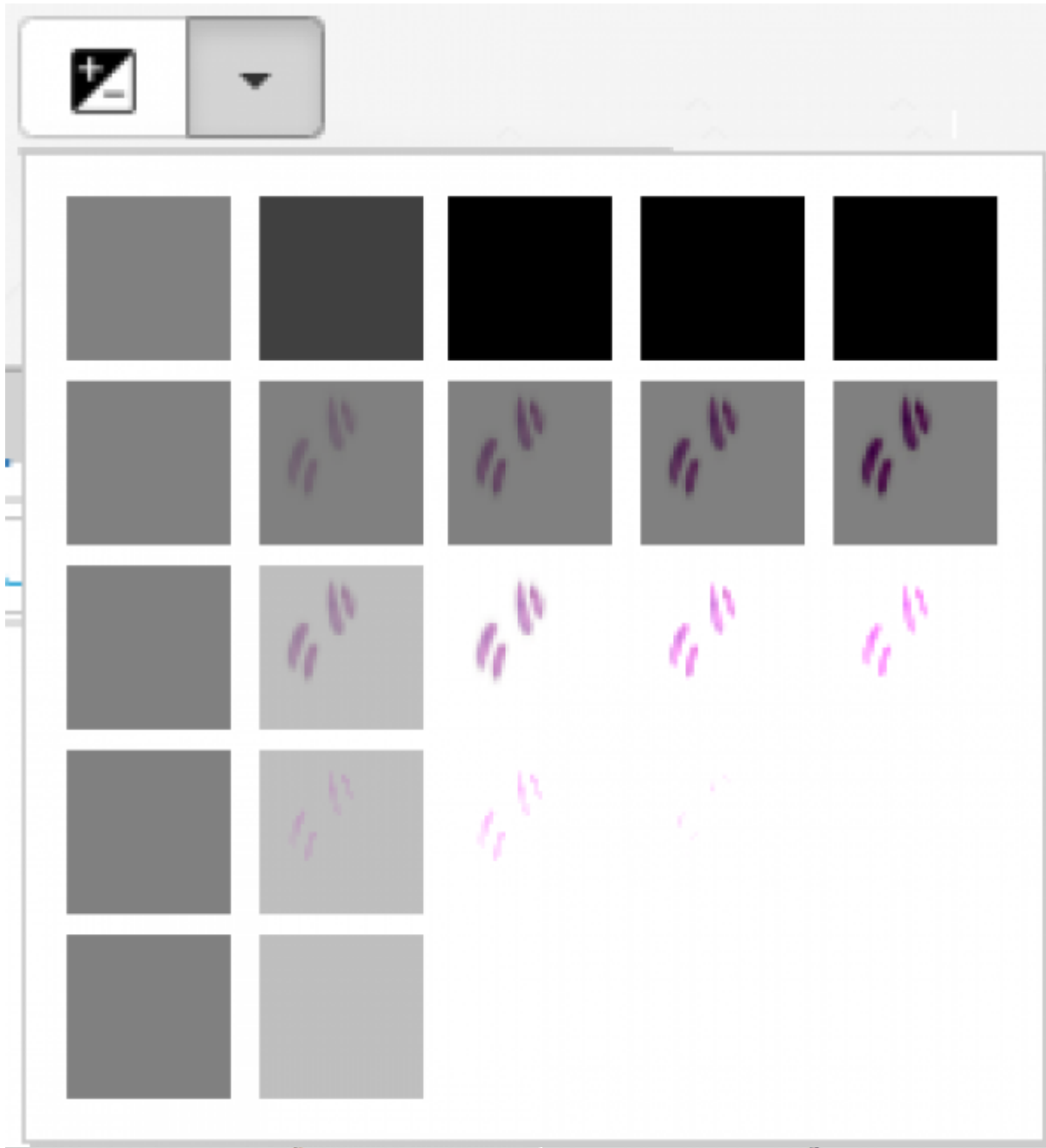

From:  
<https://docs.pathomation.com/pimsls/3.1.1/> - **PIMS LS 3.1.1**

Permanent link:  
<https://docs.pathomation.com/pimsls/3.1.1/doku.php?id=synchronize&rev=1747061479>

Last update: **2025/05/12 17:51**

