

Access control

Forms can be specified to apply to only specific [root-directories](#). Click on the “Form accessibility” button in the forms meta-data view:

Forms

Lookup lists

Name	Version	Description	Forms	Submissions	
Bil trephine biopsy quality score	1		5	53	
Fouls_case	1		7	196	
Fouls_private	1	Further Case Information	1	58	
Fouls_slide	1		3	373	
Gepts_case	1	select case-level data from 1965 Willy Gepts paper	5	22	
Gepts_slide	1	Slide level data	1	138	
Gepts_specimen	1	specimen level data	1	18	

Create new

Form availability

You now see a roster that displays all the available root-directories along the X-axis, along with all available forms on the Y-access.

Form availability overview

Home > Forms > Form availability overview

Form availability overview

Save

Form / Directory	ICON	MOC	Workgroup	_in_aperio	_in_fuo_bd	_in_fuo_nikon	_in_fuo_zeiss	anapath_cases	dcb	fuelt study	fouls	gepts
Gepts_case												
Gepts_specimen												
Gepts_slide												
Fouls_case												
Fouls_slide												
Fouls_private												
Bil trephine biopsy quality score												

Check the form/root combinations that you want to make exclusive, and press the save button in the top-right corner to apply the changes.

When no restrictions are set, the form is available for all root-directories (as is the case with the “BM trephine” form in the screenshot).

From:
<https://docs.pathomation.com/pma.core/2.0.0/> - **PMA.core 2.x**

Permanent link:
https://docs.pathomation.com/pma.core/2.0.0/doku.php?id=access_control&rev=1648558801

Last update: **2022/03/29 16:00**