


## Elmah log

[Elmah error logs](#) are a standard way of capturing runtime errors in software. While these logs are not specific to root directories, they can help troubleshoot intermittent problems. Examples include:

- connection information is (temporarily) no longer correct
- a file format may have been modified by its vendor and now no longer renders correctly

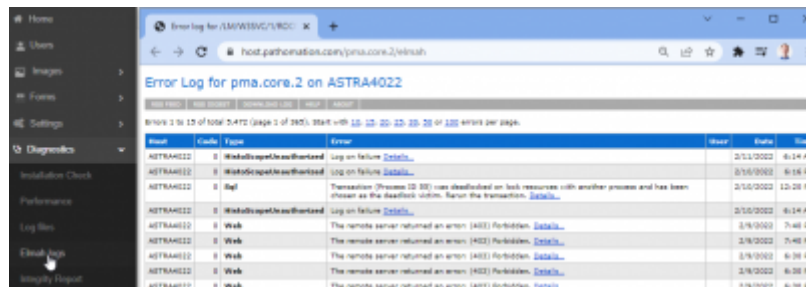

| Host      | Code | Type                 | Error                                                                                                                                    | User | Date     | Time     |
|-----------|------|----------------------|------------------------------------------------------------------------------------------------------------------------------------------|------|----------|----------|
| ASTRA4022 | 0    | WebResourceException | Log on failure                                                                                                                           |      | 2/1/2022 | 6:14 AM  |
| ASTRA4022 | 0    | WebResourceException | Log on failure                                                                                                                           |      | 2/1/2022 | 6:16 AM  |
| ASTRA4022 | 0    | Http                 | Threadpool (Process ID 30) ran out of resources with another process and has been chosen as the default victim. Restart the transaction. |      | 2/1/2022 | 12:28 PM |
| ASTRA4022 | 0    | WebResourceException | Log on failure                                                                                                                           |      | 2/1/2022 | 6:14 AM  |
| ASTRA4022 | 0    | Web                  | The remote server returned an error (400): Forbidden.                                                                                    |      | 2/8/2022 | 7:48 AM  |
| ASTRA4022 | 0    | Web                  | The remote server returned an error (400): Forbidden.                                                                                    |      | 2/8/2022 | 7:48 AM  |
| ASTRA4022 | 0    | Web                  | The remote server returned an error (400): Forbidden.                                                                                    |      | 2/8/2022 | 6:38 AM  |
| ASTRA4022 | 0    | Web                  | The remote server returned an error (400): Forbidden.                                                                                    |      | 2/8/2022 | 6:38 AM  |
| ASTRA4022 | 0    | Web                  | The remote server returned an error (400): Forbidden.                                                                                    |      | 2/8/2022 | 6:38 AM  |

From:

<https://docs.pathomation.com/pma.core/2.0.0/> - **PMA.core 2.x**

Permanent link:

<https://docs.pathomation.com/pma.core/2.0.0/doku.php?id=elmah&rev=1647256401>

Last update: **2022/03/14 14:13**

