


Elmah log

[Elmah error logs](#) are a standard way of capturing runtime errors in software. While these logs are not specific to root directories, they can help troubleshoot intermittent problems. Examples include:

- connection information is (temporarily) no longer correct
- a file format may have been modified by its vendor and now no longer renders correctly

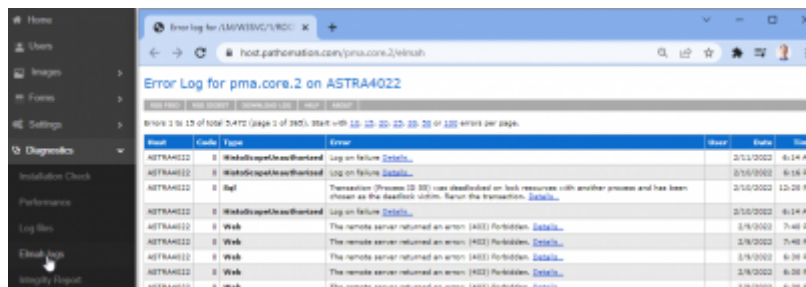

The screenshot shows a web browser displaying the 'Error Log for pma.core.2 on ASTRA4022'. The interface includes a sidebar with navigation options like Home, Users, Images, Forms, Settings, Diagnostics, Installation Check, Performance, Log Box, and Elmah Log (which is selected). The main content area shows a table of error logs with columns for Host, Code, Type, Error, Host, Date, and Time. The table lists several errors, including 'WebResourceUnauthorized', 'Log on failure', 'Transaction (Process ID 30) was deadlocked on lock resources with another process and has been chosen as the deadlock victim. Rerun the transaction.', and 'The remote server returned an error (403): Forbidden.'.

Host	Code	Type	Error	Host	Date	Time
ASTRA4022	0	WebResourceUnauthorized	Log on failure		2/13/2022	6:14 AM
ASTRA4022	0	WebResourceUnauthorized	Log on failure		2/13/2022	6:18 AM
ASTRA4022	0	Log	Transaction (Process ID 30) was deadlocked on lock resources with another process and has been chosen as the deadlock victim. Rerun the transaction.		2/13/2022	12:28 PM
ASTRA4022	0	WebResourceUnauthorized	Log on failure		2/13/2022	6:14 AM
ASTRA4022	0	Web	The remote server returned an error (403): Forbidden.		2/8/2022	7:48 AM
ASTRA4022	0	Web	The remote server returned an error (403): Forbidden.		2/8/2022	7:48 AM
ASTRA4022	0	Web	The remote server returned an error (403): Forbidden.		2/8/2022	6:38 AM
ASTRA4022	0	Web	The remote server returned an error (403): Forbidden.		2/8/2022	6:38 AM
ASTRA4022	0	Web	The remote server returned an error (403): Forbidden.		2/8/2022	6:38 AM

From:

<https://docs.pathomation.com/pma.core/2.0.0/> - **PMA.core 2.x**

Permanent link:

<https://docs.pathomation.com/pma.core/2.0.0/doku.php?id=elmah&rev=1647256409>

Last update: **2022/03/14 14:13**

