

Integrity report

The integrity report section will perform a recursive scan on the selected root directories and report virtual slides that cannot be parsed by the system.

The screenshot shows the 'Integrity report' interface. At the top, there's a breadcrumb 'Home > Integrity Report'. Below it, a section titled 'Root directories to check' contains a table with columns 'Name' and 'Path'. The table has two rows: 'Local images' with path 'E:\storage\images' and 'Shared slides' with path 'C:\MSDCHE'. A 'Start check' button is below the table. The 'Results' section shows a progress bar and the following text: 'Loading directories recursively...', 'Finished loading directories...', 'Checking 57 images...'. Below this, two red error messages are displayed: 'Cannot read Local images\JPK-32003-1.svs' and 'Cannot read Local images\BIP0_10_201211261531.tif'.

Name	Path
Local images	E:\storage\images
Shared slides	C:\MSDCHE

Start check

Results

Loading directories recursively ...
Finished loading directories ...
Checking 57 images ...

Cannot read Local images\JPK-32003-1.svs

Cannot read Local images\BIP0_10_201211261531.tif

You can select one or more root-directories to be scanned at the time.

From:
<https://docs.pathomation.com/pma.core/2.0.0/> - **PMA.core 2.x**

Permanent link:
https://docs.pathomation.com/pma.core/2.0.0/doku.php?id=integrity_report&rev=1591181197

Last update: **2020/06/03 13:46**

