


License setup

First time users

When starting the application for the first time a license will be required.


Close to expiration

When license expiration is less than a month away, a warning message appears:

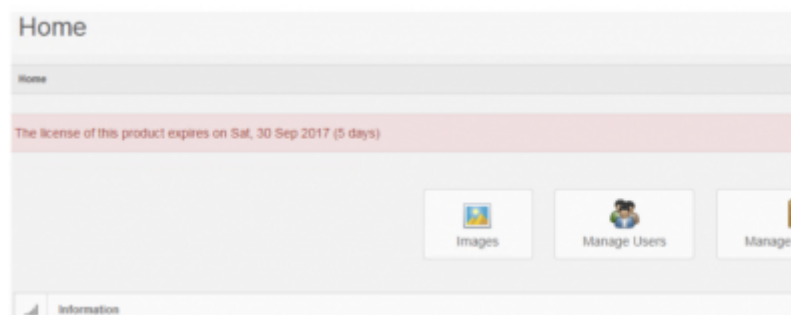


Expired licenses


When the license is expired, you are notified of this right after logging in:

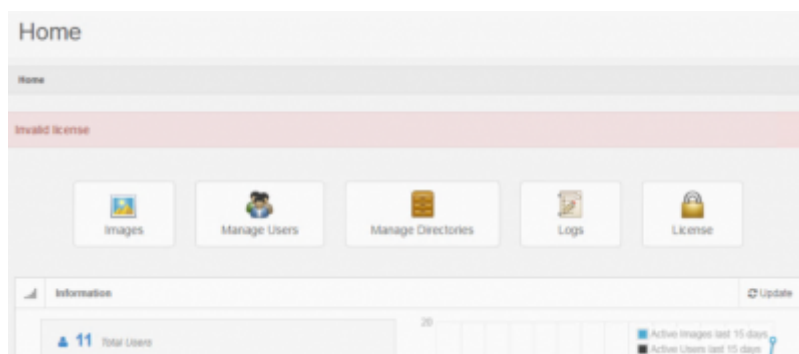


The system will display a section where license information can be found. The user must copy the request code, which can be found in the second tab and send it to the vendor:

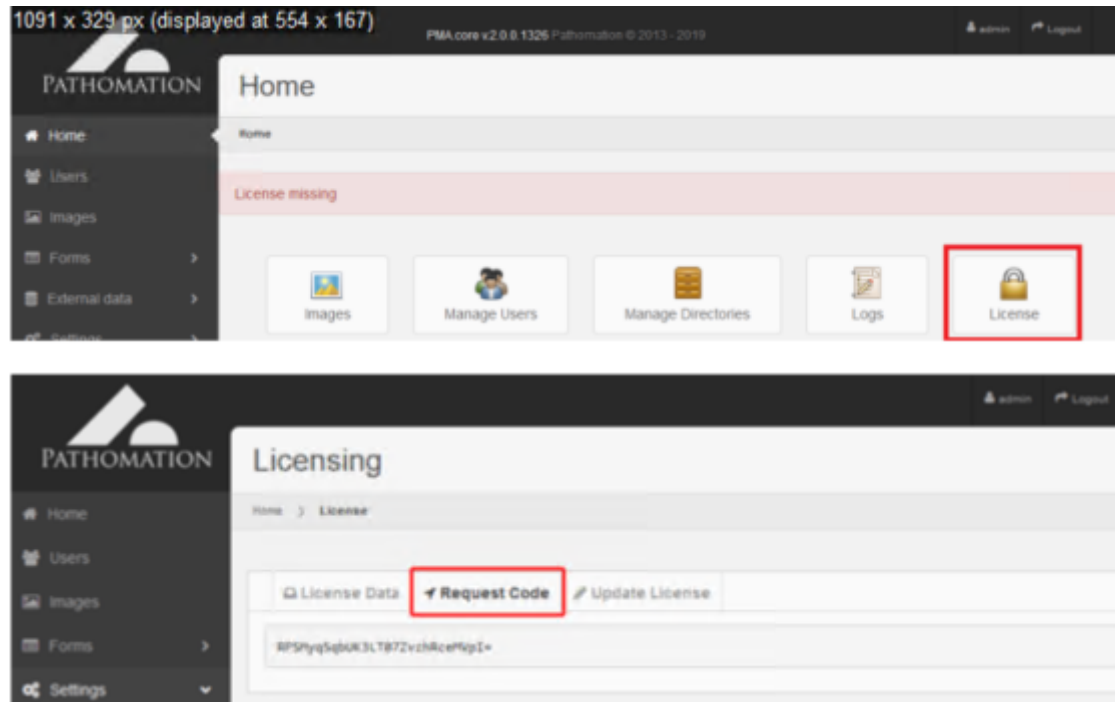

The vendor will provide the user with an appropriate license file, which can be installed from within the third tab:

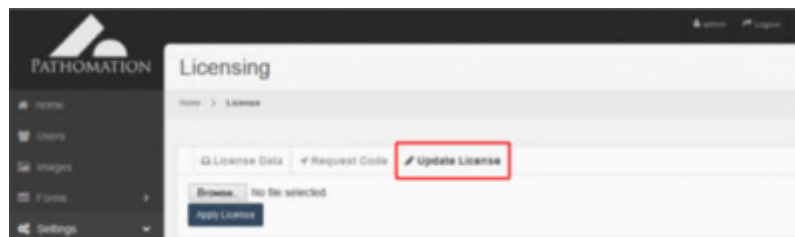

Once the license has been successfully installed, the system will display the appropriate message.

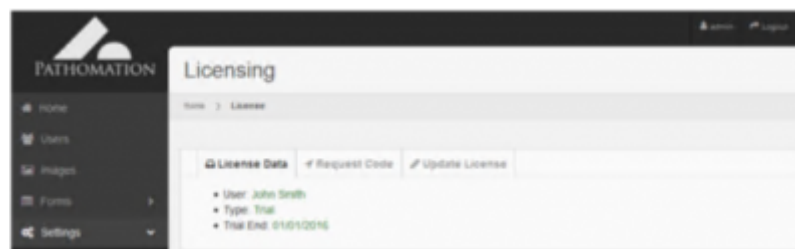

From:
<https://docs.pathomation.com/pma.core/2.0.0/> - **PMA.core 2.x**

Permanent link:
https://docs.pathomation.com/pma.core/2.0.0/doku.php?id=license_setup&rev=1646653379

Last update: **2022/03/07 14:43**

