

# License handling

This wiki entry is about the technical aspects of licensing in PMA.core. Pathomation's [license policy is described on its website](#).


Pathomation has a software licensing scheme that involves the exchange of license files.

Based on the duration of your agreement, we issue you a license file that incorporates parameters like expiration date and number of concurrent roaming seats. The license file is tied to your particular installation of PMA.core and non-transferrable to other instances (this includes virtual machines) or installation folders.


The PMA.core user interface includes everything you need to initiate and update your own installation and licensing scheme.

## Access to license management

You gain access to PMA.core's license management by clicking on the icon from the main management interface:

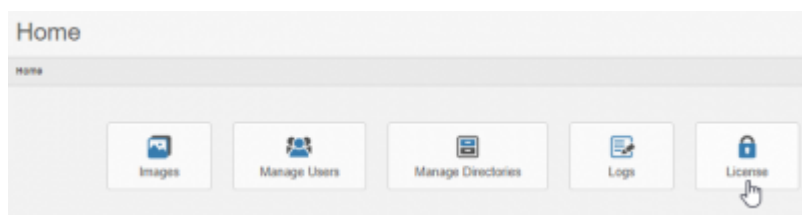


Alternatively, you can access the license management at any time from anywhere in the interface via the menu option in the left navigation menu:

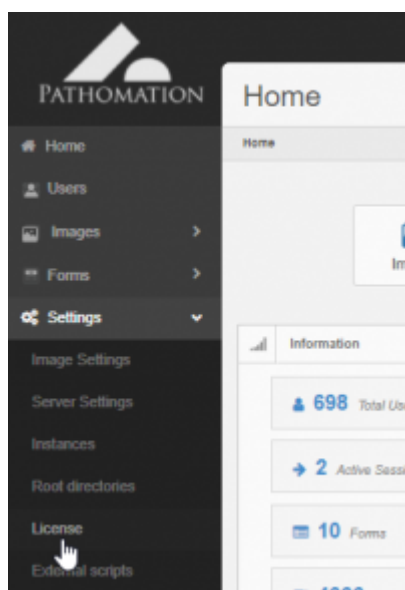


## Missing or expired license

If no license is installed, you'll see an error message immediately after logging in. You should [contact](#)


Pathomation to obtain a [license](#) in this case.

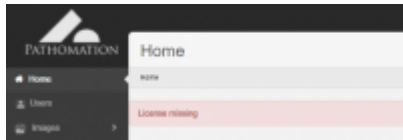

When your license has expired or has become invalid for any other reason, you can still keep using the software. You receive a notification of this after logging in:

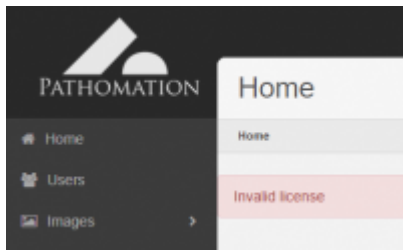

In addition to this administrative message, the tiles served through the tile server show as “faded”. Each tile is also provided with a notification warning, making it rather obvious and that you should [contact Pathomation](#) to resolve whatever issue is the source of the problem.

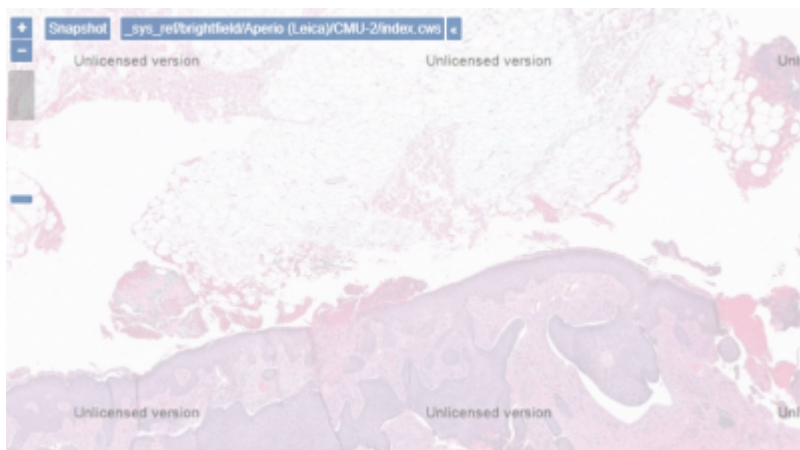

## Exceeded concurrency

From:  
<https://docs.pathomation.com/pma.core/2.0.0/> - **PMA.core 2.x**

Permanent link:  
<https://docs.pathomation.com/pma.core/2.0.0/doku.php?id=licensing&rev=1630492022>

Last update: **2021/09/01 13:27**

