



## Access control

Forms can be specified to apply to only specific [root-directories](#). Click on the “Form accessibility” button in the forms meta-data view:

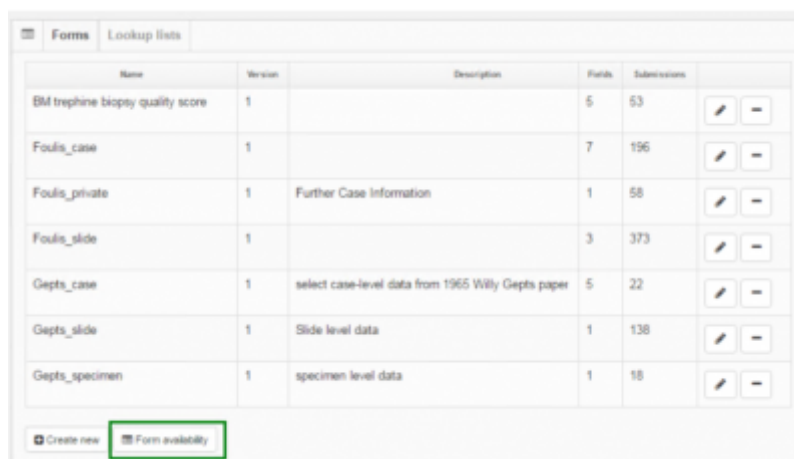

Name	Version	Description	Forms	Submissions	
Bil trephine biopsy quality score	1		5	53	
Fouls_case	1		7	196	
Fouls_private	1	Further Case Information	1	58	
Fouls_slide	1		3	373	
Gepts_case	1	select case-level data from 1965 Willy Gepts paper	5	22	
Gepts_slide	1	Slide level data	1	138	
Gepts_specimen	1	specimen level data	1	18	

You now see a roster that displays all the available root-directories along the X-axis, along with all available forms on the Y-axis.

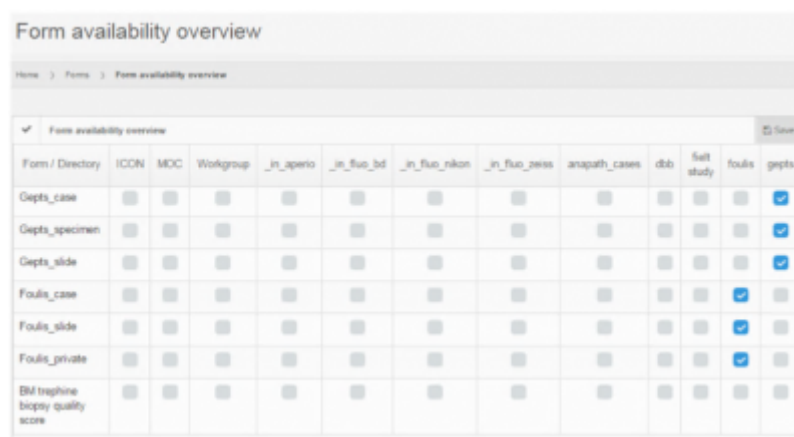

Form / Directory	ICON	MOC	Workgroup	_in_aperio	_in_fluo_bd	_in_fluo_nikon	_in_fluo_zeiss	anapath_cases	dcb	full study	fouls	gepts
Gepts_case	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input checked="" type="checkbox"/>
Gepts_specimen	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input checked="" type="checkbox"/>
Gepts_slide	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input checked="" type="checkbox"/>
Fouls_case	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Fouls_slide	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Fouls_private	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Bil trephine biopsy quality score	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>

Check the form/root combinations that you want to make exclusive, and press the save button in the top-right corner to apply the changes.

When no restrictions are set, the form is available for all root-directories (as is the case with the “BM trephine” form in the screenshot).

From:  
<https://docs.pathomation.com/pma.core/2.0.1/> - **PMA.core 2.x**

Permanent link:  
[https://docs.pathomation.com/pma.core/2.0.1/doku.php?id=access\\_control&rev=1648558801](https://docs.pathomation.com/pma.core/2.0.1/doku.php?id=access_control&rev=1648558801)

Last update: **2022/03/29 16:00**

