


Image settings

In the image settings section image compression and tile related values can be configured.

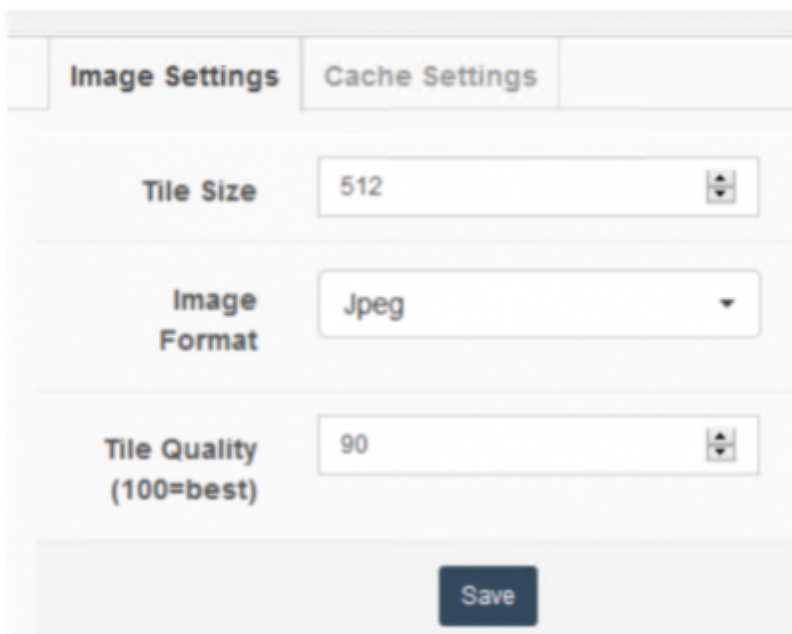

Field	Description
Tile Size	The size of each side of the served tiles in pixels
Image Format	The image format of all the served images (tiles, thumbnails, snapshots etc.). This can be JPEG (lossy compression) or PNG (lossless compression)
Image Quality	1 to 100 value that defines the image quality for lossy compression schemes. Default value is 100 (equal to 100% quality). The parameter does NOT apply when the Image Format is set to PNG. See what impact this has on data traffic.
Image Format Parameters	Format specific parameters that may be required

From: <https://docs.pathomation.com/pma.core/2.0.1/> - PMA.core 2.x

Permanent link: https://docs.pathomation.com/pma.core/2.0.1/doku.php?id=settings_image&rev=1648483205

Last update: 2022/03/28 19:00

