


# Image settings

In the image settings section image compression and tile related values can be configured.

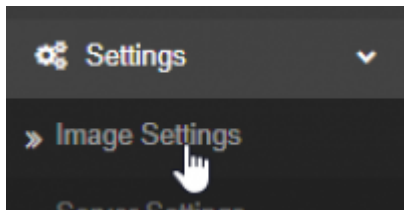

## Image settings

The 'Image Settings' configuration page. It has a breadcrumb 'Home > Image settings'. There are three tabs: 'Image Settings' (selected), 'Cache Settings', and 'Barcode Settings'. The 'Image Settings' tab contains three input fields: 'Tile Size' with the value '512', 'Image Format' with a dropdown menu set to 'Jpeg', and 'Tile Quality (100=best)' with the value '90'. A 'Save' button is at the bottom.

Field	Description
Tile size	The size of each side of the served tiles in pixels. A value of 400 indicates that resulting tiles will be 400 x 400 pixels in size
Image format	The image format for all the served images (tiles, thumbnails, snapshots etc.). This can be JPEG ( <a href="#">lossy compression</a> ) or PNG ( <a href="#">lossless compression</a> )
Tile quality	A value in the 1 to 100 range. This number defines the image quality for lossy compression schemes. Default value is 100 (equal to 100% quality). The parameter does <i>not</i> apply when the image format is set to PNG. <a href="#">See what impact this has on data traffic.</a>

Don't forget to press the "Save" button before navigating away from this page.

## Cache settings

### Cache Settings

Home > Cache settings

Image Settings **Cache Settings** Barcode Settings

Use cache  (cached tiles will be stored in Jpeg format at 90 % quality)

Cache Path

Retention Days

Disk free space required for caching  in GB

Tile Cache 5.71 KB

Cloud Storage Cache 23.38 MB

Drive size 2.73 TB (2.01 TB free)

Total cache size 23.39 MB (0.00% of total size)

Empty cache older than retention days   Empty cache for orphan slides   **Empty all cache**

**Save**

Field	Description
Use cache	...
Cache path	...
Retention days	...
Disk free space required for caching	
Tile cache	...
Cloud storage cache	...
Drive size	...
Total cache size	...

Other menu options:

- Empty cache older than retention days: ...
- Empty cache for orphan slides: ...
- Empty all cache: ...

Don't forget to press the "Save" button before navigating away from this page.

## Barcode settings

# Barcode Settings

Image Settings	Cache Settings	<b>Barcode Settings</b>
<p>Try harder to read barcode texts <input type="checkbox"/></p> <p>Disable barcode scanning <input type="checkbox"/></p> <p><b>Save</b></p>		

From: <https://docs.pathomation.com/pma.core/2.0.1/> - **PMA.core 2.x**

Permanent link: [https://docs.pathomation.com/pma.core/2.0.1/doku.php?id=settings\\_image&rev=1648485073](https://docs.pathomation.com/pma.core/2.0.1/doku.php?id=settings_image&rev=1648485073)

Last update: **2022/03/28 19:31**

