


User monitoring

Overview table


User details view

By clicking on the user's login in the overview table, you can see a summary of the user's settings and activity. This page provides an overview of the user's information and the root directories they can access.

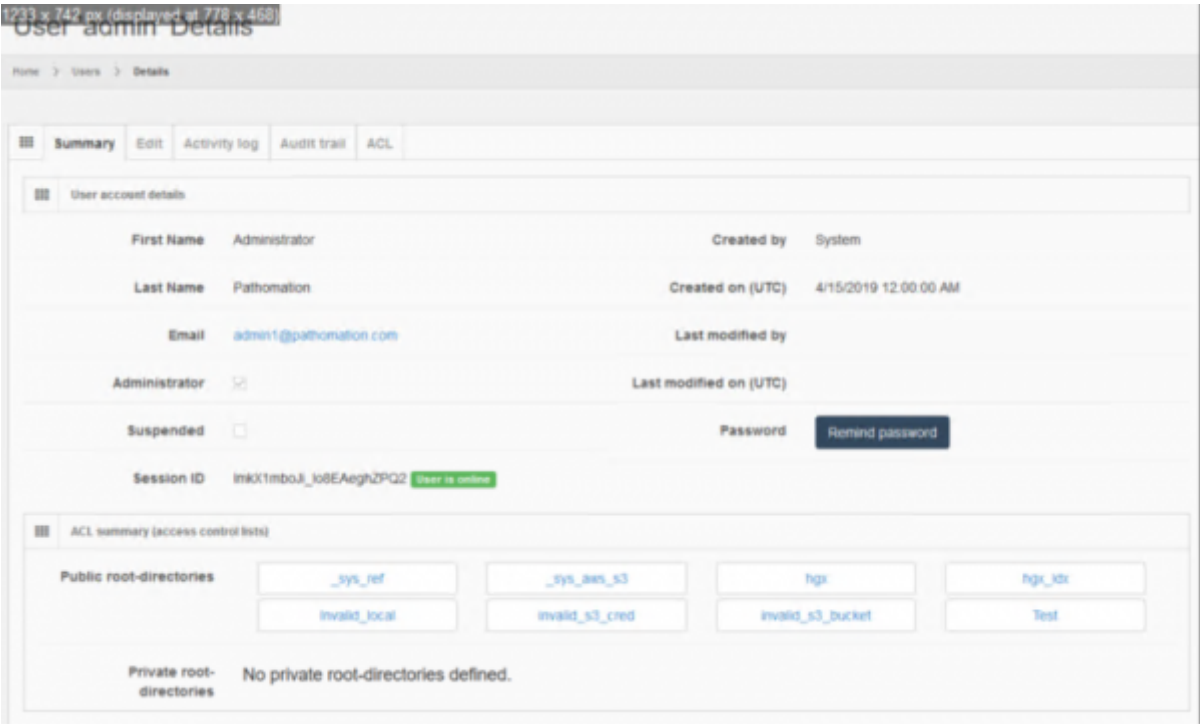

Access rights

The Access rights (ACL) section allows control over the [private root directories](#) the particular user may access. If no private root directories are defined, this list will be empty.

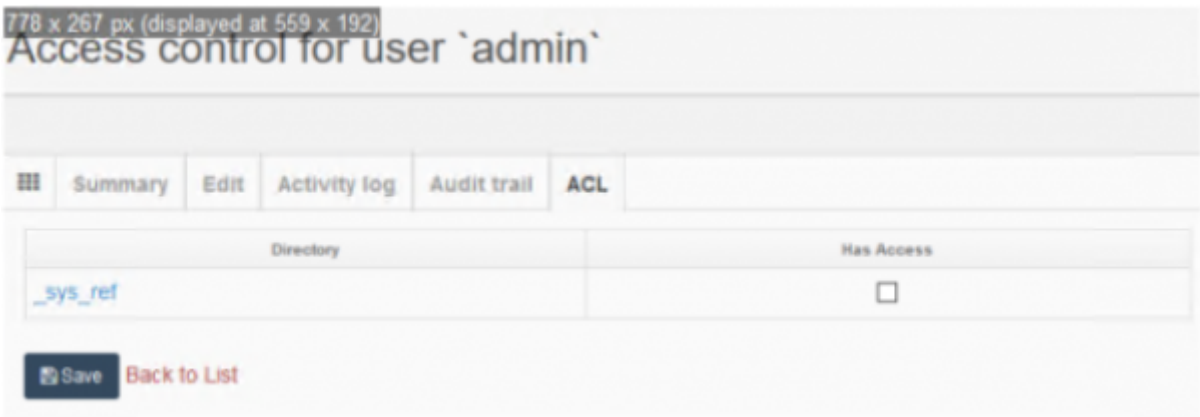

Activity log

The activity log section provides a detailed list of events that involve the displayed user.

1227 x 817 px (displayed at 772 x 514)

User admin Log

Home > Users > Log

Summary Edit **Activity log** Audit trail ACL

System Access Log

Session ID	First Accessed	Last Accessed	Host Address	Host Name	Application	Success	Failure Reason
lnkX1mboJl_lo8EAeghZPQ2	4/15/2019, 9:50:49 AM	4/15/2019, 2:15:31 PM	85.73.248.94	85.73.248.94	PMA core	true	
	4/15/2019, 4:34:35 AM	4/15/2019, 4:34:35 AM	127.0.0.1	127.0.0.1	test	false	Invalid username or password
upjGsSqVVKQpkiH268MA2	4/15/2019, 4:32:52 AM	4/15/2019, 9:45:22 AM	85.73.248.94	85.73.248.94	PMA core	true	

Showing 1 to 3 of 3 entries

Previous 1 Next

Slide Access Log

First Accessed	Last Accessed	Session ID	Slide Path	Slide Name
4/15/2019, 12:49:17 PM	4/15/2019, 2:15:32 PM	lnkX1mboJl_lo8EAeghZPQ2	._sys_ref/brightfield/Smartzoom	cmu-1.5zi
4/15/2019, 12:48:25 PM	4/15/2019, 2:15:32 PM	lnkX1mboJl_lo8EAeghZPQ2	._sys_ars_s3/brightfield/Hanamatsu/CMU-1-VMS	CMU-1-40x - 2010-01-12 13:24:05 vms
4/15/2019, 12:47:59 PM	4/15/2019, 2:15:32 PM	lnkX1mboJl_lo8EAeghZPQ2	._sys_ref/brightfield/Hanamatsu/CMU-1-VMS	CMU-1-40x - 2010-01-12 13:24:05 vms
4/15/2019, 12:47:57 PM	4/15/2019, 2:15:32 PM	lnkX1mboJl_lo8EAeghZPQ2	._sys_ars_s3/brightfield/Generic_Png	png01_julia_4K.png
4/15/2019, 12:47:54 PM	4/15/2019, 2:15:32 PM	lnkX1mboJl_lo8EAeghZPQ2	._sys_ref/brightfield/Generic_Png	png01_julia_4K.png

Audit trail

The audit trail section displays a log of modification events that involve the current user.

1230 x 471 px (displayed at 854 x 327)

Audit trail for user: test

Home > Users > Audit trail

Summary Edit Activity log **Audit trail** ACL

Audit trail

EntityId	EventId	ActionType	EventDate	UserLogin	UserHost	Administrator	CanAnnotate	Description	Email	FirstName	LastName
0	174	Update	4/15/2019, 4:34:15 PM	admin	85.73.248.94	False	True	Test user	test123@pathomation.com	Test	test
0	173	Create	4/15/2019, 4:34:00 PM	admin	85.73.248.94	False	False	Test user	test123@pathomation.com	Test	test

Showing 1 to 2 of 2 entries

Previous 1 Next

From:

<https://docs.pathomation.com/pma.core/2.0.1/> - **PMA.core 2.x**

Permanent link:

https://docs.pathomation.com/pma.core/2.0.1/doku.php?id=user_monitoring&rev=1646672364

Last update: **2022/03/07 19:59**

