



## Access control

Forms can be specified to apply to only specific [root-directories](#). Click on the “Form accessibility” button in the forms meta-data view:

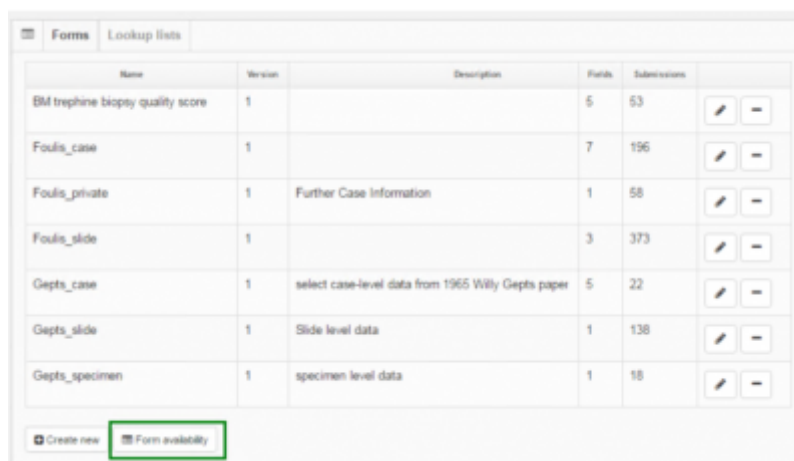

| Name                              | Version | Description                                        | Forms | Submissions |  |
|-----------------------------------|---------|----------------------------------------------------|-------|-------------|--|
| Bil trephine biopsy quality score | 1       |                                                    | 5     | 53          |  |
| Fouls_case                        | 1       |                                                    | 7     | 196         |  |
| Fouls_private                     | 1       | Further Case Information                           | 1     | 58          |  |
| Fouls_slide                       | 1       |                                                    | 3     | 373         |  |
| Gepts_case                        | 1       | select case-level data from 1965 Willy Gepts paper | 5     | 22          |  |
| Gepts_slide                       | 1       | Slide level data                                   | 1     | 138         |  |
| Gepts_specimen                    | 1       | specimen level data                                | 1     | 18          |  |

You now see a roster that displays all the available root-directories along the X-axis, along with all available forms on the Y-axis.

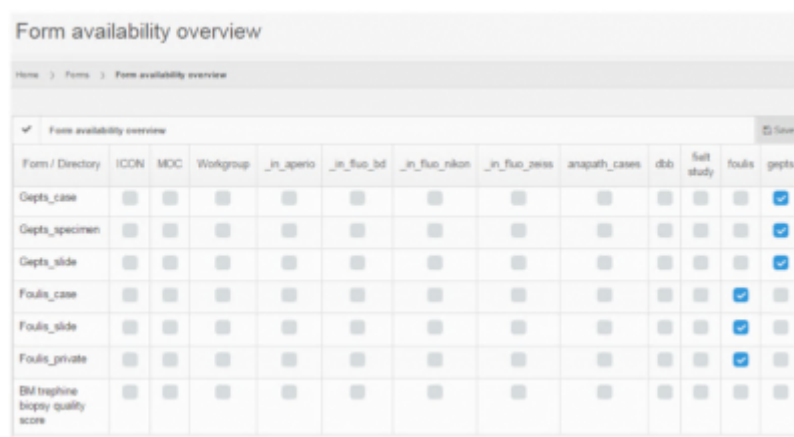

| Form / Directory                  | ICON                     | MOC                      | Workgroup                | _in_aperio               | _in_fluo_bd              | _in_fluo_nikon           | _in_fluo_zeiss           | anapath_cases            | dcb                      | flut study                          | fouls                    | gepts                               |
|-----------------------------------|--------------------------|--------------------------|--------------------------|--------------------------|--------------------------|--------------------------|--------------------------|--------------------------|--------------------------|-------------------------------------|--------------------------|-------------------------------------|
| Gepts_case                        | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/>            | <input type="checkbox"/> | <input checked="" type="checkbox"/> |
| Gepts_specimen                    | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/>            | <input type="checkbox"/> | <input checked="" type="checkbox"/> |
| Gepts_slide                       | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/>            | <input type="checkbox"/> | <input checked="" type="checkbox"/> |
| Fouls_case                        | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input checked="" type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/>            |
| Fouls_slide                       | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input checked="" type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/>            |
| Fouls_private                     | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input checked="" type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/>            |
| Bil trephine biopsy quality score | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/>            | <input type="checkbox"/> | <input type="checkbox"/>            |

Check the form/root combinations that you want to make exclusive, and press the save button in the top-right corner to apply the changes.

When no restrictions are set, the form is available for all root-directories (as is the case with the “BM trephine” form in the screenshot).

From:

<https://docs.pathomation.com/pma.core/2.0.2/> - **PMA.core 2.x**

Permanent link:

[https://docs.pathomation.com/pma.core/2.0.2/doku.php?id=access\\_control](https://docs.pathomation.com/pma.core/2.0.2/doku.php?id=access_control)

Last update: **2022/03/29 16:00**

