

Access control

Forms can be specified to apply to only specific root-directories. Click on the “Form accessibility” button in the forms meta-data view:

Name	Version	Description	Fields	Submissions	
BM trephine biopsy quality score	1		5	53	
Fouls_case	1		7	196	
Fouls_private	1	Further Case Information	1	58	
Fouls_slide	1		3	373	
Gepts_case	1	select case-level data from 1965 Willy Gepts paper	5	22	
Gepts_slide	1	Slide level data	1	138	
Gepts_specimen	1	specimen level data	1	18	

Create new **Form availability**

You now see a roster that displays all the available root-directories along the X-axis, along with all available forms on the Y-access.

Form / Directory	ICON	MOC	Workgroup	_in_aperio	_in_fuo_bld	_in_fuo_nikon	_in_fuo_zeiss	anapath_cases	dcb	full study	fouls	gepts
Gepts_case	☐	☐	☐	☐	☐	☐	☐	☐	☐	☐	☐	☒
Gepts_specimen	☐	☐	☐	☐	☐	☐	☐	☐	☐	☐	☐	☒
Gepts_slide	☐	☐	☐	☐	☐	☐	☐	☐	☐	☐	☐	☒
Fouls_case	☐	☐	☐	☐	☐	☐	☐	☐	☐	☒	☐	☐
Fouls_slide	☐	☐	☐	☐	☐	☐	☐	☐	☐	☒	☐	☐
Fouls_private	☐	☐	☐	☐	☐	☐	☐	☐	☐	☒	☐	☐
BM trephine biopsy quality score	☐	☐	☐	☐	☐	☐	☐	☐	☐	☐	☐	☐

Save

Check the form/root combinations that you want to make exclusive, and press the save button in the top-right corner to apply the changes.

When no restrictions are set, the form is available for all root-directories (as is the case with the “BM trephine” form in the screenshot).

From:
<https://docs.pathomation.com/pma.core/2.0.2/> - **PMA.core 2.x**

Permanent link:
https://docs.pathomation.com/pma.core/2.0.2/doku.php?id=access_control&rev=1591180751

Last update: **2020/06/03 13:39**

