

Cache settings

In the cache settings section the cache strategy of the system can be defined.

702 x 560 px (displayed at 674 x 474)

Image Settings **Cache Settings**

Use cache	<input checked="" type="checkbox"/> (cached tiles will be stored in jpeg format at 90 % quality)
Cache Path	<input type="text" value="~/App_Data/Cache"/>
Retention Days	<input type="text" value="365"/>
Tile Cache	3.22 GB
S3 Cache	5.7 GB
Total cache size	8.93 GB

Field	Description
Use cache	Indicates whether or not the system caches requested tiles. By default, caching is turned off.
Cache Path	The directory where cache files are stored. This is a web application relative path.
Retention Days	The number of days after which a cache file is automatically erased by the system.
Tile Cache	Indication of current space consumption of cached tiles.
S3 Cache	Indication of current space consumption of cached chunks of slides that reside on S3 storage.
Total cache size	Accumulated cache space consumption.

Operations:

Operation	Description
Empty cache older than retention days	Erases cache files that haven't been consulted recently.
Empty cache for orphan slides	Erases cache files for slides that do not exist anymore.
Empty all cache	Erases all cache files.
Save	Saves the cache settings.

From:
<https://docs.pathomation.com/pma.core/2.0.2/> - **PMA.core 2.x**

Permanent link:
https://docs.pathomation.com/pma.core/2.0.2/doku.php?id=cache_settings&rev=1591104252

Last update: **2020/06/02 16:24**

