

Cache settings

In the cache settings section the cache strategy of the system can be defined.

702 x 560 px (displayed at 674 x 474)

Image Settings

Cache Settings

Use cache

☒ (cached tiles will be stored in jpeg format at 90 % quality)

Cache Path

Retention Days

Tile Cache

3.22 GB

S3 Cache

5.7 GB

Total cache size

8.93 GB

Empty cache older than retention days

Empty cache for orphan slides

Empty all cache

Save

Field	Description
Use cache	Indicates whether or not the system caches requested tiles. By default, caching is turned off.
Cache path	The directory where cache files are stored. This is a web application relative path
Retention days	The number of days after which a cache file is automatically erased by the system
Tile cache	Indication of current space consumption of cached tiles
S3 cache	Indication of current space consumption of cached chunks of slides that reside on S3 storage
Total cache size	Accumulated cache space consumption

Operations

Operation	Description
Empty cache older than retention days	Erase cache files that haven't been consulted recently
Empty cache for orphan slides	Erases cache files for slides that do not exist anymore
Empty all cache	Erases all cache files
Save	Saves the cache settings

Field	Description
Use cache	Indicates whether or not the system caches requested tiles. By default, caching is turned off.
Cache Path	The directory where cache files are stored. This is a web application relative path.
Retention Days	The number of days after which a cache file is automatically erased by the system.
Tile Cache	Indication of current space consumption of cached tiles.
S3 Cache	Indication of current space consumption of cached chunks of slides that reside on S3 storage.
Total cache size	Accumulated cache space consumption.

Operations:

Operation	Description
Empty cache older than retention days	Erases cache files that haven't been consulted recently.
Empty cache for orphan slides	Erases cache files for slides that do not exist anymore.
Empty all cache	Erases all cache files.
Save	Saves the cache settings.

From:  
<https://docs.pathomation.com/pma.core/2.0.2/> - **PMA.core 2.x**

Permanent link:  
[https://docs.pathomation.com/pma.core/2.0.2/doku.php?id=cache\\_settings&rev=1647856859](https://docs.pathomation.com/pma.core/2.0.2/doku.php?id=cache_settings&rev=1647856859)

Last update: **2022/03/21 13:01**

