


Elmah log

[Elmah error logs](#) are a standard way of capturing runtime errors in software. While these logs are not specific to root directories, they can help troubleshoot intermittent problems. Examples include:

- connection information is (temporarily) no longer correct
- a file format may have been modified by its vendor and now no longer renders correctly

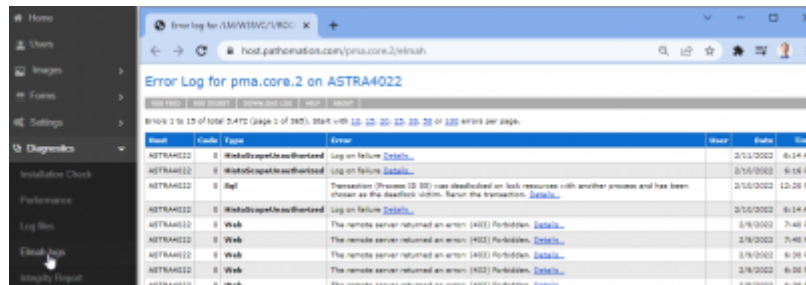

The screenshot shows a web browser displaying the 'Error Log for pma.core.2 on ASTRA4022'. The interface includes a sidebar with navigation options like Home, Users, Images, Forms, Settings, Diagnostics, and a main content area with a table of error logs. The table has columns for Host, Code, Type, Error, Host, Date, and File. The errors listed include 'WebResourceUnauthorized', 'Log on failure', 'Transaction (Process ID 39) was deadlocked on lock resources with another process and has been chosen as the deadlock victim. Rerun the transaction.', and 'The remote server returned an error (403): Forbidden.'.

Host	Code	Type	Error	Host	Date	File
ASTRA4022	0	WebResourceUnauthorized	Log on failure	ASTRA4022	2/13/2022	6:14 AM
ASTRA4022	0	WebResourceUnauthorized	Log on failure	ASTRA4022	2/13/2022	6:18 AM
ASTRA4022	0	Sql	Transaction (Process ID 39) was deadlocked on lock resources with another process and has been chosen as the deadlock victim. Rerun the transaction.	ASTRA4022	2/13/2022	12:28 PM
ASTRA4022	0	WebResourceUnauthorized	Log on failure	ASTRA4022	2/13/2022	6:14 AM
ASTRA4022	0	Web	The remote server returned an error (403): Forbidden.	ASTRA4022	2/8/2022	7:48 AM
ASTRA4022	0	Web	The remote server returned an error (403): Forbidden.	ASTRA4022	2/8/2022	7:48 AM
ASTRA4022	0	Web	The remote server returned an error (403): Forbidden.	ASTRA4022	2/8/2022	6:38 AM
ASTRA4022	0	Web	The remote server returned an error (403): Forbidden.	ASTRA4022	2/8/2022	6:38 AM
ASTRA4022	0	Web	The remote server returned an error (403): Forbidden.	ASTRA4022	2/8/2022	6:38 AM

From:

<https://docs.pathomation.com/pma.core/2.0.2/> - **PMA.core 2.x**

Permanent link:

<https://docs.pathomation.com/pma.core/2.0.2/doku.php?id=elmah&rev=1647256409>

Last update: **2022/03/14 14:13**

