


Integrity report

The integrity report section will perform a recursive scan on the selected root directories and report virtual slides that cannot be parsed by the system.

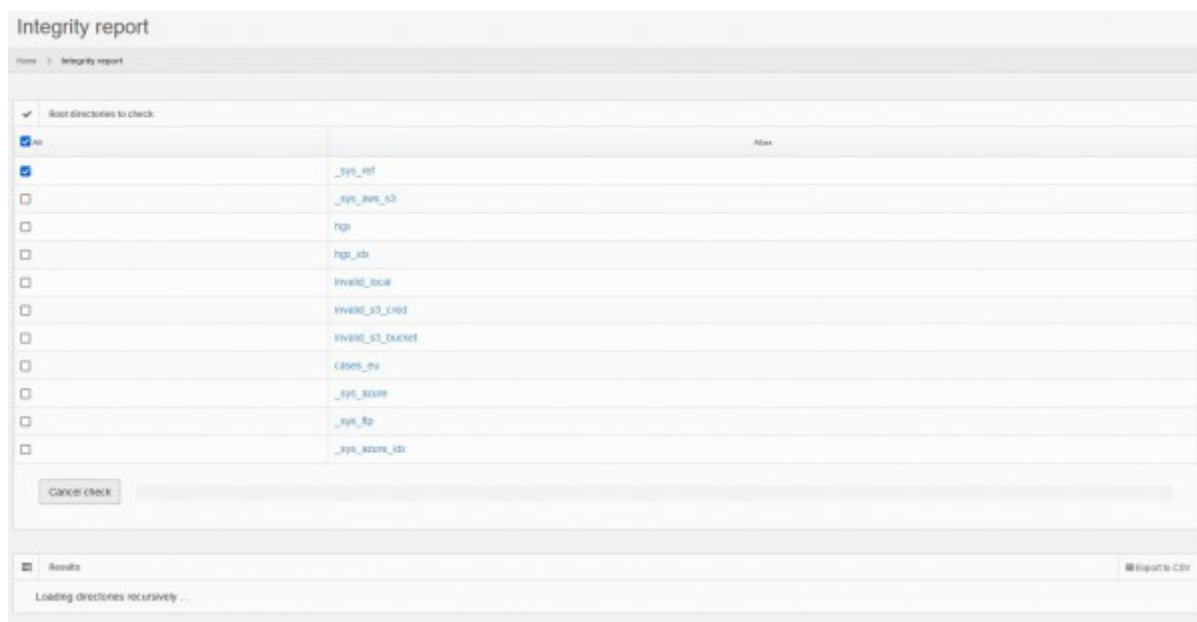

The screenshot shows the 'Integrity report' interface. At the top, there's a breadcrumb 'Home > Integrity report'. Below it, a section titled 'Select directories to check:' contains a table with columns for checkboxes, directory names, and an 'Alias' column. The table lists several directories, with the first one selected. Below the table is a 'Cancel check' button. At the bottom, there's a 'Results' section with a 'Loading directories recursively ...' status and an 'Export to CSV' button.

		Alias
☒	...	
☒	..._sys_ref	
☐	..._sys_refs_55	
☐	hgs	
☐	hgs_idx	
☐	invald_local	
☐	invald_53_crd	
☐	invald_53_bucket	
☐	cases_ev	
☐	..._sys_ref	
☐	..._sys_ref	
☐	..._sys_ref_idx	

To perform an integrity check you simply

- select one or more root-directories to be scanned at the time
- Click *Start Check*

At the bottom of the page, the result list will be populated with any errors as soon as PMA.core finds them. You can also click the **Export To CSV** button to download the list of errors in CSV format

From:

<https://docs.pathomation.com/pma.core/2.0.2/> - **PMA.core 2.x**

Permanent link:

https://docs.pathomation.com/pma.core/2.0.2/doku.php?id=integrity_report

Last update: **2022/04/04 12:04**

