


# License setup

## First time users

When starting the application for the first time a license will be required.


## Close to expiration

When license expiration is less than a month away, a warning message appears:

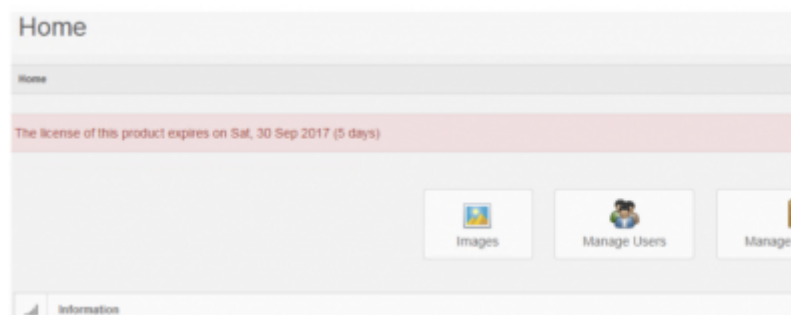


## Expired licenses


When the license is expired, you are notified of this right after logging in:

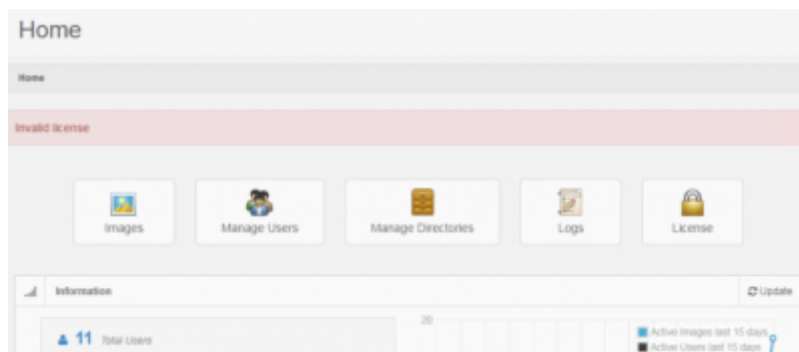


The system will display a section where license information can be found. The user must copy the request code, which can be found in the second tab and send it to the vendor:

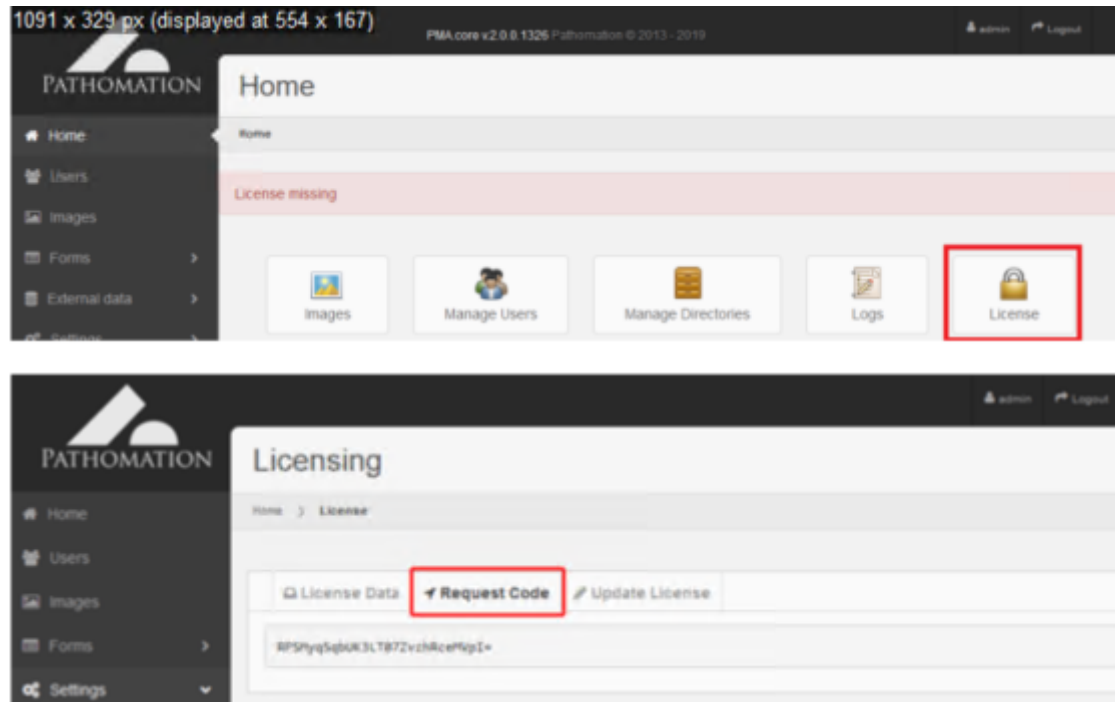

The vendor will provide the user with an appropriate license file, which can be installed from within the third tab:

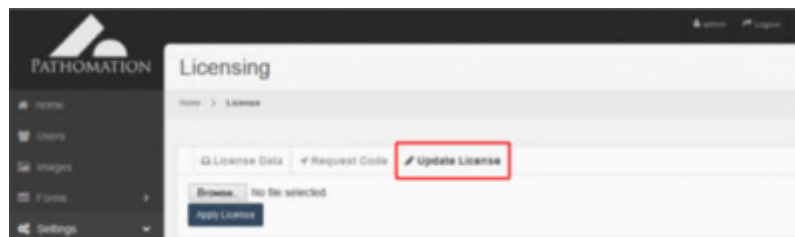

Once the license has been successfully installed, the system will display the appropriate message.

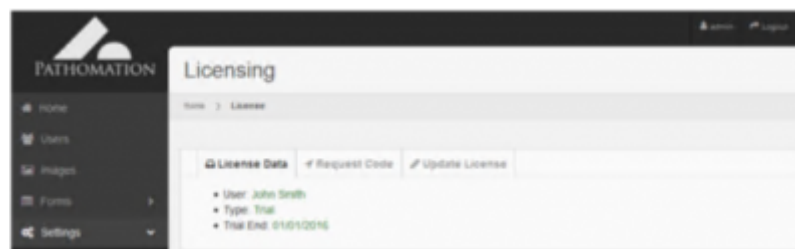

From:  
<https://docs.pathomation.com/pma.core/2.0.2/> - PMA.core 2.x

Permanent link:  
[https://docs.pathomation.com/pma.core/2.0.2/doku.php?id=license\\_setup&rev=1646653379](https://docs.pathomation.com/pma.core/2.0.2/doku.php?id=license_setup&rev=1646653379)

Last update: 2022/03/07 14:43

