

## Image settings

In the image settings section image compression and tile related values can be configured.

Image Settings

Cache Settings

Tile Size

512

Image Format

Jpeg

Tile Quality  
(100=best)

90

Save

Field	Description
Tile Size	The size of each side of the served tiles in pixels
Image Format	The image format of all the served images (tiles, thumbnails, snapshots etc.). This can be JPEG (lossy compression) or PNG (lossless compression)
Image Quality	1 to 100 value that defines the image quality for lossy compression schemes. Default value is 100 (equal to 100% quality). The parameter does NOT apply when the Image Format is set to PNG. <a href="#">See what impact this has on data traffic.</a>
Image Format Parameters	Format specific parameters that may be required

## Image settings

# Image Settings

Home > Image settings

Image Settings

Cache Settings

Barcode Settings

Tile Size

512

Image Format

Jpeg ▾

Tile Quality  
(100=best)

90

Save

Cache settings

# Cache Settings

Home > Cache settings

Image Settings

Cache Settings

Barcode Settings

Use cache

☒ (cached tiles will be stored in jpeg format at 90 % quality)

Cache Path

~/App\_Data/Cache

Retention Days

365

Disk free space  
required for  
caching

0

in GB

Tile Cache

5.71 KB

Cloud Storage  
Cache

23.38 MB

Drive size

2.73 TB (2.01 TB free)

Total cache  
size

23.39 MB (0.00% of total size)

Empty cache older than retention days

Empty cache for orphan slides

Empty all cache

Save

Barcode settings

Barcode Settings

Image Settings

Cache Settings

Barcode Settings

Try harder to read barcode texts

☐

Disable barcode scanning

☐

Save

From:  
<https://docs.pathomation.com/pma.core/2.0.2/> - PMA.core 2.x

Permanent link:  
[https://docs.pathomation.com/pma.core/2.0.2/doku.php?id=settings\\_image&rev=1648483554](https://docs.pathomation.com/pma.core/2.0.2/doku.php?id=settings_image&rev=1648483554)

Last update: 2022/03/28 19:05

