





Submitted data

The data submitted by each user for each form can be obtained in the “Submitted data” tab in the formats:

- [HTML](#)
- [CSV](#)
- [XML](#)
- [ARFF](#) (Attribute-Relational file format)

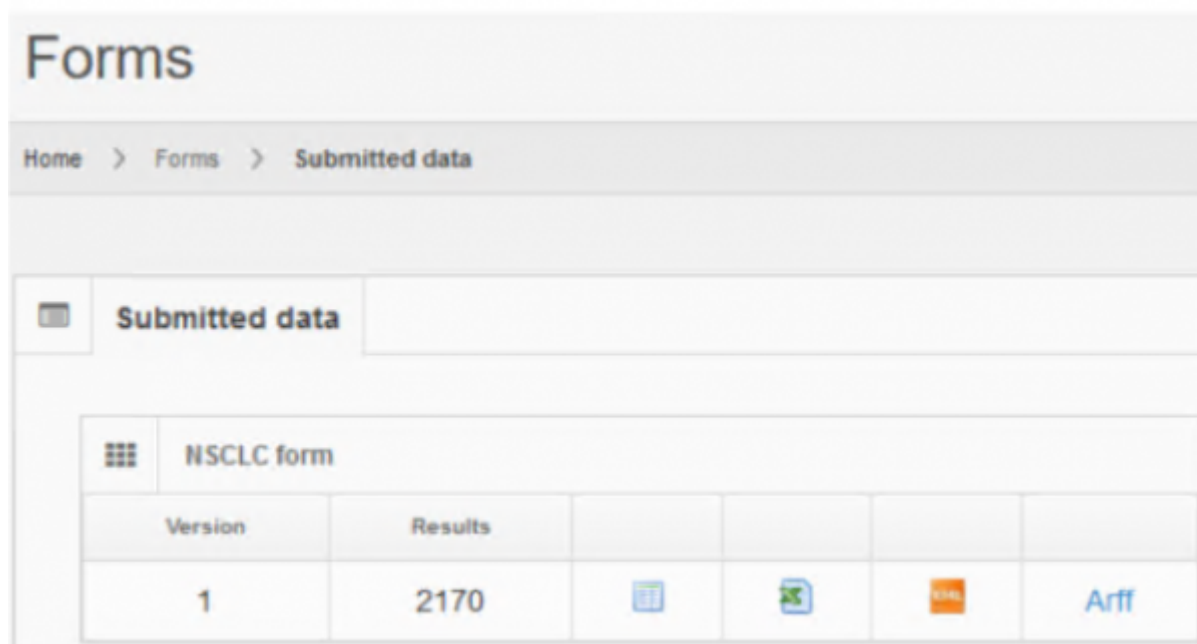

The screenshot shows a web interface titled "Forms". Below the title is a breadcrumb trail: "Home > Forms > Submitted data". A tab labeled "Submitted data" is selected. Below this, a table titled "NSCLC form" is displayed. The table has two columns: "Version" and "Results". The first row shows "1" under "Version" and "2170" under "Results". To the right of the "Results" column, there are four icons representing different export formats: a calendar icon, a document icon, a file icon, and a text icon. The word "Arff" is also visible next to the text icon.

Version	Results				
1	2170				Arff

HTML

Clicking on the export to HTML format will take you to an interactive html page displaying all submitted data. Here you can see the data in tabulated format, with pagination and also a search feature. This is very useful for some quick inspection and searching for specific data.

UC form - version 1

10 records per page

Search

WSI ref	User	Fingerprint	Created By	Created On	Last modified By	Last modified on	IQ%	POS/NEG	Comments	Audit Trail
_sys_refAnnotations (CMU-1.mrs)	consensus		system	4/11/2018 12:00:00 AM			15	POS	Acceptable H&E. Tumor area has muscle with immune cells, not a lot of necrosis. Invasive muscular-urothelial carcinoma, macrophages may mimic tumor cells. Intratumoral and stromal invasive immune cells. Combination of single cell and aggregate immune cell	Audit trail
_sys_refAnnotations (CMU-1.mrs)	fonttestuser100		system	4/11/2018 12:00:00 AM			5	POS		Audit trail
_sys_refAnnotations (CMU-1.mrs)	fonttestuser101		system	4/11/2018 12:00:00 AM			10	POS		Audit trail
_sys_refAnnotations (CMU-1.mrs)	fonttestuser102		system	4/11/2018 12:00:00 AM			26	POS		Audit trail
_sys_refAnnotations (CMU-1.mrs)	fonttestuser103		system	4/11/2018 12:00:00 AM			1	NEG		Audit trail
_sys_refAnnotations (CMU-1.mrs)	fonttestuser104		system	4/11/2018 12:00:00 AM			15	POS		Audit trail
_sys_refAnnotations (CMU-1.mrs)	fonttestuser11		system	4/11/2018 12:00:00 AM			4	NEG		Audit trail
_sys_refAnnotations (CMU-1.mrs)	fonttestuser116		system	4/11/2018 12:00:00 AM			4	NEG		Audit trail
_sys_refAnnotations (CMU-1.mrs)	fonttestuser130		system	4/11/2018 12:00:00 AM			1	NEG		Audit trail
_sys_refAnnotations (CMU-1.mrs)	fonttestuser144		system	4/11/2018 12:00:00 AM			5	NEG		Audit trail

Showing 1 to 10 of 1,404 entries

Previous

1

2

3

4

5

...

141

Next

CSV

The CSV (comma seperated values) file is a very common text file used primarily for programmatically input/output to various programming languages. It is very useful for scripting, and transferring data to other software. It is also a format readable by Excel and other spreadsheet programs.

Csv Options

Home > Forms > Csv options

Csv Formatting Options

Separator

(comma)

IQ%

☒

Comments

☒

POS/NEG

☒

Download Csv

Cancel

Clicking on the Export to CSV button will redirect to the page to configure some parameters required. Specifically you need to select the seperator of data fields, usually comma but based on your locale settings you may need to change this. You can also select whether to export any lookup-lists defined in the form

A	B	C	D	E	F	G	H	I	J	K	L	M	N	O	P	Q	R	S	T	U	V	W
1	data.xml	user	AuditTrailability	AuditTrailCondition	AuditTrailAuditTrail	AuditTrail	POS/NOS	Comments														
2	_sys_ref/Annotations/CML-1.arms	formtestuser000	system	4/11/2018 0:00			15 POS	Acceptable H&E, Tumor area has muscle with immune cells, not a lot of necrosis. Invasive muscular epithelial carcinoma, macrophages may mimic tumor														
3	_sys_ref/Annotations/CML-1.arms	formtestuser001	system	4/11/2018 0:00			8 POS															
4	_sys_ref/Annotations/CML-1.arms	formtestuser002	system	4/11/2018 0:00			18 POS															
5	_sys_ref/Annotations/CML-1.arms	formtestuser003	system	4/11/2018 0:00			20 POS															
6	_sys_ref/Annotations/CML-1.arms	formtestuser004	system	4/11/2018 0:00			1 NEG															
7	_sys_ref/Annotations/CML-1.arms	formtestuser005	system	4/11/2018 0:00			15 POS															
8	_sys_ref/Annotations/CML-1.arms	formtestuser006	system	4/11/2018 0:00			4 NEG															
9	_sys_ref/Annotations/CML-1.arms	formtestuser007	system	4/11/2018 0:00			4 NEG															
10	_sys_ref/Annotations/CML-1.arms	formtestuser008	system	4/11/2018 0:00			1 NEG															
11	_sys_ref/Annotations/CML-1.arms	formtestuser009	system	4/11/2018 0:00			2 NEG															
12	_sys_ref/Annotations/CML-1.arms	formtestuser010	system	4/11/2018 0:00			3 NEG															
13	_sys_ref/Annotations/CML-1.arms	formtestuser011	system	4/11/2018 0:00			12 POS															
14	_sys_ref/Annotations/CML-1.arms	formtestuser012	system	4/11/2018 0:00			2 NEG															
15	_sys_ref/Annotations/CML-1.arms	formtestuser013	system	4/11/2018 0:00			4 NEG															
16	_sys_ref/Annotations/CML-1.arms	formtestuser014	system	4/11/2018 0:00			1 NEG															
17	_sys_ref/Annotations/CML-1.arms	formtestuser015	system	4/11/2018 0:00			8 POS															
18	_sys_ref/Annotations/CML-1.arms	formtestuser016	system	4/11/2018 0:00			1 NEG															
19	_sys_ref/Annotations/CML-1.arms	formtestuser017	system	4/11/2018 0:00			12 POS															
20	_sys_ref/Annotations/CML-1.arms	formtestuser018	system	4/11/2018 0:00			10 POS															
21	_sys_ref/Annotations/CML-1.arms	formtestuser019	system	4/11/2018 0:00			8 POS															
22	_sys_ref/Annotations/CML-1.arms	formtestuser020	system	4/11/2018 0:00			15 POS															
23	_sys_ref/Annotations/CML-1.arms	formtestuser021	system	4/11/2018 0:00			3 NEG															
24	_sys_ref/Annotations/CML-1.arms	formtestuser022	system	4/11/2018 0:00			2 NEG															
25	_sys_ref/Annotations/CML-1.arms	formtestuser023	system	4/11/2018 0:00			18 POS															
26	_sys_ref/Annotations/CML-1.arms	formtestuser024	system	4/11/2018 0:00			NEG															
27	_sys_ref/Annotations/CML-1.arms	formtestuser025	system	4/11/2018 0:00			POS															
28	_sys_ref/Annotations/CML-1.arms	formtestuser026	system	4/11/2018 0:00			10 POS															
29	_sys_ref/Annotations/CML-1.arms	formtestuser027	system	4/11/2018 0:00			3 POS															
30	_sys_ref/Annotations/CML-1.arms	formtestuser028	system	4/11/2018 0:00			NEG															
31	_sys_ref/Annotations/CML-1.arms	formtestuser029	system	4/11/2018 0:00			18 POS															
32	_sys_ref/Annotations/CML-1.arms	formtestuser030	system	4/11/2018 0:00			3 POS															
33	_sys_ref/Annotations/CML-1.arms	formtestuser031	system	4/11/2018 0:00			2 NEG															
34	_sys_ref/Annotations/CML-1.arms	formtestuser032	system	4/11/2018 0:00			5 POS															
35	_sys_ref/Annotations/CML-1.arms	formtestuser033	system	4/11/2018 0:00			3 NEG															
36	_sys_ref/Annotations/CML-1.arms	formtestuser034	system	4/11/2018 0:00			8 NEG															
37	_sys_ref/Annotations/CML-1.arms	formtestuser035	system	4/11/2018 0:00			15 POS															
38	_sys_ref/Annotations/CML-1.arms	formtestuser036	system	4/11/2018 0:00			10 POS															
39	_sys_ref/Annotations/CML-1.arms	formtestuser037	system	4/11/2018 0:00			2 NEG															
40	_sys_ref/Annotations/CML-1.arms	formtestuser038	system	4/11/2018 0:00			2 NEG															
41	_sys_ref/Annotations/CML-1.arms	formtestuser039	system	4/11/2018 0:00			5 POS															

XML

The XML format is another format to exchange data between applications and software, used mainly for scripting and programmatically input/output of data.

```
1 <?xml version="1.0" encoding="UTF-8" ?>
2 <Submission Path="_sys_ref/Annotations/CML-1.arms" Login="conesens" AuditCreatedBy="system" AuditCreatedOn="4/11/2018 12:00:00 AM" AuditLastModifiedBy="" AuditLastModifiedOn="">
3   <FieldValue FieldId="1" Label="ICX">15</FieldValue>
4   <FieldValue FieldId="8" Label="Comments">Acceptable H&E, Tumor area has muscle with immune cells, not a lot of necrosis. Invasive muscular epithelial carcinoma, macrophages may mimic
5     tumor cells. Intratumoral and stromal invasive immune cells. Combination of single cell and aggregate immune cel</FieldValue>
6 </Submission>
7 <Submission Path="_sys_ref/Annotations/CML-1.arms" Login="formtestuser00" AuditCreatedBy="system" AuditCreatedOn="4/11/2018 12:00:00 AM" AuditLastModifiedBy="" AuditLastModifiedOn="">
8   <FieldValue FieldId="3" Label="POS/NEG">POS</FieldValue>
9   <FieldValue FieldId="8" Label="Comments">c</FieldValue>
10 </Submission>
11 <Submission Path="_sys_ref/Annotations/CML-1.arms" Login="formtestuser01" AuditCreatedBy="system" AuditCreatedOn="4/11/2018 12:00:00 AM" AuditLastModifiedBy="" AuditLastModifiedOn="">
12   <FieldValue FieldId="3" Label="POS/NEG">POS</FieldValue>
13   <FieldValue FieldId="8" Label="Comments">c</FieldValue>
14 </Submission>
15 <Submission Path="_sys_ref/Annotations/CML-1.arms" Login="formtestuser02" AuditCreatedBy="system" AuditCreatedOn="4/11/2018 12:00:00 AM" AuditLastModifiedBy="" AuditLastModifiedOn="">
16   <FieldValue FieldId="3" Label="POS/NEG">POS</FieldValue>
17   <FieldValue FieldId="8" Label="Comments">c</FieldValue>
18 </Submission>
19 <Submission Path="_sys_ref/Annotations/CML-1.arms" Login="formtestuser03" AuditCreatedBy="system" AuditCreatedOn="4/11/2018 12:00:00 AM" AuditLastModifiedBy="" AuditLastModifiedOn="">
20   <FieldValue FieldId="3" Label="POS/NEG">NEG</FieldValue>
21   <FieldValue FieldId="8" Label="Comments">c</FieldValue>
22 </Submission>
23 <Submission Path="_sys_ref/Annotations/CML-1.arms" Login="formtestuser04" AuditCreatedBy="system" AuditCreatedOn="4/11/2018 12:00:00 AM" AuditLastModifiedBy="" AuditLastModifiedOn="">
24   <FieldValue FieldId="3" Label="POS/NEG">POS</FieldValue>
25   <FieldValue FieldId="8" Label="Comments">c</FieldValue>
26 </Submission>
27 <Submission Path="_sys_ref/Annotations/CML-1.arms" Login="formtestuser05" AuditCreatedBy="system" AuditCreatedOn="4/11/2018 12:00:00 AM" AuditLastModifiedBy="" AuditLastModifiedOn="">
28   <FieldValue FieldId="3" Label="POS/NEG">NEG</FieldValue>
29   <FieldValue FieldId="8" Label="Comments">c</FieldValue>
30 </Submission>
31 <Submission Path="_sys_ref/Annotations/CML-1.arms" Login="formtestuser06" AuditCreatedBy="system" AuditCreatedOn="4/11/2018 12:00:00 AM" AuditLastModifiedBy="" AuditLastModifiedOn="">
32   <FieldValue FieldId="3" Label="POS/NEG">NEG</FieldValue>
33   <FieldValue FieldId="8" Label="Comments">c</FieldValue>
34 </Submission>
35 <Submission Path="_sys_ref/Annotations/CML-1.arms" Login="formtestuser07" AuditCreatedBy="system" AuditCreatedOn="4/11/2018 12:00:00 AM" AuditLastModifiedBy="" AuditLastModifiedOn="">
36   <FieldValue FieldId="3" Label="POS/NEG">NEG</FieldValue>
37   <FieldValue FieldId="8" Label="Comments">c</FieldValue>
38 </Submission>
39 <Submission Path="_sys_ref/Annotations/CML-1.arms" Login="formtestuser08" AuditCreatedBy="system" AuditCreatedOn="4/11/2018 12:00:00 AM" AuditLastModifiedBy="" AuditLastModifiedOn="">
40   <FieldValue FieldId="3" Label="POS/NEG">NEG</FieldValue>
41   <FieldValue FieldId="8" Label="Comments">c</FieldValue>
42 </Submission>
43 </XML>
```

ARFF

An ARFF (Attribute-Relation File Format) file is an ASCII text file that describes a list of instances sharing a set of attributes. ARFF files were developed by the Machine Learning Project at the Department of Computer Science of The University of Waikato for use with the Weka machine learning software.

```
1 #!usr/bin/perl -w
2 #!usr/bin/perl -w
3 #!usr/bin/perl -w
4 #!usr/bin/perl -w
5 #!usr/bin/perl -w
6 #!usr/bin/perl -w
7 #!usr/bin/perl -w
8 #!usr/bin/perl -w
9 #!usr/bin/perl -w
10 #!usr/bin/perl -w
11 #!usr/bin/perl -w
12 #!usr/bin/perl -w
13 #!usr/bin/perl -w
14 #!usr/bin/perl -w
15 #!usr/bin/perl -w
16 #!usr/bin/perl -w
17 #!usr/bin/perl -w
18 #!usr/bin/perl -w
19 #!usr/bin/perl -w
20 #!usr/bin/perl -w
21 #!usr/bin/perl -w
22 #!usr/bin/perl -w
23 #!usr/bin/perl -w
24 #!usr/bin/perl -w
25 #!usr/bin/perl -w
26 #!usr/bin/perl -w
27 #!usr/bin/perl -w
28 #!usr/bin/perl -w
29 #!usr/bin/perl -w
30 #!usr/bin/perl -w
31 #!usr/bin/perl -w
32 #!usr/bin/perl -w
33 #!usr/bin/perl -w
34 #!usr/bin/perl -w
35 #!usr/bin/perl -w
36 #!usr/bin/perl -w
37 #!usr/bin/perl -w
38 #!usr/bin/perl -w
39 #!usr/bin/perl -w
40 #!usr/bin/perl -w
41 #!usr/bin/perl -w
42 #!usr/bin/perl -w
43 #!usr/bin/perl -w
44 #!usr/bin/perl -w
45 #!usr/bin/perl -w
46 #!usr/bin/perl -w
47 #!usr/bin/perl -w
48 #!usr/bin/perl -w
49 #!usr/bin/perl -w
50 #!usr/bin/perl -w
51 #!usr/bin/perl -w
52 #!usr/bin/perl -w
53 #!usr/bin/perl -w
54 #!usr/bin/perl -w
55 #!usr/bin/perl -w
56 #!usr/bin/perl -w
57 #!usr/bin/perl -w
58 #!usr/bin/perl -w
59 #!usr/bin/perl -w
60 #!usr/bin/perl -w
61 #!usr/bin/perl -w
62 #!usr/bin/perl -w
63 #!usr/bin/perl -w
64 #!usr/bin/perl -w
65 #!usr/bin/perl -w
66 #!usr/bin/perl -w
67 #!usr/bin/perl -w
68 #!usr/bin/perl -w
69 #!usr/bin/perl -w
70 #!usr/bin/perl -w
71 #!usr/bin/perl -w
72 #!usr/bin/perl -w
73 #!usr/bin/perl -w
74 #!usr/bin/perl -w
75 #!usr/bin/perl -w
76 #!usr/bin/perl -w
77 #!usr/bin/perl -w
78 #!usr/bin/perl -w
79 #!usr/bin/perl -w
80 #!usr/bin/perl -w
81 #!usr/bin/perl -w
82 #!usr/bin/perl -w
83 #!usr/bin/perl -w
84 #!usr/bin/perl -w
85 #!usr/bin/perl -w
86 #!usr/bin/perl -w
87 #!usr/bin/perl -w
88 #!usr/bin/perl -w
89 #!usr/bin/perl -w
90 #!usr/bin/perl -w
91 #!usr/bin/perl -w
92 #!usr/bin/perl -w
93 #!usr/bin/perl -w
94 #!usr/bin/perl -w
95 #!usr/bin/perl -w
96 #!usr/bin/perl -w
97 #!usr/bin/perl -w
98 #!usr/bin/perl -w
99 #!usr/bin/perl -w
100 #!usr/bin/perl -w
```

From:
<https://docs.pathomation.com/pma.core/2.0.2/> - **PMA.core 2.x**

Permanent link:
https://docs.pathomation.com/pma.core/2.0.2/doku.php?id=submitted_data&rev=1648821869

Last update: 2022/04/01 17:04

