

Integrity report

The integrity report section will perform a recursive scan on the selected root directories and report virtual slides that cannot be parsed by the system.

The screenshot shows the 'Integrity report' interface. At the top, there's a breadcrumb 'Home > Integrity Report'. Below it, a section titled 'Root directories to check' contains a table with columns 'Name' and 'Path'. The table has three rows: 'Local images' with path 'E:\storage\images', 'Shared slides' with path 'C:\MSB\CDH\...', and an empty row. A 'Start check' button is below the table. The 'Results' section shows a progress bar and a list of results. The first result is 'Loading directories recursively...' followed by 'Finished loading directories...' and 'Checking 57 images...'. Below this, two red error messages are displayed: 'Cannot read Local images\JP2K-32000-1.avi' and 'Cannot read Local images\BIFR_10_201211261831.tif'.

Name	Path
Local images	E:\storage\images\
Shared slides	C:\MSB\CDH\...

Start check

Results

Loading directories recursively ...
Finished loading directories ...
Checking 57 images ...

Cannot read Local images\JP2K-32000-1.avi

Cannot read Local images\BIFR_10_201211261831.tif

You can select one or more root-directories to be scanned at the time.

From:

<https://docs.pathomation.com/pma.core/2.0/> - **PMA.core 2.x**

Permanent link:

https://docs.pathomation.com/pma.core/2.0/doku.php?id=integrity_report&rev=1648557440

Last update: **2022/03/29 15:37**

