

Image settings

In the image settings section image compression and tile related values can be configured.

The screenshot shows a configuration panel with two tabs: 'Image Settings' (selected) and 'Cache Settings'. Under 'Image Settings', there are three rows of controls: 'Tile Size' with a numeric input field containing '512', 'Image Format' with a dropdown menu showing 'Jpeg', and 'Tile Quality (100=best)' with a numeric input field containing '90'. A 'Save' button is located at the bottom center of the panel.

Field	Description
Tile Size	The size of each side of the served tiles in pixels
Image Format	The image format of all the served images (tiles, thumbnails, snapshots etc.). This can be JPEG (lossy compression) or PNG (lossless compression)
Image Quality	1 to 100 value that defines the image quality for lossy compression schemes. Default value is 100 (equal to 100% quality). The parameter does NOT apply when the Image Format is set to PNG. See what impact this has on data traffic.
Image Format Parameters	Format specific parameters that may be required

Image settings

Image Settings

Home > Image settings

Image Settings	Cache Settings	Barcode Settings
-----------------------	----------------	------------------

Tile Size

Image Format

Tile Quality (100=best)

Cache settings

Cache Settings

Home > Cache settings

Image Settings	Cache Settings	Barcode Settings
----------------	-----------------------	------------------

Use cache (cached tiles will be stored in Jpeg format at 90 % quality)

Cache Path

Retention Days

Disk free space required for caching
in GB

Tile Cache 5.71 KB

Cloud Storage Cache 23.38 MB

Drive size 2.73 TB (2.01 TB free)

Total cache size 23.39 MB (0.00% of total size)

Barcode settings

Barcode Settings

	Image Settings	Cache Settings	Barcode Settings	
<p>Try harder to read barcode texts <input type="checkbox"/></p> <p>Disable barcode scanning <input type="checkbox"/></p> <p>Save</p>				

From:
<https://docs.pathomation.com/pma.core/2.0/> - **PMA.core 2.x**

Permanent link:
https://docs.pathomation.com/pma.core/2.0/doku.php?id=settings_image&rev=1648483554

Last update: **2022/03/28 19:05**

