


Submitted data

The data submitted by each user for each form can be obtained in the “Submitted data” tab in the formats:

- [HTML](#)
- [CSV](#)
- [XML](#)
- [ARFF](#) (Attribute-Relational file format)

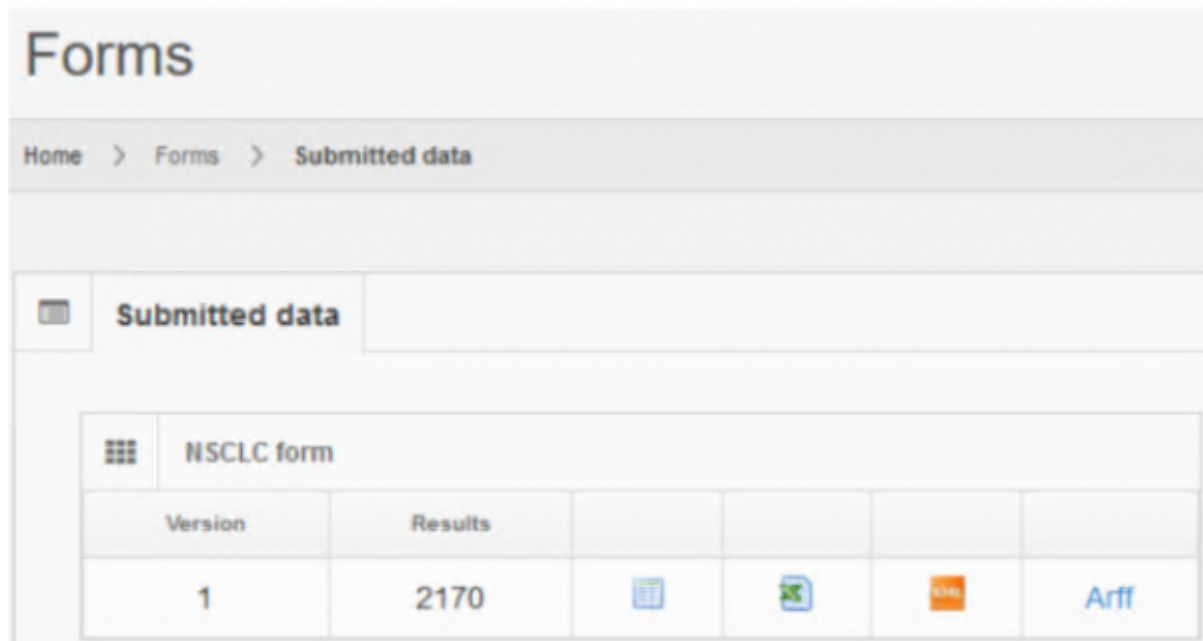

The screenshot shows a web interface titled "Forms". Below the title is a breadcrumb trail: "Home > Forms > Submitted data". A tab labeled "Submitted data" is selected. Below this, a table titled "NSCLC form" is displayed. The table has two columns: "Version" and "Results". The first row shows "1" under "Version" and "2170" under "Results". To the right of the "Results" column, there are four icons representing different export formats: a calendar icon (HTML), a document icon (CSV), a file icon (XML), and a file icon (ARFF).

Version	Results				
1	2170				Arff

HTML

Clicking on the export to HTML format will take you to an interactive html page displaying all submitted data. Here you can see the data in tabulated format, with pagination and also a search feature. This is very useful for some quick inspection and searching for specific data.

UC form - version 1

10 records per page

Search

WSI Ref	User	Fingerprint	Created By	Created On	Last modified By	Last modified on	IC%	POS/NEG	Comments	Audit Trail
_sys_refAnnotations (CMJ-1 nms)	consensus		system	4/11/2018 12:00:00 AM			15	POS	Acceptable H&E. Tumor area has muscle with immune cells, not a lot of necrosis. Invasive muscular urothelial carcinoma, macrophages may mimic tumor cells. Intratumoral and stromal invasive immune cells. Combination of single cell and aggregate immune cell	Audit trail
_sys_refAnnotations (CMJ-1 nms)	fonttestuser100		system	4/11/2018 12:00:00 AM			5	POS		Audit trail
_sys_refAnnotations (CMJ-1 nms)	fonttestuser101		system	4/11/2018 12:00:00 AM			10	POS		Audit trail
_sys_refAnnotations (CMJ-1 nms)	fonttestuser102		system	4/11/2018 12:00:00 AM			26	POS		Audit trail
_sys_refAnnotations (CMJ-1 nms)	fonttestuser103		system	4/11/2018 12:00:00 AM			1	NEG		Audit trail
_sys_refAnnotations (CMJ-1 nms)	fonttestuser104		system	4/11/2018 12:00:00 AM			15	POS		Audit trail
_sys_refAnnotations (CMJ-1 nms)	fonttestuser11		system	4/11/2018 12:00:00 AM			4	NEG		Audit trail
_sys_refAnnotations (CMJ-1 nms)	fonttestuser116		system	4/11/2018 12:00:00 AM			4	NEG		Audit trail
_sys_refAnnotations (CMJ-1 nms)	fonttestuser130		system	4/11/2018 12:00:00 AM			1	NEG		Audit trail
_sys_refAnnotations (CMJ-1 nms)	fonttestuser144		system	4/11/2018 12:00:00 AM			5	NEG		Audit trail

Showing 1 to 10 of 1,404 entries

Previous 1 2 3 4 5 ... 141 Next

CSV

The CSV (comma seperated values) file is a very common text file used primarily for programmatically input/output to various programming languages. It is very useful for scripting, and transferring data to other software. It is also a format readable by Excel and other spreadsheet programs.

Csv Options

Home > Forms > Csv options

Csv Formatting Options

Separator: (COMMA)

IC% ☒

Comments ☒

POS/NEG ☒

Download Csv Cancel

Clicking on the Export to CSV button will redirect to the page to configure some parameters required. Specifically you need to select the seperator of data fields, usually comma but based on your locale settings you may need to change this. You can also select whether to export any lookup-lists defined in the form

A	B	C	D	E	F	G	H	I	J	K	L	M	N	O	P	Q	R	S	T	U	V	W
1	data.xml	user	AuditTrailability	AuditTrailCondition	AuditTrailAuditTrail	AuditTrail	POS/NOS	Comments														
2	_sys_ref/annotations/CML-1.arms	formtestaser00	system	4/11/2018 0:00			15 POS	Acceptable HSE, Tumor area has muscle with immune cells, not a lot of necrosis. Invasive muscular epithelial carcinoma, macrophages may mimic tumor														
3	_sys_ref/annotations/CML-1.arms	formtestaser00	system	4/11/2018 0:00			8 POS															
4	_sys_ref/annotations/CML-1.arms	formtestaser01	system	4/11/2018 0:00			18 POS															
5	_sys_ref/annotations/CML-1.arms	formtestaser02	system	4/11/2018 0:00			20 POS															
6	_sys_ref/annotations/CML-1.arms	formtestaser03	system	4/11/2018 0:00			1 NEG															
7	_sys_ref/annotations/CML-1.arms	formtestaser04	system	4/11/2018 0:00			15 POS															
8	_sys_ref/annotations/CML-1.arms	formtestaser11	system	4/11/2018 0:00			4 NEG															
9	_sys_ref/annotations/CML-1.arms	formtestaser13	system	4/11/2018 0:00			4 NEG															
10	_sys_ref/annotations/CML-1.arms	formtestaser130	system	4/11/2018 0:00			1 NEG															
11	_sys_ref/annotations/CML-1.arms	formtestaser144	system	4/11/2018 0:00			2 NEG															
12	_sys_ref/annotations/CML-1.arms	formtestaser189	system	4/11/2018 0:00			3 NEG															
13	_sys_ref/annotations/CML-1.arms	formtestaser187	system	4/11/2018 0:00			12 POS															
14	_sys_ref/annotations/CML-1.arms	formtestaser148	system	4/11/2018 0:00			2 NEG															
15	_sys_ref/annotations/CML-1.arms	formtestaser149	system	4/11/2018 0:00			4 NEG															
16	_sys_ref/annotations/CML-1.arms	formtestaser151	system	4/11/2018 0:00			1 NEG															
17	_sys_ref/annotations/CML-1.arms	formtestaser152	system	4/11/2018 0:00			8 POS															
18	_sys_ref/annotations/CML-1.arms	formtestaser153	system	4/11/2018 0:00			1 NEG															
19	_sys_ref/annotations/CML-1.arms	formtestaser155	system	4/11/2018 0:00			12 POS															
20	_sys_ref/annotations/CML-1.arms	formtestaser17	system	4/11/2018 0:00			18 POS															
21	_sys_ref/annotations/CML-1.arms	formtestaser111	system	4/11/2018 0:00			8 POS															
22	_sys_ref/annotations/CML-1.arms	formtestaser112	system	4/11/2018 0:00			15 POS															
23	_sys_ref/annotations/CML-1.arms	formtestaser113	system	4/11/2018 0:00			3 NEG															
24	_sys_ref/annotations/CML-1.arms	formtestaser12	system	4/11/2018 0:00			1 NEG															
25	_sys_ref/annotations/CML-1.arms	formtestaser124	system	4/11/2018 0:00			18 POS															
26	_sys_ref/annotations/CML-1.arms	formtestaser125	system	4/11/2018 0:00			NEG															
27	_sys_ref/annotations/CML-1.arms	formtestaser127	system	4/11/2018 0:00			POS															
28	_sys_ref/annotations/CML-1.arms	formtestaser128	system	4/11/2018 0:00			18 POS															
29	_sys_ref/annotations/CML-1.arms	formtestaser129	system	4/11/2018 0:00			3 POS															
30	_sys_ref/annotations/CML-1.arms	formtestaser130	system	4/11/2018 0:00			NEG															
31	_sys_ref/annotations/CML-1.arms	formtestaser131	system	4/11/2018 0:00			18 POS															
32	_sys_ref/annotations/CML-1.arms	formtestaser132	system	4/11/2018 0:00			3 POS															
33	_sys_ref/annotations/CML-1.arms	formtestaser133	system	4/11/2018 0:00			2 NEG															
34	_sys_ref/annotations/CML-1.arms	formtestaser134	system	4/11/2018 0:00			5 POS															
35	_sys_ref/annotations/CML-1.arms	formtestaser136	system	4/11/2018 0:00			3 NEG															
36	_sys_ref/annotations/CML-1.arms	formtestaser138	system	4/11/2018 0:00			8 NEG															
37	_sys_ref/annotations/CML-1.arms	formtestaser135	system	4/11/2018 0:00			15 POS															
38	_sys_ref/annotations/CML-1.arms	formtestaser139	system	4/11/2018 0:00			18 POS															
39	_sys_ref/annotations/CML-1.arms	formtestaser177	system	4/11/2018 0:00			2 NEG															
40	_sys_ref/annotations/CML-1.arms	formtestaser178	system	4/11/2018 0:00			2 NEG															
41	_sys_ref/annotations/CML-1.arms	formtestaser179	system	4/11/2018 0:00			3 POS															

XML

The XML format is another format to exchange data between applications and software, used mainly for scripting and programmatically input/output of data.

```
1 <Submission Path="_sys_ref/annotations/CML-1.arms" Log="consensus" AuditCreatedBy="system" AuditCreatedOn="4/11/2018 12:00:00 AM" AuditLastModifiedBy="" AuditLastModifiedOn="">
2   <fieldvalue fieldid="1" label="ICX">15</fieldvalue>
3   <fieldvalue fieldid="8" label="Comments">Acceptable HSE, Tumor area has muscle with immune cells, not a lot of necrosis. Invasive muscular epithelial carcinoma, macrophages may mimic
4   tumor cells. Intratumoral and stromal invasive immune cells. Combination of single cell and aggregate immune cel</fieldvalue>
5 </Submission>
6 <Submission Path="_sys_ref/annotations/CML-1.arms" Log="formtestaser00" AuditCreatedBy="system" AuditCreatedOn="4/11/2018 12:00:00 AM" AuditLastModifiedBy="" AuditLastModifiedOn="">
7   <fieldvalue fieldid="3" label="POS/NEG">POS</fieldvalue>
8   <fieldvalue fieldid="8" label="Comments">c</fieldvalue>
9 </Submission>
10 <Submission Path="_sys_ref/annotations/CML-1.arms" Log="formtestaser01" AuditCreatedBy="system" AuditCreatedOn="4/11/2018 12:00:00 AM" AuditLastModifiedBy="" AuditLastModifiedOn="">
11   <fieldvalue fieldid="3" label="POS/NEG">POS</fieldvalue>
12   <fieldvalue fieldid="8" label="Comments">c</fieldvalue>
13 </Submission>
14 <Submission Path="_sys_ref/annotations/CML-1.arms" Log="formtestaser02" AuditCreatedBy="system" AuditCreatedOn="4/11/2018 12:00:00 AM" AuditLastModifiedBy="" AuditLastModifiedOn="">
15   <fieldvalue fieldid="3" label="POS/NEG">POS</fieldvalue>
16   <fieldvalue fieldid="8" label="Comments">c</fieldvalue>
17 </Submission>
18 <Submission Path="_sys_ref/annotations/CML-1.arms" Log="formtestaser03" AuditCreatedBy="system" AuditCreatedOn="4/11/2018 12:00:00 AM" AuditLastModifiedBy="" AuditLastModifiedOn="">
19   <fieldvalue fieldid="3" label="POS/NEG">POS</fieldvalue>
20   <fieldvalue fieldid="8" label="Comments">c</fieldvalue>
21 </Submission>
22 <Submission Path="_sys_ref/annotations/CML-1.arms" Log="formtestaser04" AuditCreatedBy="system" AuditCreatedOn="4/11/2018 12:00:00 AM" AuditLastModifiedBy="" AuditLastModifiedOn="">
23   <fieldvalue fieldid="3" label="POS/NEG">POS</fieldvalue>
24   <fieldvalue fieldid="8" label="Comments">c</fieldvalue>
25 </Submission>
26 <Submission Path="_sys_ref/annotations/CML-1.arms" Log="formtestaser11" AuditCreatedBy="system" AuditCreatedOn="4/11/2018 12:00:00 AM" AuditLastModifiedBy="" AuditLastModifiedOn="">
27   <fieldvalue fieldid="3" label="POS/NEG">POS</fieldvalue>
28   <fieldvalue fieldid="8" label="Comments">c</fieldvalue>
29 </Submission>
30 <Submission Path="_sys_ref/annotations/CML-1.arms" Log="formtestaser13" AuditCreatedBy="system" AuditCreatedOn="4/11/2018 12:00:00 AM" AuditLastModifiedBy="" AuditLastModifiedOn="">
31   <fieldvalue fieldid="3" label="POS/NEG">POS</fieldvalue>
32   <fieldvalue fieldid="8" label="Comments">c</fieldvalue>
33 </Submission>
34 <Submission Path="_sys_ref/annotations/CML-1.arms" Log="formtestaser130" AuditCreatedBy="system" AuditCreatedOn="4/11/2018 12:00:00 AM" AuditLastModifiedBy="" AuditLastModifiedOn="">
35   <fieldvalue fieldid="3" label="POS/NEG">POS</fieldvalue>
36   <fieldvalue fieldid="8" label="Comments">c</fieldvalue>
37 </Submission>
38 <Submission Path="_sys_ref/annotations/CML-1.arms" Log="formtestaser144" AuditCreatedBy="system" AuditCreatedOn="4/11/2018 12:00:00 AM" AuditLastModifiedBy="" AuditLastModifiedOn="">
39   <fieldvalue fieldid="3" label="POS/NEG">POS</fieldvalue>
40   <fieldvalue fieldid="8" label="Comments">c</fieldvalue>
41 </Submission>
```

ARFF

An ARFF (Attribute-Relation File Format) file is an ASCII text file that describes a list of instances sharing a set of attributes. ARFF files were developed by the Machine Learning Project at the Department of Computer Science of The University of Waikato for use with the Weka machine learning software.

[illegible]

From:
<https://docs.pathomation.com/pma.core/2.0/> - **PMA.core 2.x**

Permanent link:
https://docs.pathomation.com/pma.core/2.0/doku.php?id=submitted_data&rev=1648821869

Last update: **2022/04/01 17:04**

