

User monitoring

Overview table

- The “Last login” column displays each user's last successful login date.
- The “Local” column indicates whether or not a user is managed by PMA.core or LDAP.
- The “Administrator” column indicates whether or not a user can access PMA.core. Non administrator users cannot perform system management tasks within PMA.core.
- The “Can Annotate” column indicates whether or not a user is allowed to create and edit annotations.
- The “Suspended” column indicates whether or not a user is currently allowed to login to either PMA.core or downstream applications. A suspended login is treated as invalid by the system.
- The “Locked out” column indicates whether or not a user is currently locked out due to consecutive invalid log in attempts.
- Clicking on a user's email address launches your default email client.
- Clicking on a user's login name displays a summary of the user's activity.

Next to each entry the following buttons appear:

Button	Description
Edit	Edits the selected user; this option allows the modification of the user's password, email and name. Furthermore the access level of the user can be changed (administrator or not) as well as whether or not the user is allowed to create and manage annotations . Finally a user can be (un)suspended via this option
Delete	Deletes the selected user
Remind password	Sends a notification email to the selected use that contains their password in plain text
Reset and remind password	Sets a random password for the selected user and sends them a notification email that contains the new password in plain text
Unlock	Lifts a lock-out

User details view

By clicking on the user's login in the overview table, you can see a summary of the user's settings and activity. This page provides an overview of the user's information and the root directories they can access.

1233 x 742 px (displayed at 778 x 468)

User admin Details

Home > Users > Details

Summary

Edit

Activity log

Audit trail

ACL

User account details

First Name

Administrator

Created by

System

Last Name

Pathomation

Created on (UTC)

4/15/2019 12:00:00 AM

Email

admin1@pathomation.com

Last modified by

Administrator

☒

Last modified on (UTC)

Suspended

☐

Password

Remind password

Session ID

ImkX1mboJi_lo8EAeghZPQ2

User is online

ACL summary (access control lists)

Public root-directories

_sys_ref

invalid_local

_sys_aws_s3

invalid_s3_cred

hgx

invalid_s3_bucket

hgx_idx

Test

Private root-directories

No private root-directories defined.

Access rights

The Access rights (ACL) section allows control over the [private root directories](#) the particular user may access. If no private root directories are defined, this list will be empty.

778 x 267 px (displayed at 559 x 192)

Access control for user `admin`

Summary

Edit

Activity log

Audit trail

ACL

Directory	Has Access
_sys_ref	<input type="checkbox"/>

Save

Back to List

Activity log

The activity log section provides a detailed list of events that involve the displayed user.

1227 x 817 px (displayed at 772 x 514)

User admin Log

Home > Users > Log

Summary Edit **Activity log** Audit trail ACL

System Access Log

Session ID	First Accessed	Last Accessed	Host Address	Host Name	Application	Success	Failure Reason
lnkX1mboJl_lo8EAeghZPQ2	4/15/2019, 9:50:49 AM	4/15/2019, 2:15:31 PM	85.73.248.94	85.73.248.94	PMA.core	true	
	4/15/2019, 4:34:35 AM	4/15/2019, 4:34:35 AM	127.0.0.1	127.0.0.1	test	false	Invalid username or password
upjGsSqVVKQpkp6H266IA2	4/15/2019, 4:32:52 AM	4/15/2019, 9:45:22 AM	85.73.248.94	85.73.248.94	PMA.core	true	

Showing 1 to 3 of 3 entries

Previous 1 Next

Slide Access Log

First Accessed	Last Accessed	Session ID	Slide Path	Slide Name
4/15/2019, 12:49:17 PM	4/15/2019, 2:15:32 PM	lnkX1mboJl_lo8EAeghZPQ2	._sys_ref/brightfield/Smartzoom	cmu-1.sz
4/15/2019, 12:48:25 PM	4/15/2019, 2:15:32 PM	lnkX1mboJl_lo8EAeghZPQ2	._sys_ars_s3/brightfield/Hamamatsu/CMU-1-VMS	CMU-1-40x - 2010-01-12 13.24.05.vms
4/15/2019, 12:47:59 PM	4/15/2019, 2:15:32 PM	lnkX1mboJl_lo8EAeghZPQ2	._sys_ref/brightfield/Hamamatsu/CMU-1-VMS	CMU-1-40x - 2010-01-12 13.24.05.vms
4/15/2019, 12:47:57 PM	4/15/2019, 2:15:32 PM	lnkX1mboJl_lo8EAeghZPQ2	._sys_ars_s3/brightfield/Generic_Png	png01_julia_4K.png
4/15/2019, 12:47:54 PM	4/15/2019, 2:15:32 PM	lnkX1mboJl_lo8EAeghZPQ2	._sys_ref/brightfield/Generic_Png	png01_julia_4K.png

Audit trail

The audit trail section displays a log of modification events that involve the current user.

1230 x 471 px (displayed at 854 x 327)

Audit trail for user: test

Home > Users > Audit trail

Summary Edit **Audit trail** Activity log ACL

Audit trail

EntryId	EventId	ActionType	EventDate	UserLogin	UserHost	Administrator	CanAnnotate	Description	Email	Firstname	Lastname
0	174	Update	4/15/2019, 4:34:15 PM	admin	85.73.248.94	False	True	Test user	test123@pathomation.com	Test	test
0	173	Create	4/15/2019, 4:34:00 PM	admin	85.73.248.94	False	False	Test user	test123@pathomation.com	Test	test

Showing 1 to 2 of 2 entries

Previous 1 Next

From:

<https://docs.pathomation.com/pma.core/2.0/> - **PMA.core 2.x**

Permanent link:

https://docs.pathomation.com/pma.core/2.0/doku.php?id=user_monitoring

Last update: **2022/03/25 20:07**

