

Global settings (administration)

This admin page allows us to set global settings for PMA.Studio application. Page is divided into two set of fields:

Global settings has 6 fields: ·

1. Language
2. Application IconNo file chosen
3. Email For Help
4. Conference Server Url
5. Snapshot Filename Template

Annotation settings has 5 fields: ·

1. Show Annotation Dialog
2. Auto Save Annotations
3. Restart Annotations Drawing
4. Show Slide Info Button
5. Show Full Slide Path In Viewport

Edit Global Settings

Global application settings

Language: en (English)

Show Research Use Only Dialog: ☐

Application Icon: Choose File No file chosen

Email For Help: help@pathomation.com

Conference Server Url: https://host.pathomation.com/pma.live/

Snapshot Filename Template: snapshot_%date%_%time%

Annotation-related settings

Show Annotation Dialog: ☒

Auto Save Annotations: ☐

Restart Annotations Drawing: ☐

Show Slide Info Button: ☒

Show Full Slide Path In Viewport: ☒

Save Cancel

From:

<https://docs.pathomation.com/pma.studio/2.1.0/> - **PMA.studio 2.1.0**

Permanent link:

https://docs.pathomation.com/pma.studio/2.1.0/doku.php?id=admin_global

Last update: **2023/04/06 22:58**

