


Ribbon layout (administration)

This admin page allows you to setup default Ribbon layout that consists of different tabs and controls. Each tab consists of different controls associated to it and you can configure appearance of tabs and controls using this page.

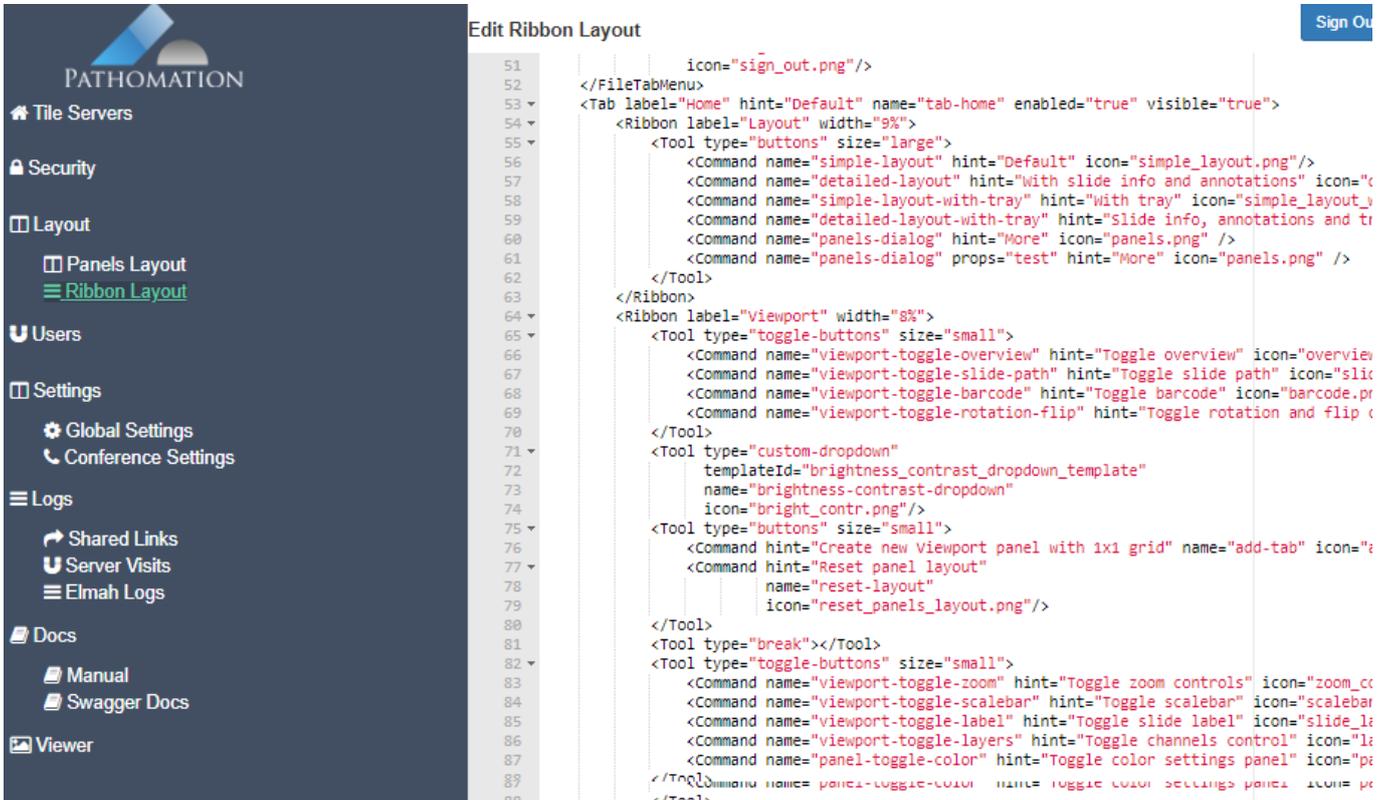

The screenshot shows the 'Edit Ribbon Layout' page in PMA.Studio. The left sidebar is a dark blue navigation menu with the Pathomation logo at the top. The main content area is titled 'Edit Ribbon Layout' and shows a code editor with XML code for configuring the ribbon layout. The code includes tabs like 'Home' and 'Layout', and various tools and commands for controlling the ribbon's appearance and functionality. A 'Sign Out' button is visible in the top right corner.

Below are some of the tabs and corresponding controls used in PMA.Studio.

Home tab has 8 sections:

1. View
2. Grid
3. Snap
4. Magnify
5. Directory
6. Slide
7. Share
8. Search



Grid tab supplies instruments to manage grid and it has 6 sections:

1. ItemSize
2. Synchronize
3. Rows/Columns
4. Slide
5. Snapshot
6. Share



Annotations tab (available in PMA.Studio) supplies instruments for working with annotations and has 4 sections:

- 1. Annotations
- 2. Shapes
- 3. Export
- 4. Import




Pre-set Annotations tab (available in PMA.Studio) consists of 3 sections:

- 1. Annotations
- 2. Stamps
- 3. Protocols



Conferencing tab has 2 sections:

- 1. Conferencing
- 2. Server

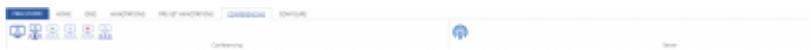

Configure tab has 4 sections:

- 1. Server
- 2. Panels
- 3. Pre-set layouts
- 4. Viewport



From: <https://docs.pathomation.com/pma.studio/2.1.0/> - **PMA.studio 2.1.0**

Permanent link: https://docs.pathomation.com/pma.studio/2.1.0/doku.php?id=admin_ribbon&rev=1674470694

Last update: **2023/01/23 13:44**

