


Tile servers (administration)


In this section of PMA.studio's administrative portal, you can manage the different PMA.core instances that are used by PMA.studio to retrieve content from.

PMA.studio (PMA.vue) does not function without connectivity to a PMA.core tile server. If you want to learn more about what a tile server is, [we have a YouTube video on this topic](#).

This page allows you to add different types of servers used by PMA.Studio including [PMA.core](#), [PMA.search](#), [PMA.start](#), [My Pathomation](#) and [PMA.live](#). You can press “Add New” button to add a new server.

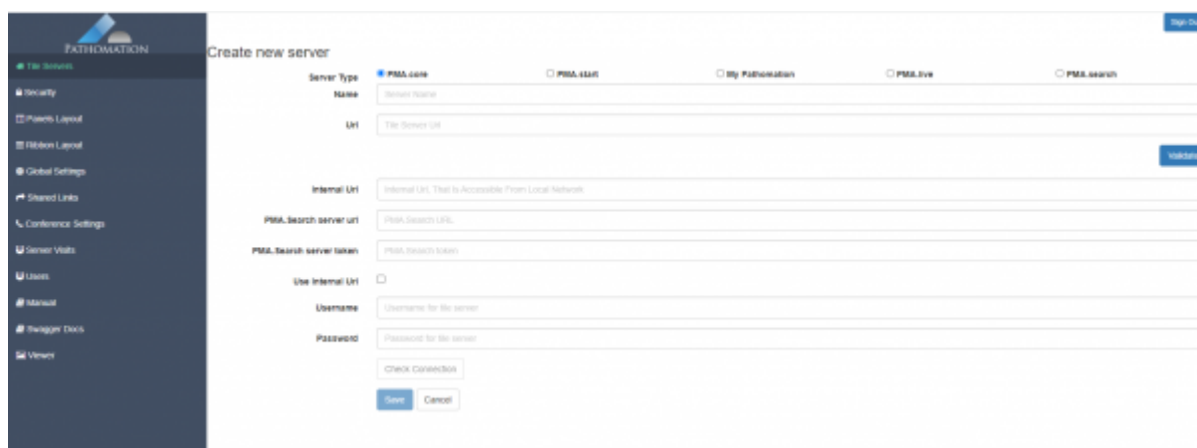

It allows you to manage the list of created servers where you can view server status (online/offline) along with setting the order of these servers to appear in PMA.Studio viewer. Besides you can delete a server as per your convenience using the delete button.

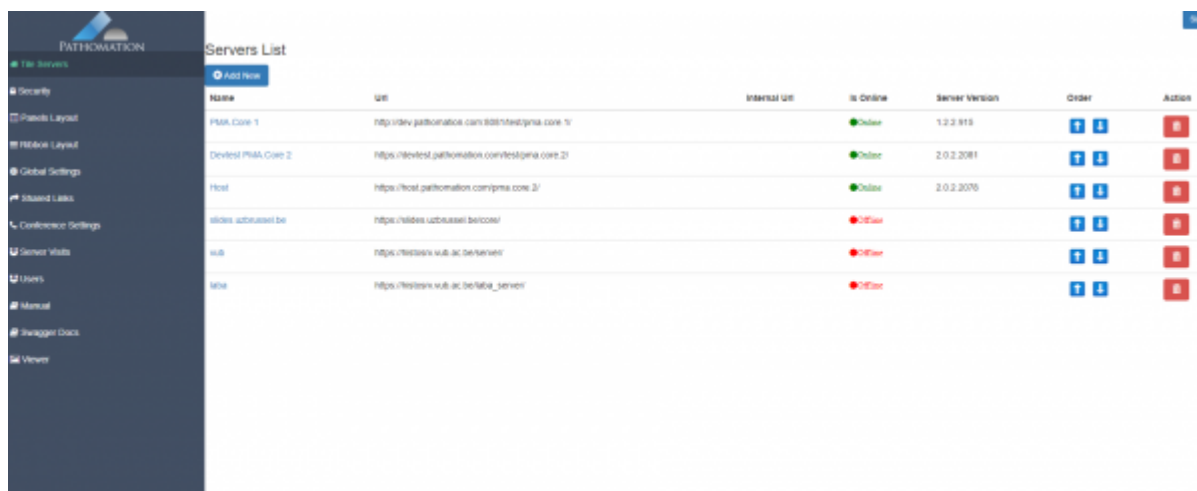

Name	URI	Internal URI	Is Online	Server Version	Order	Action
PMA Core 1	http://dev.pathomation.com/identifypma_core/1/		Online	1.2.2.819	1	[Up] [Down] [Delete]
Devtest PMA Core 2	https://devtest.pathomation.com/testpma_core/2/		Online	2.0.2.2081	2	[Up] [Down] [Delete]
Host	https://host.pathomation.com/pma_core/3/		Online	2.0.2.2070	3	[Up] [Down] [Delete]
uvides-uzbrussell.be	https://uvides-uzbrussell.be/cores/		Offline		4	[Up] [Down] [Delete]
sub	https://testpma-sub.ac.be/server/		Offline		5	[Up] [Down] [Delete]
live	https://testpma-sub.ac.be/live_server/		Offline		6	[Up] [Down] [Delete]

Notes on PMA.live

More background on PMA.live can be found at <https://www.pathomation.com/pma.live>.

From:

<https://docs.pathomation.com/pma.studio/2.1.0/> - **PMA.studio 2.1.0**

Permanent link:

https://docs.pathomation.com/pma.studio/2.1.0/doku.php?id=admin_tileservers

Last update: **2023/04/06 22:58**

