


Tile servers (administration)

In this section of PMA.studio's administrative portal, you can manage the different PMA.core instances that are used by PMA.studio to retrieve content from.

PMA.studio (PMA.vue) does not function without connectivity to a PMA.core tile server. If you want to learn more about what a tile server is, [we have a YouTube video on this topic](#).

This page allows you to add different types of servers used by PMA.Studio including [PMA.core](#), [PMA.search](#), [PMA.start](#), [My Pathomation](#) and [PMA.live](#). You can press "Add New" button to add a new server.

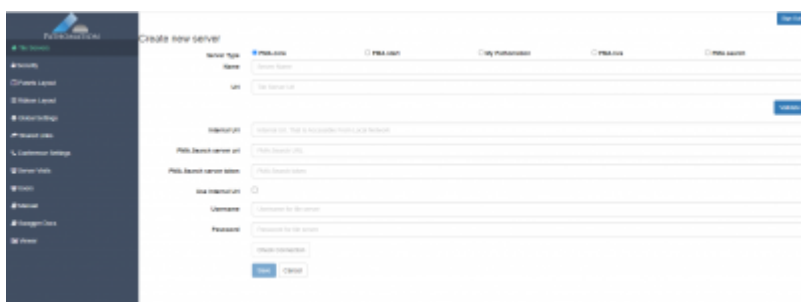

It allows you to manage the list of created servers where you can view server status (online/offline) along with setting the order of these servers to appear in PMA.Studio viewer. Besides you can delete a server as per your convenience using the delete button.



From: <https://docs.pathomation.com/pma.studio/2.1.0/> - **PMA.studio 2.1.0**

Permanent link: https://docs.pathomation.com/pma.studio/2.1.0/doku.php?id=admin_tileservers&rev=1674040454

Last update: **2023/01/18 14:14**

