

# Brush and magic wand


Brush and magic wand are two special tools that allow you to draw annotation on specific area of a slide.

(Brush Tool)

Brush tool allows you to paint specific area and create annotation on it.

(Wand Tool)

Wand tool allows you to select different areas on a slide by simply clicking on the slide. Once you click on a slide that specific area is automatically selected and annotation can be created on that area by clicking "create annotation from selection" button.

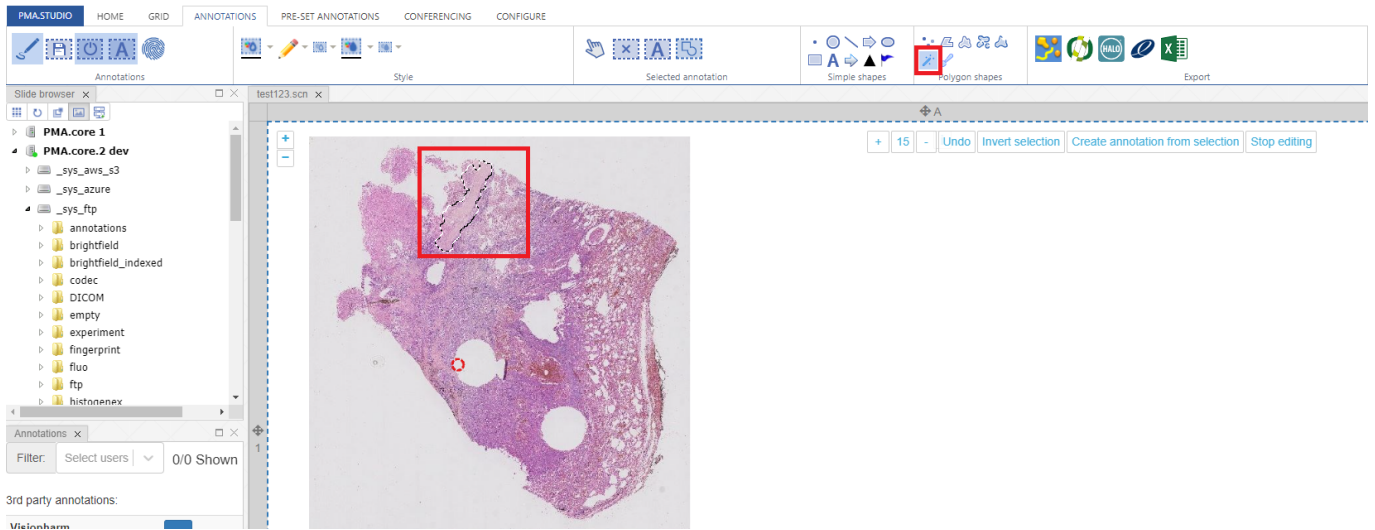

From:  
<https://docs.pathomation.com/pma.studio/2.1.0/> - **PMA.studio 2.1.0**

Permanent link:  
[https://docs.pathomation.com/pma.studio/2.1.0/doku.php?id=annotations\\_magic&rev=1650563038](https://docs.pathomation.com/pma.studio/2.1.0/doku.php?id=annotations_magic&rev=1650563038)

Last update: **2022/04/21 20:43**

