

# Simple shapes

Below are the set of simple shapes that can be added to a annotation.



- Point
- Circle
- Line
- Arrow
- Ellipse
- Rectangle
- Text
- Landmark 1
- Landmark 2
- Landmark 3

## Point Shape

Lets start with drawing a point, once you select the shape and click on the slide; a point will be drawn and a popup dialog would appear that would allow you to add annotation title and classification. Once you press "Add Annotation" the annotation will be saved and will appear on the slide.

The screenshot illustrates the workflow in PMA.studio. The top panel shows the 'Enter annotation text' dialog box with the following fields:

- Name: Annotation 1
- Classification: Choose a classification.

The bottom panel shows the histology slide with a red box around the point annotation. The 'Annotations' panel on the right displays the following table:


Note	Pos	Area (mm <sup>2</sup> )	Length (mm)	Polar Count	Created By
No class [1]					
Annotation 1	•	N/A	N/A	N/A	N/A

Below the table, the 'General Annotation stats' section shows:

Class	Count
No class	1
Selected	0


## Circle Shape

Next is “Circle” tool that allows you to draw circle annotation on a slide.

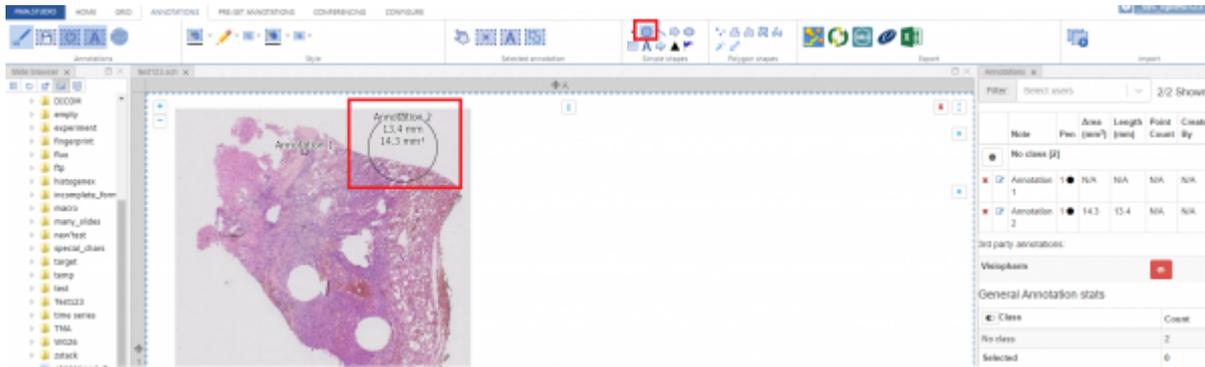


## Line Tool

Next is “Line” tool that allows you to draw line annotation on a slide.

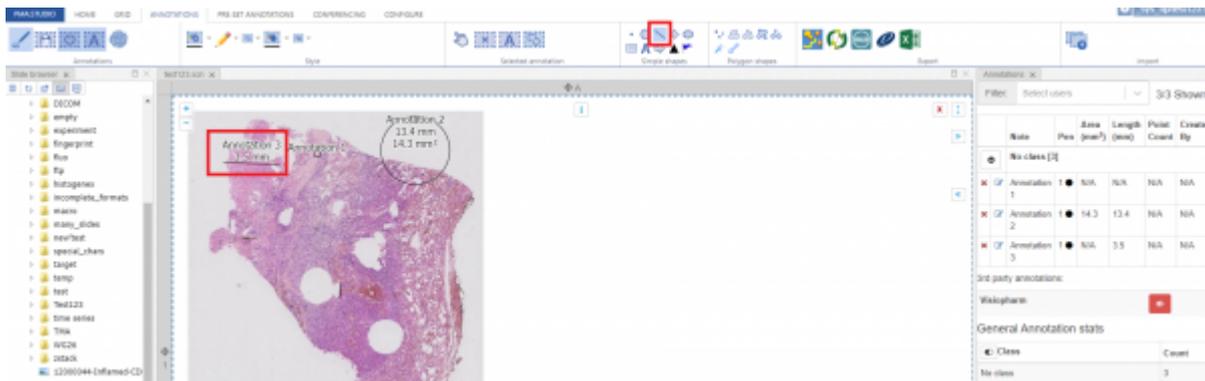


## Arrow Tool

Next is “Arrow” tool that allows you to draw arrow annotation on a slide.

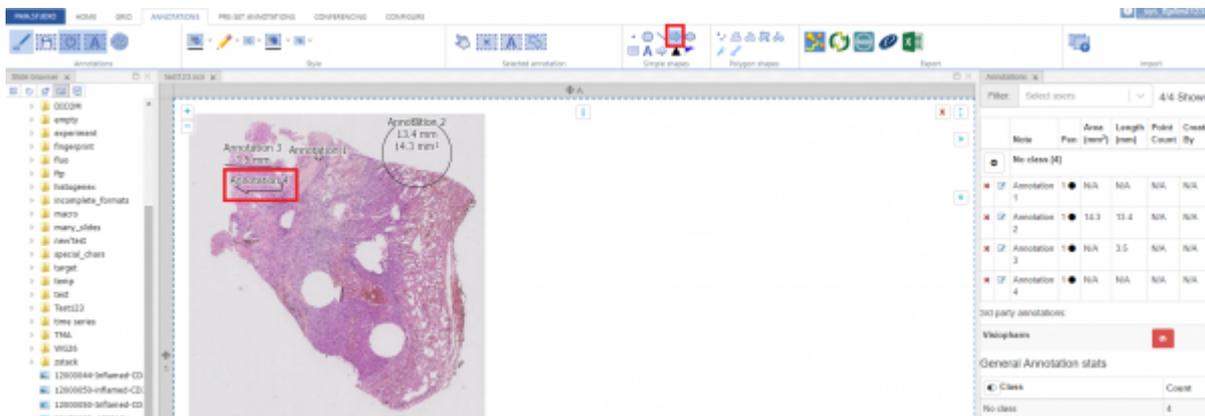

## Ellipse Tool

Next is “Ellipse ” tool that allows you to draw ellipse annotation on a slide.

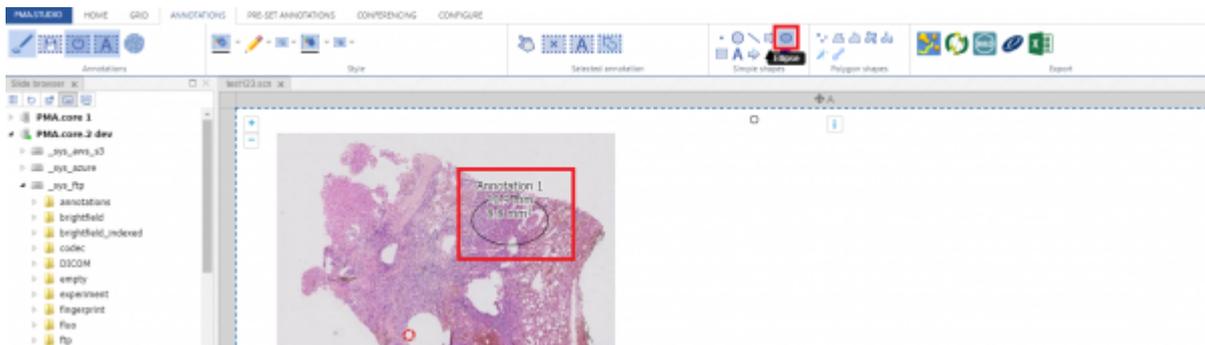


## Rectangular Tool

Next is “Rectangular” tool that allows you to draw rectangular annotation on a slide.




## Landmark 1 Tool

Next is “Landmark 1” tool that allows you to add first of the 3 landmark annotation on a slide.

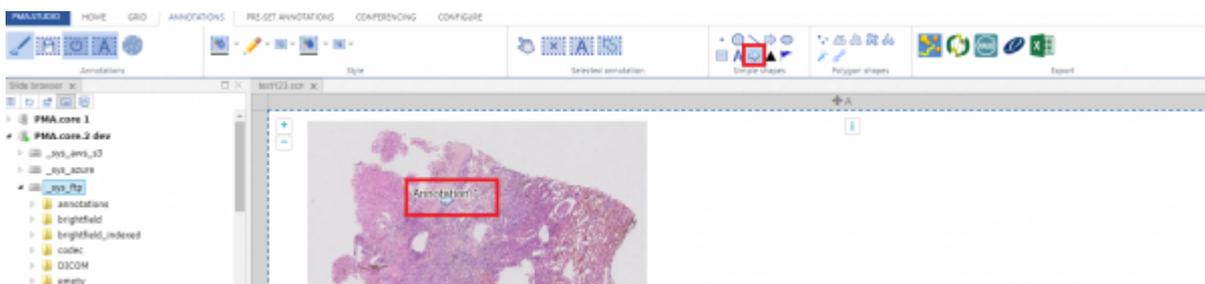


## Landmark 2 Tool

Next is “Landmark 2” tool that allows you to add 2nd of the 3 landmark annotation on a slide.

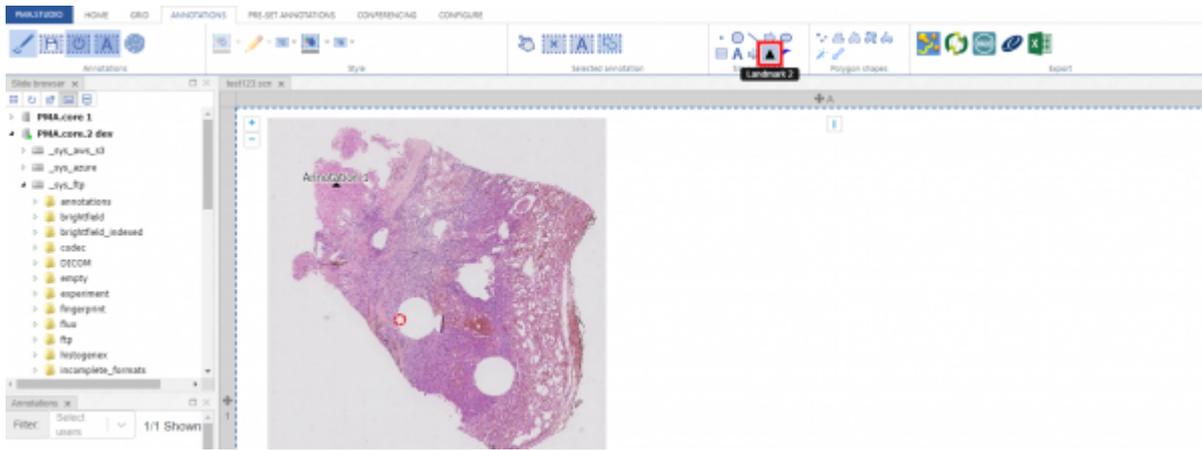

## Landmark 3 Tool

Next is "Landmark 3" tool that allows you to add last of the 3 landmark annotation on a slide.

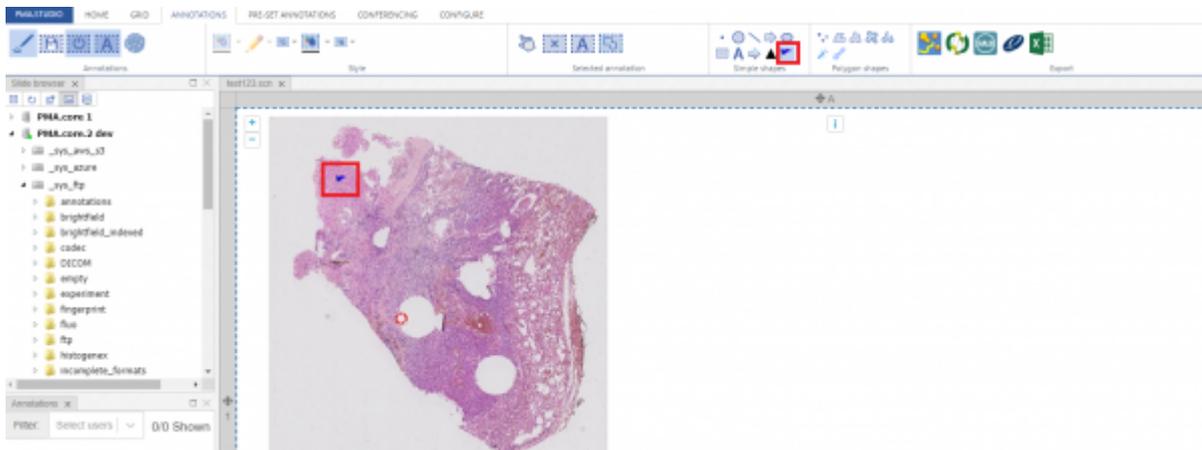

From: <https://docs.pathomation.com/pma.studio/2.1.0/> - **PMA.studio 2.1.0**

Permanent link: [https://docs.pathomation.com/pma.studio/2.1.0/doku.php?id=annotations\\_simple&rev=1650555343](https://docs.pathomation.com/pma.studio/2.1.0/doku.php?id=annotations_simple&rev=1650555343)

Last update: **2022/04/21 18:35**

