


Simple Shapes


Below are the set of simple shapes that can be added to a annotation.

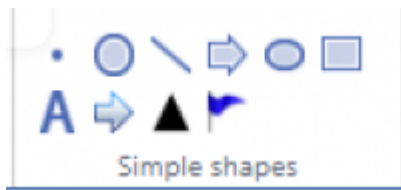

- Point
- Circle
- Line
- Arrow
- Ellipse
- Rectangle
- Text
- Landmark 1
- Landmark 2
- Landmark 3


Point Shape

Lets start with drawing a point, once you select the shape and click on the slide; a point will be drawn and a popup dialog would appear that would allow you to add annotation title and classification. Once you press "Add Annotation" the annotation will be saved and will appear on the slide.

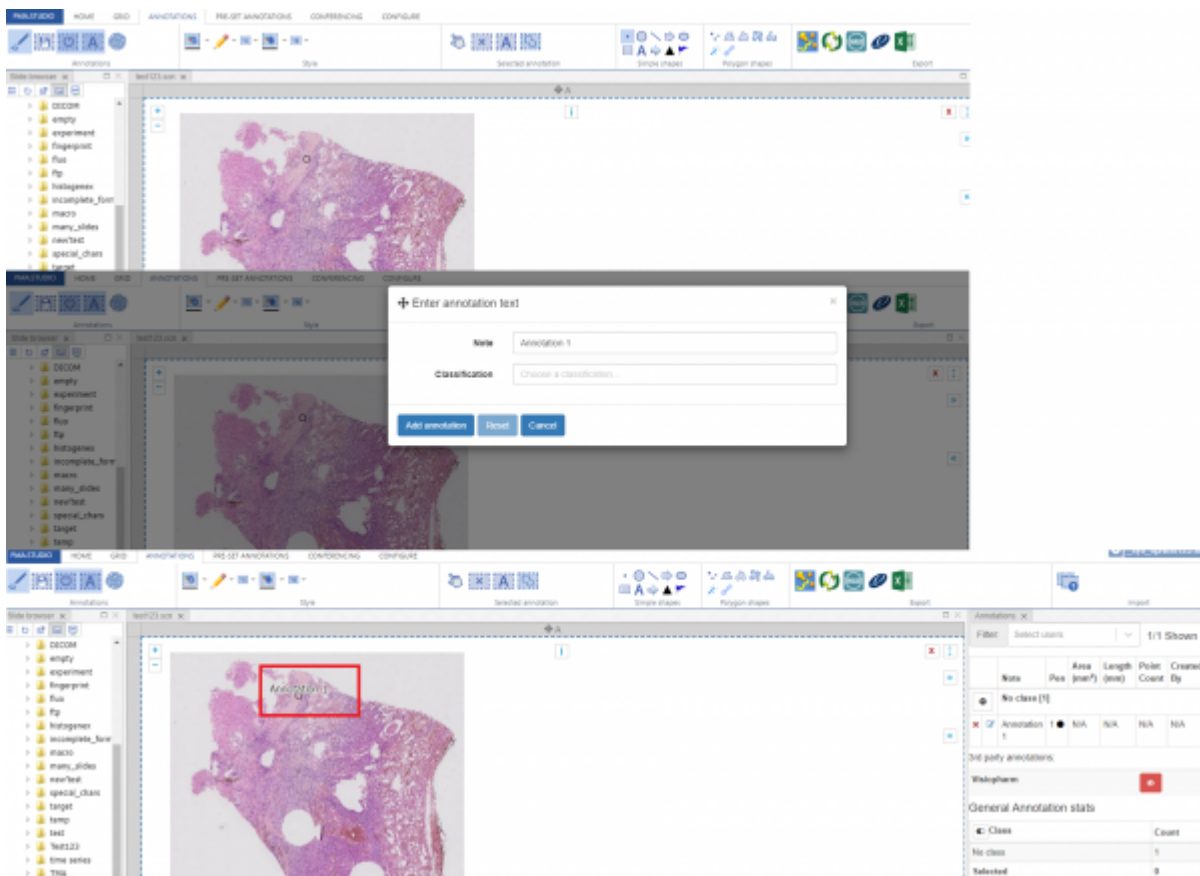


Circle Shape

Next is “Circle” shape that allows you to draw circle annotation on a slide.

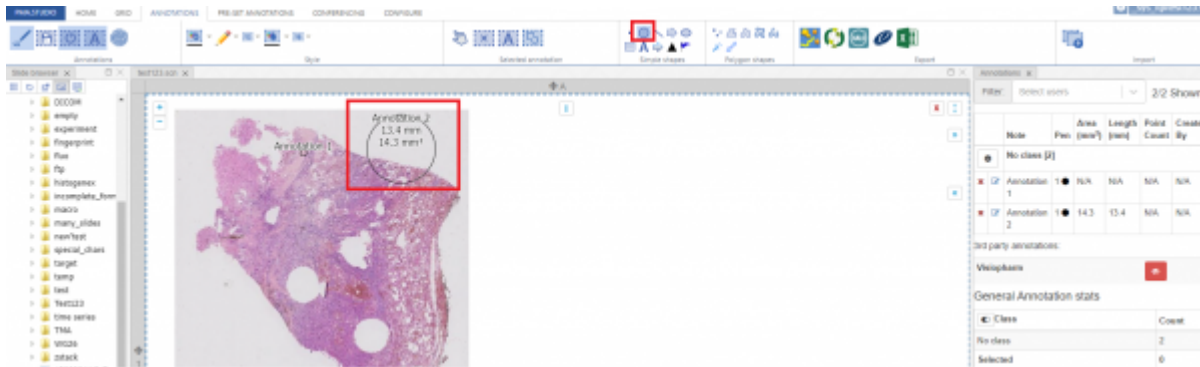


Line Shape

Next is “Line” shape that allows you to draw line annotation on a slide.

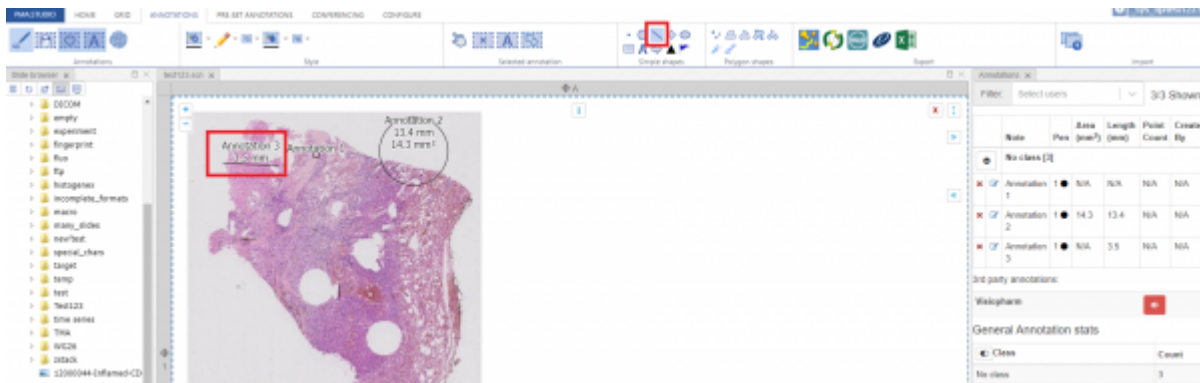


Arrow Shape

Next is “Arrow” shape that allows you to draw arrow annotation on a slide.

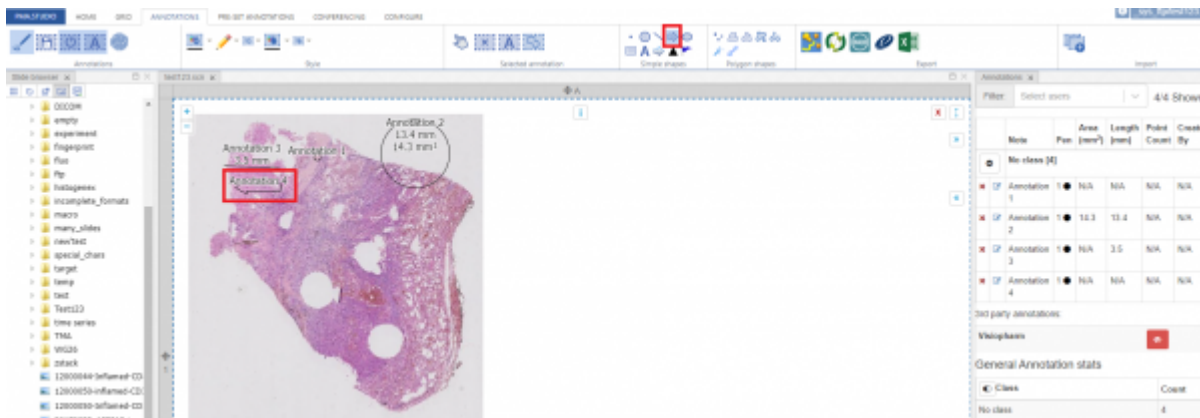

Ellipse Shape

Next is “Ellipse ” shape that allows you to draw ellipse annotation on a slide.

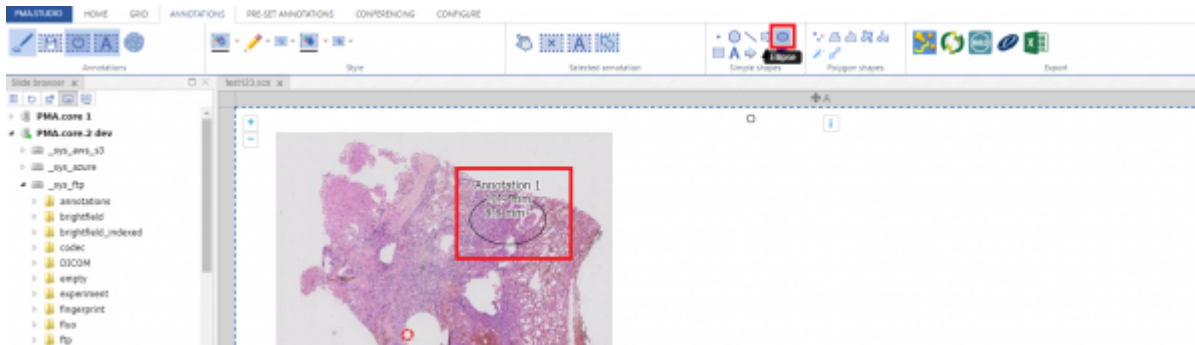


Rectangular Shape

Next is “Rectangular” shape that allows you to draw rectangular annotation on a slide.




Landmark 1 Shape

Next is “Landmark 1” shape that allows you to add first of the 3 landmark annotation on a slide.

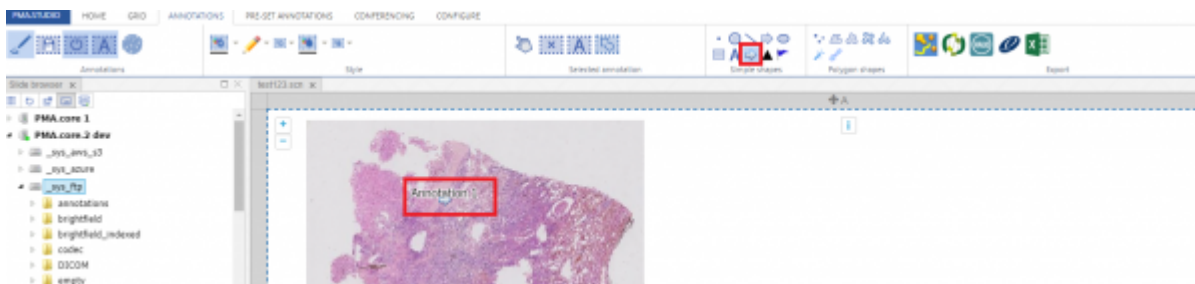


Landmark 2 Shape

Next is “Landmark 2” shape that allows you to add 2nd of the 3 landmark annotation on a slide.

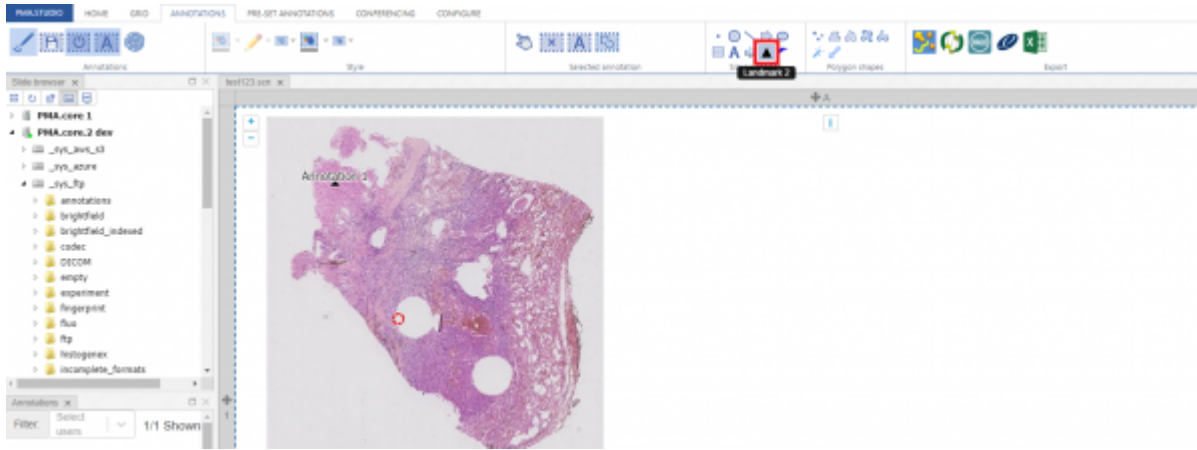

Landmark 3 Shape

Next is "Landmark 3" shape that allows you to add last of the 3 landmark annotation on a slide.

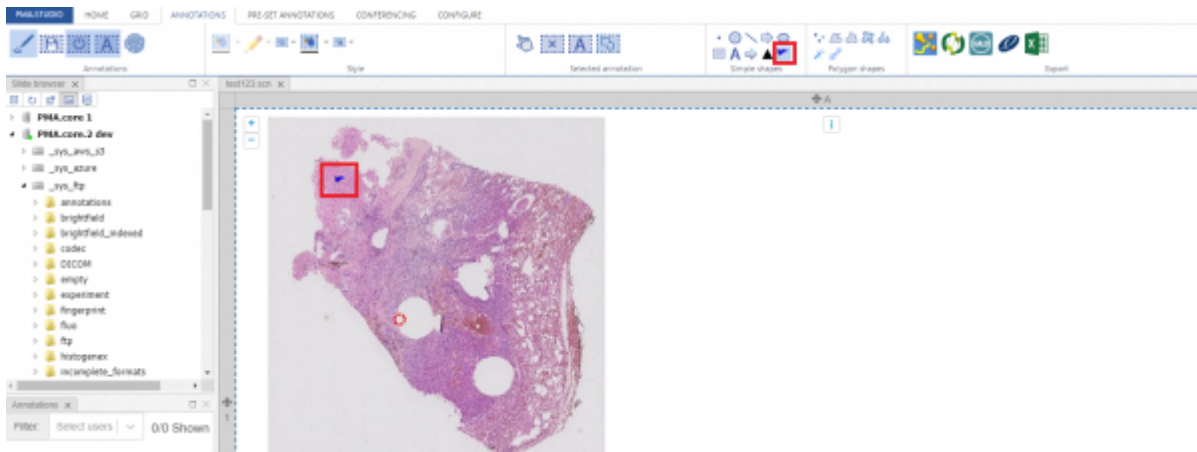

From:
<https://docs.pathomation.com/pma.studio/2.1.0/> - **PMA.studio 2.1.0**

Permanent link:
https://docs.pathomation.com/pma.studio/2.1.0/doku.php?id=annotations_simple&rev=1650555446

Last update: **2022/04/21 18:37**

