


Simple Shapes

Below are the set of tools you can use to draw annotations on slide.

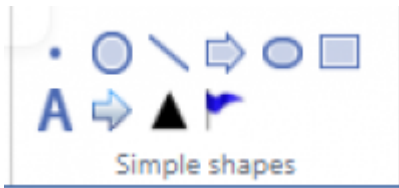

- Point
- Circle
- Line
- Arrow
- Ellipse
- Rectangle
- Text
- Landmark 1
- Landmark 2
- Landmark 3

Point Tool

Lets start with drawing a point, once you select the tool and click on the slide; a point will be drawn and a popup dialog would appear that would allow you to add annotation title and classification. Once you press "Add Annotation" the annotation will be saved and will appear on the slide.

The screenshot illustrates the workflow for using the Point Tool in PMA.studio. It shows the software interface with a histology slide, a dialog box for adding an annotation, and the final result with the annotation saved on the slide.

The dialog box 'Enter annotation text' contains the following fields:

- Name: Annotation 1
- Classification: Choose a classification.

The 'Annotations' panel on the right shows the following table:


Note	Pos	Area (mm ²)	Length (mm)	Polar Count	Created By
No class [1]					
Annotation 1	1	N/A	N/A	N/A	N/A

The 'General Annotation stats' section shows:

Class	Count
No class	1
Selected	0


Circle Tool

Next is “Circle” tool that allows you to draw circle annotation on a slide.

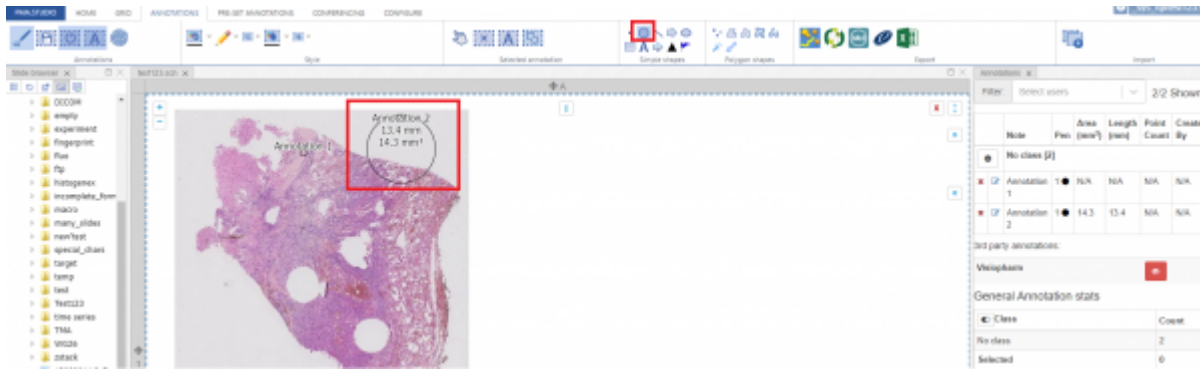


Line Tool

Next is “Line” tool that allows you to draw line annotation on a slide.

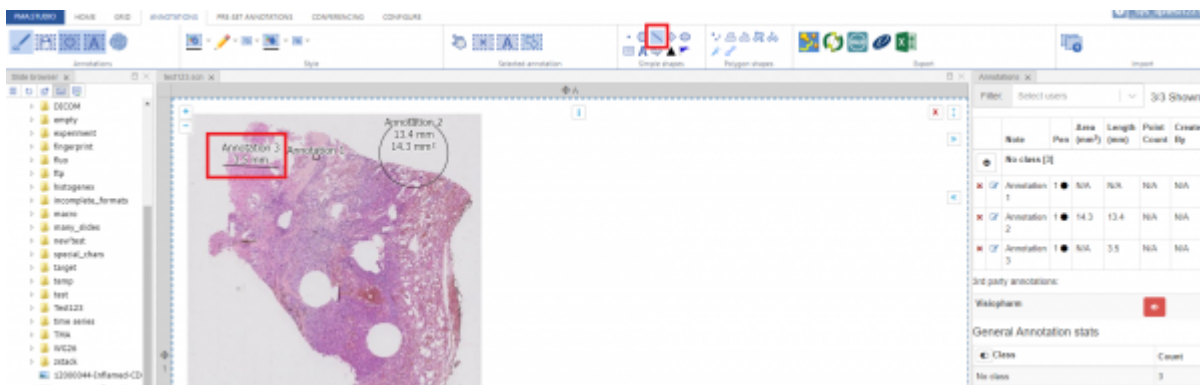


Arrow Tool

Next is “Arrow” tool that allows you to draw arrow annotation on a slide.

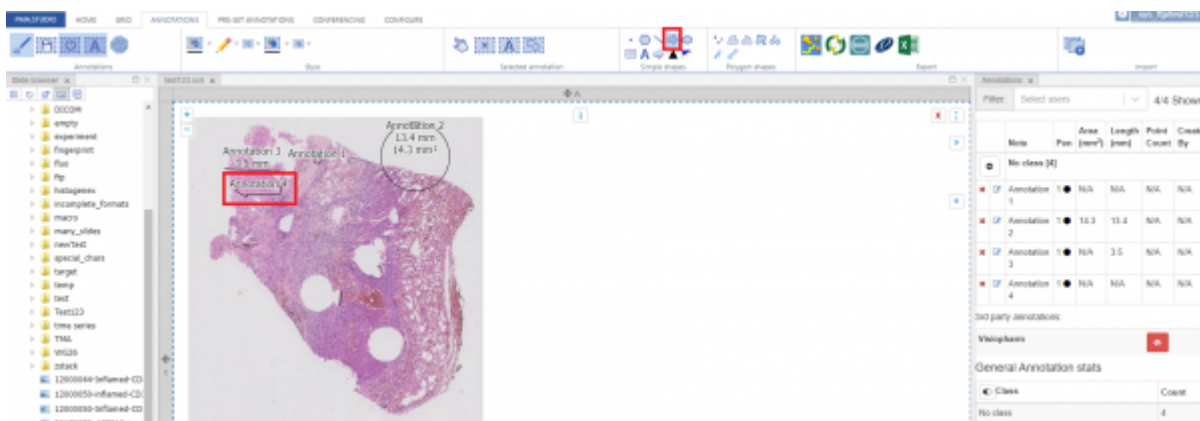

Ellipse Tool

Next is “Ellipse ” tool that allows you to draw ellipse annotation on a slide.

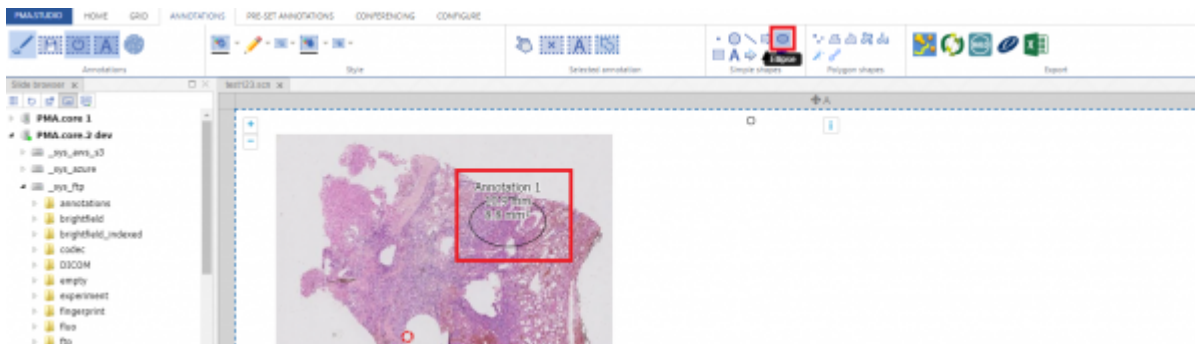


Rectangular Tool

Next is “Rectangular” tool that allows you to draw rectangular annotation on a slide.




Landmark 1 Tool

Next is “Landmark 1” shape that allows you to add first of the 3 landmark annotation on a slide.

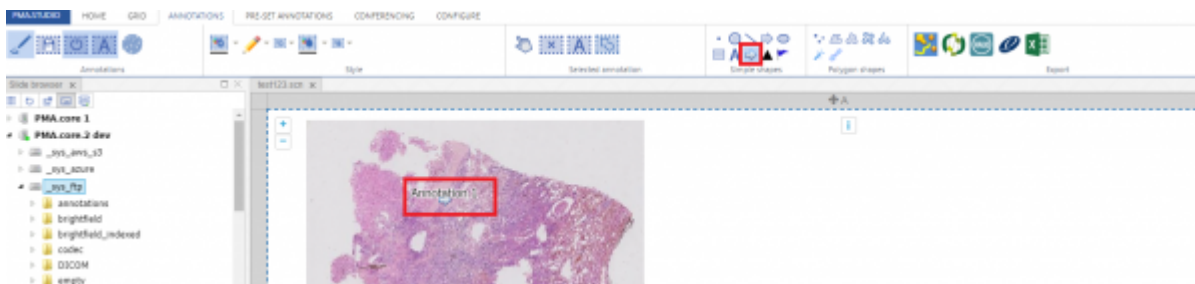


Landmark 2 Tool

Next is “Landmark 2” tool that allows you to add 2nd of the 3 landmark annotation on a slide.

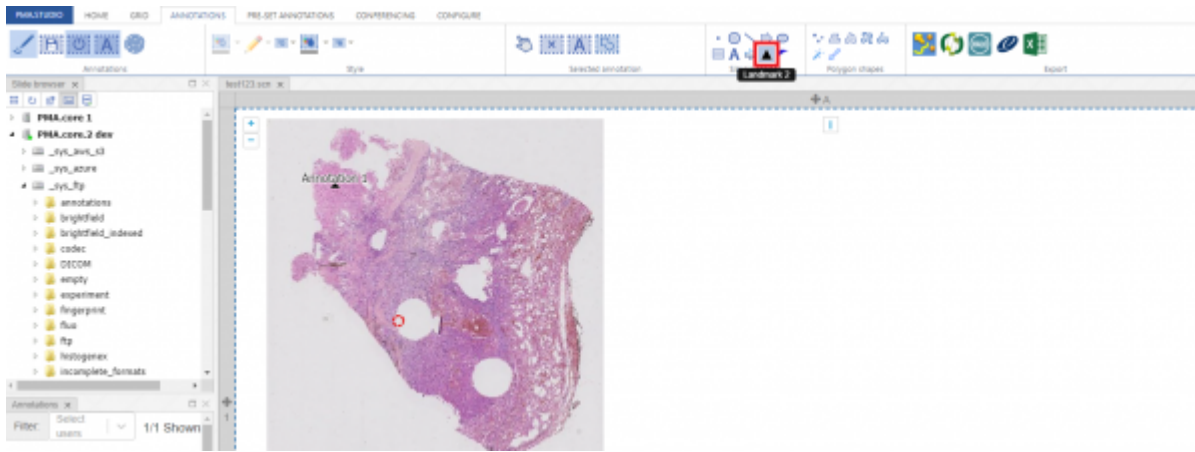

Landmark 3 Tool

Next is "Landmark 3" tool that allows you to add last of the 3 landmark annotation on a slide.

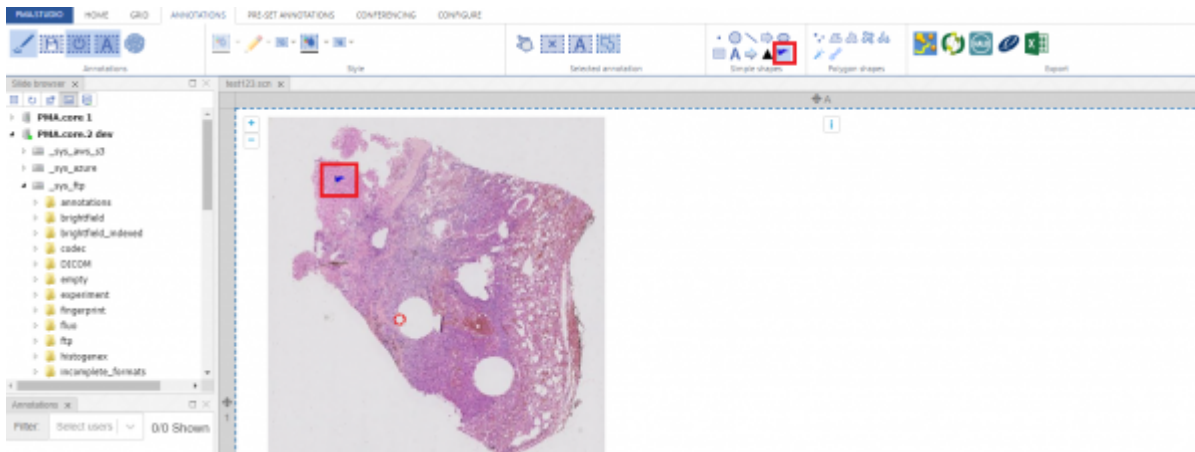

From: <https://docs.pathomation.com/pma.studio/2.1.0/> - **PMA.studio 2.1.0**

Permanent link: https://docs.pathomation.com/pma.studio/2.1.0/doku.php?id=annotations_simple&rev=1650559355

Last update: 2022/04/21 19:42

