


# Annotations

## Simple Shapes


Below are the set of tools you can use to draw annotations on slide.

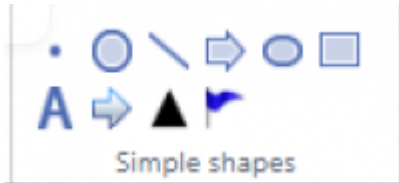

- Point
- Circle
- Line
- Arrow
- Ellipse
- Rectangle
- Text
- Landmark 1
- Landmark 2
- Landmark 3


### Point Tool

Lets start with drawing a point, once you select the tool and click on the slide; a point will be drawn and a popup dialog would appear that would allow you to add annotation title and classification. Once you press "Add Annotation" the annotation will be saved and will appear on the slide.

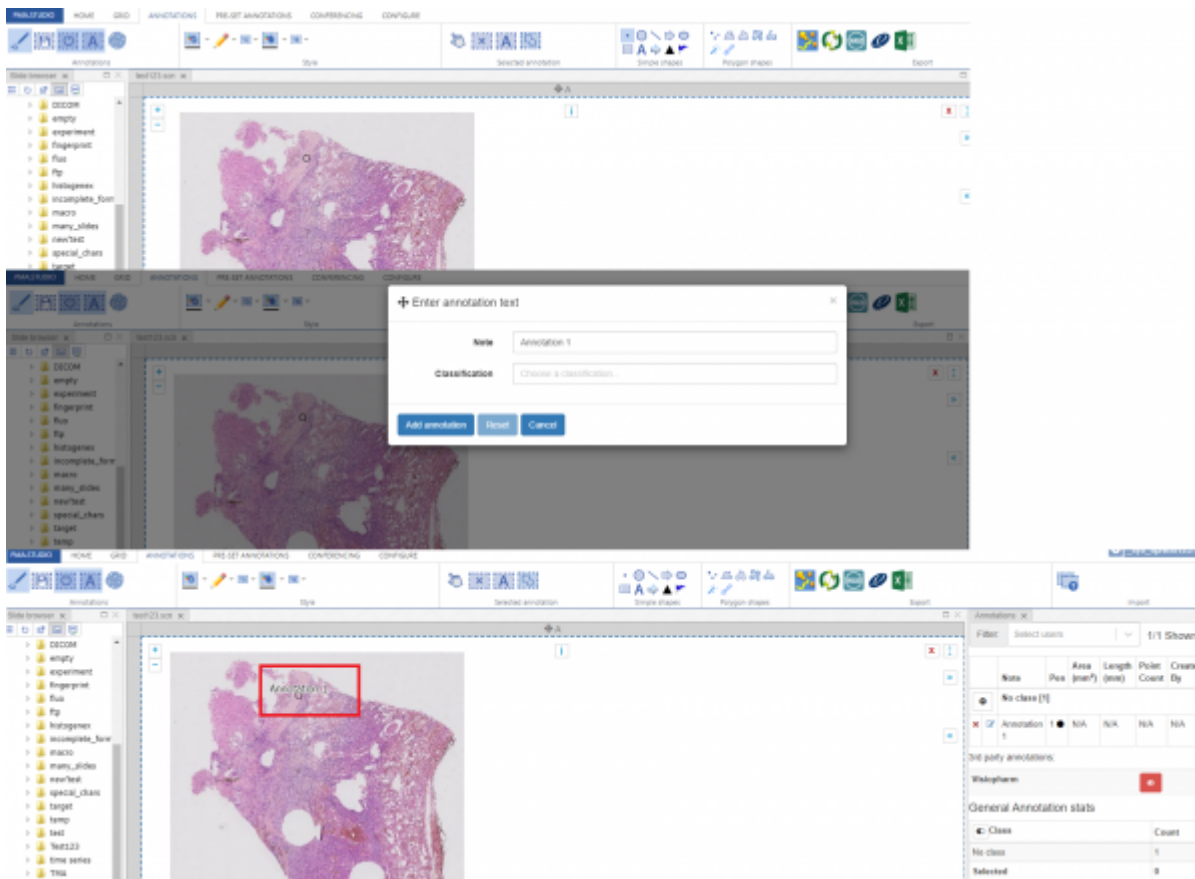


## Circle Tool

Next is "Circle" tool that allows you to draw circle annotation on a slide.

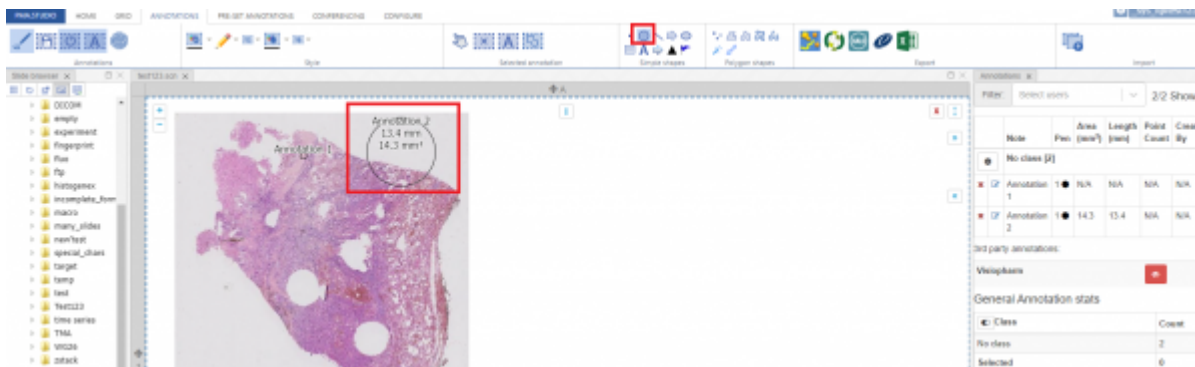


## Line Tool

Next is "Line" tool that allows you to draw line annotation on a slide.

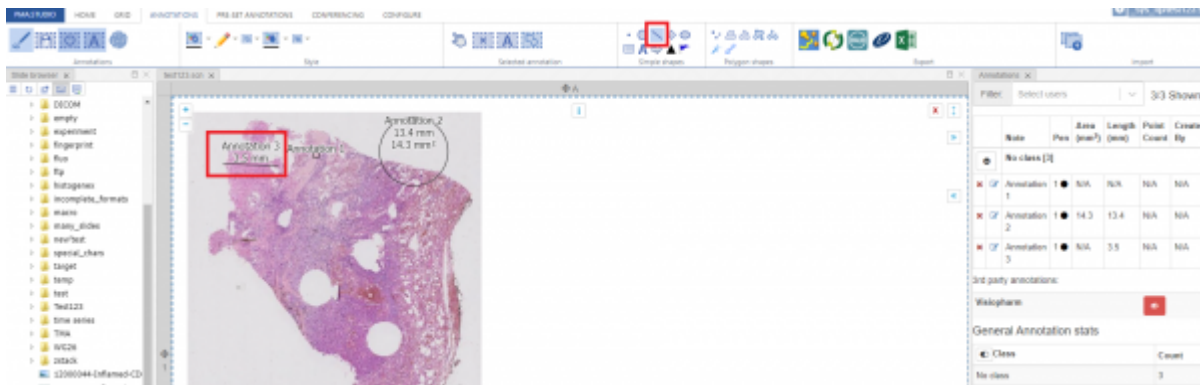


## Arrow Tool

Next is “Arrow” tool that allows you to draw arrow annotation on a slide.

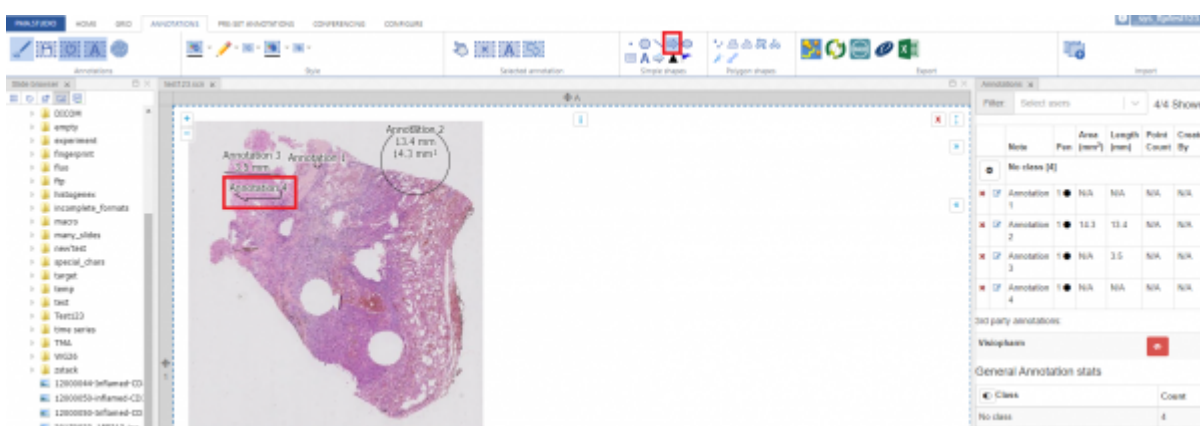

## Ellipse Tool

Next is “Ellipse ” tool that allows you to draw ellipse annotation on a slide.

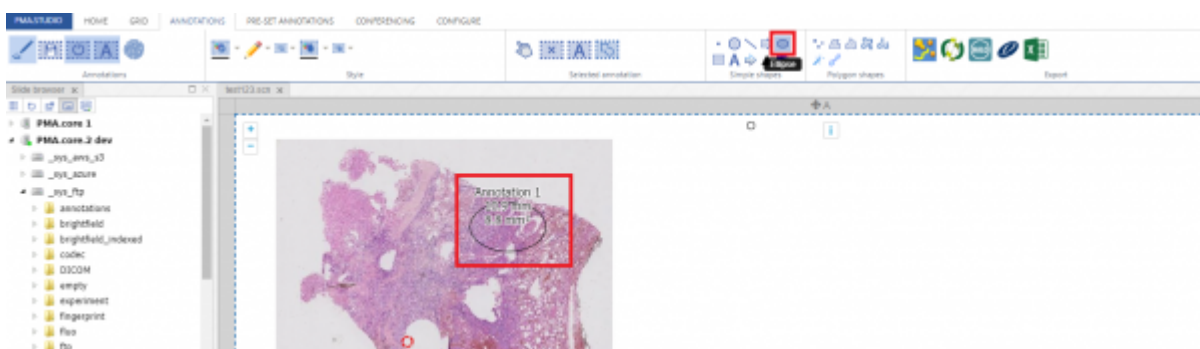


## Rectangular Tool

Next is “Rectangular” tool that allows you to draw rectangular annotation on a slide.




## Landmark 1 Tool

Next is "Landmark 1" shape that allows you to add first of the 3 landmark annotation on a slide.

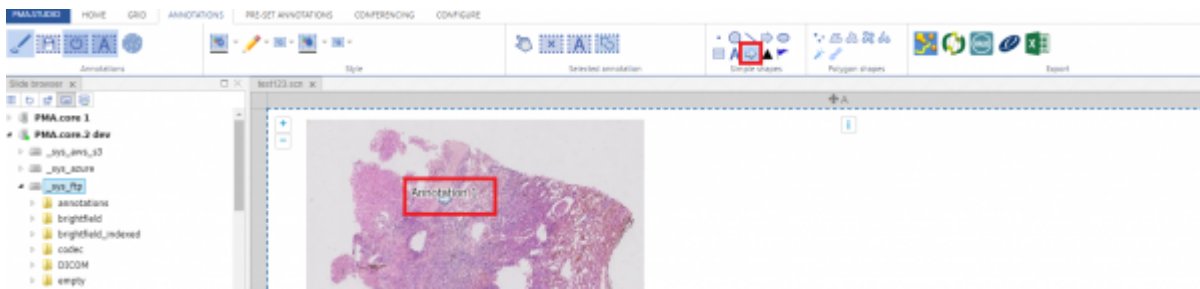


## Landmark 2 Tool

Next is "Landmark 2" tool that allows you to add 2nd of the 3 landmark annotation on a slide.

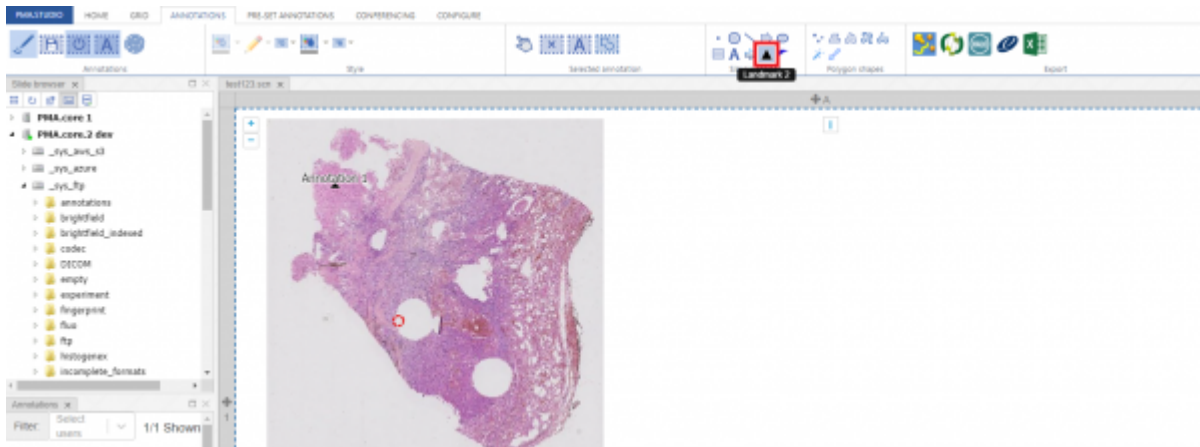

## Landmark 3 Tool

Next is "Landmark 3" tool that allows you to add last of the 3 landmark annotation on a slide.

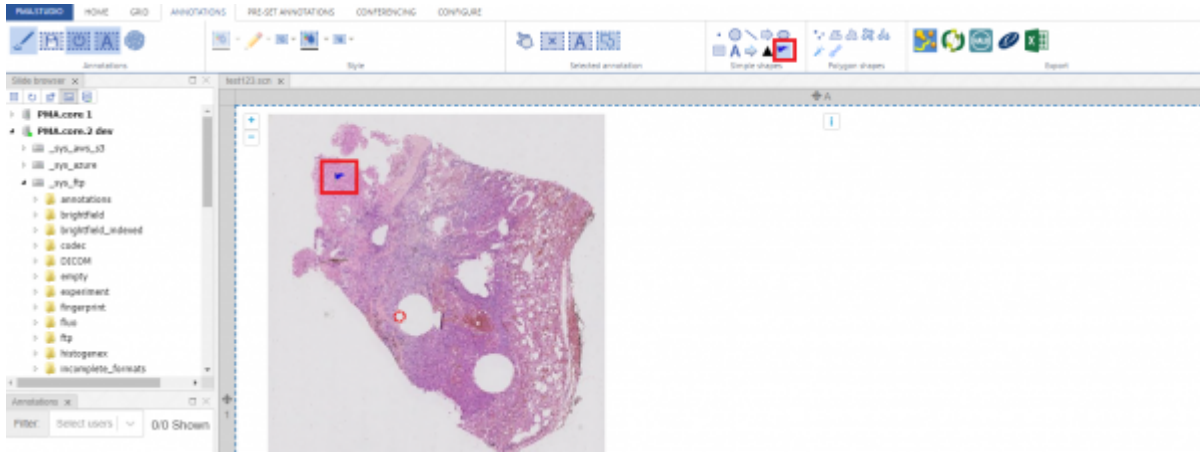

From:  
<https://docs.pathomation.com/pma.studio/2.1.0/> - **PMA.studio 2.1.0**

Permanent link:  
[https://docs.pathomation.com/pma.studio/2.1.0/doku.php?id=annotations\\_simple&rev=1650563796](https://docs.pathomation.com/pma.studio/2.1.0/doku.php?id=annotations_simple&rev=1650563796)

Last update: **2022/04/21 20:56**

