


Annotations

Simple Shapes


Below are the set of tools you can use to draw annotations on slide.

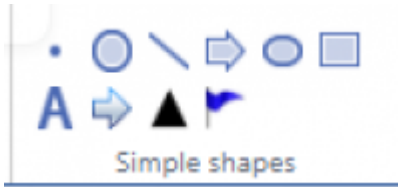

- Point
- Circle
- Line
- Arrow
- Ellipse
- Rectangle
- Text
- Landmark 1
- Landmark 2
- Landmark 3


Point Tool

Lets start with drawing a point, once you select the tool and click on the slide; a point will be drawn and a popup dialog would appear that would allow you to add annotation title and classification. Once you press "Add Annotation" the annotation will be saved and will appear on the slide.

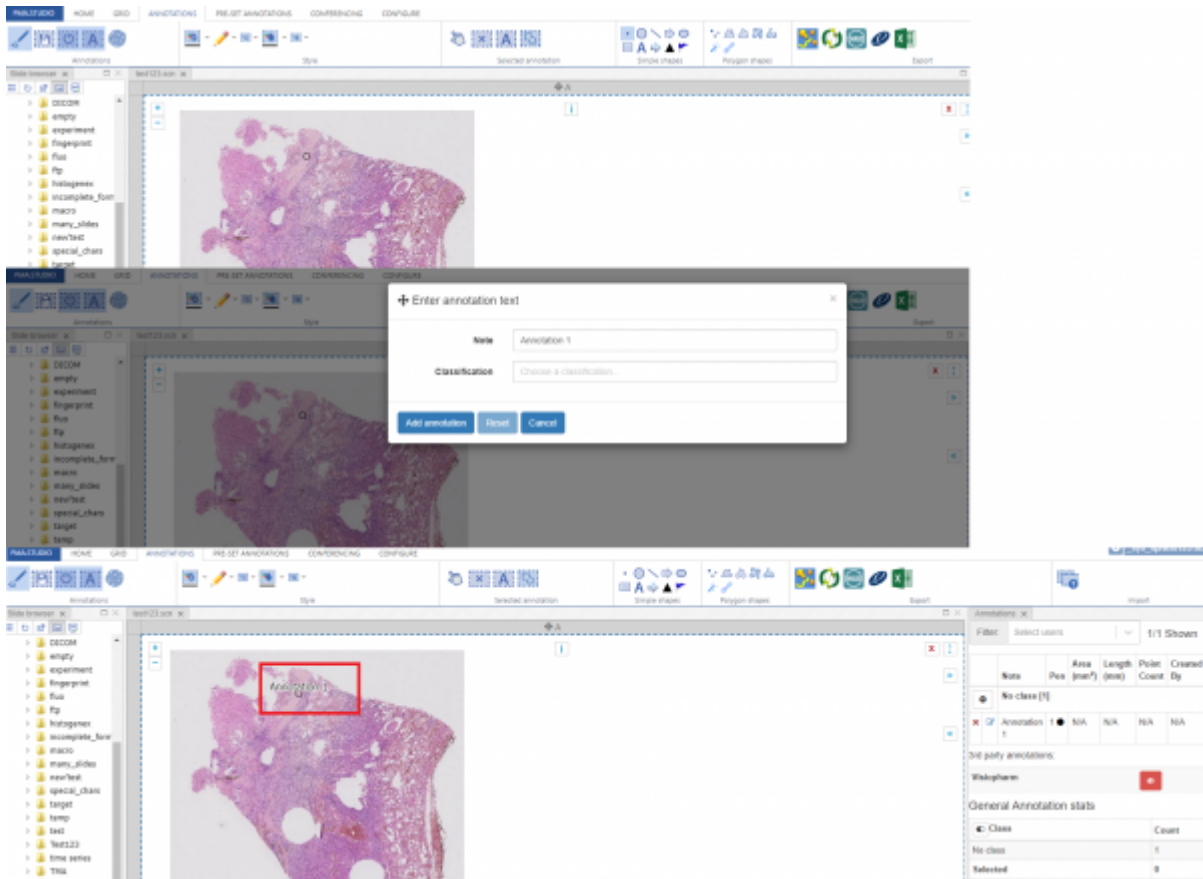


Circle Tool

Next is "Circle" tool that allows you to draw circle annotation on a slide.

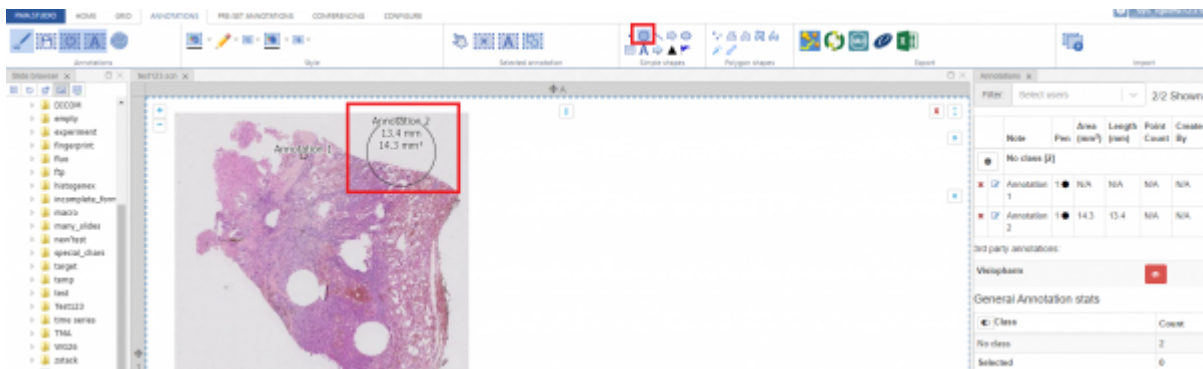


Line Tool

Next is "Line" tool that allows you to draw line annotation on a slide.

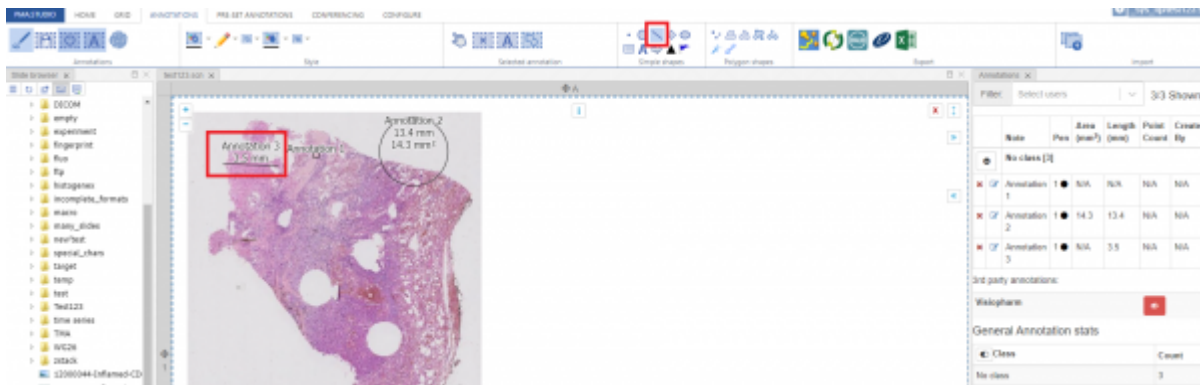


Arrow Tool

Next is “Arrow” tool that allows you to draw arrow annotation on a slide.

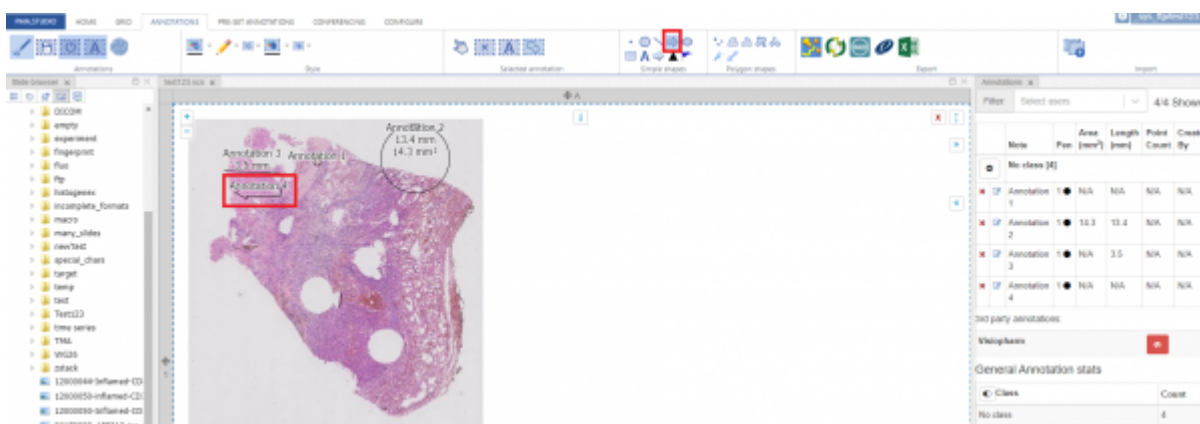

Ellipse Tool

Next is “Ellipse ” tool that allows you to draw ellipse annotation on a slide.

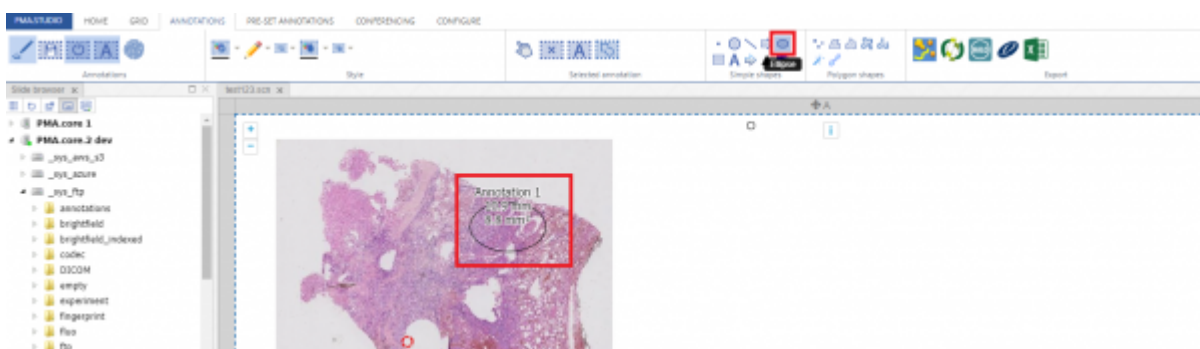


Rectangular Tool

Next is “Rectangular” tool that allows you to draw rectangular annotation on a slide.




Landmark 1 Tool

Next is "Landmark 1" shape that allows you to add first of the 3 landmark annotation on a slide.

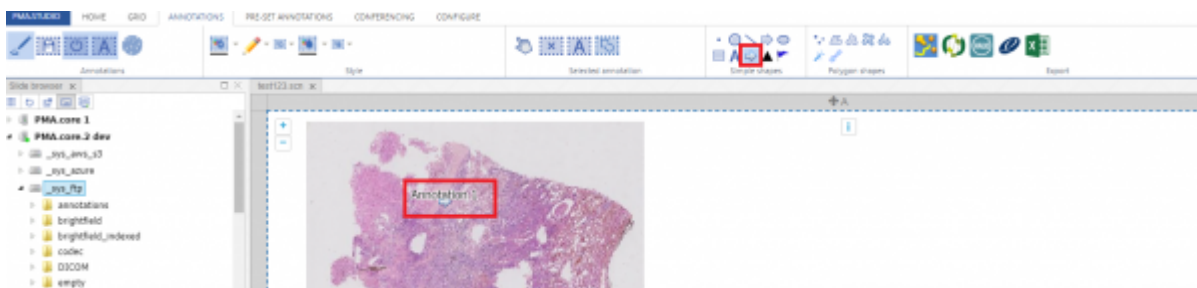


Landmark 2 Tool

Next is "Landmark 2" tool that allows you to add 2nd of the 3 landmark annotation on a slide.

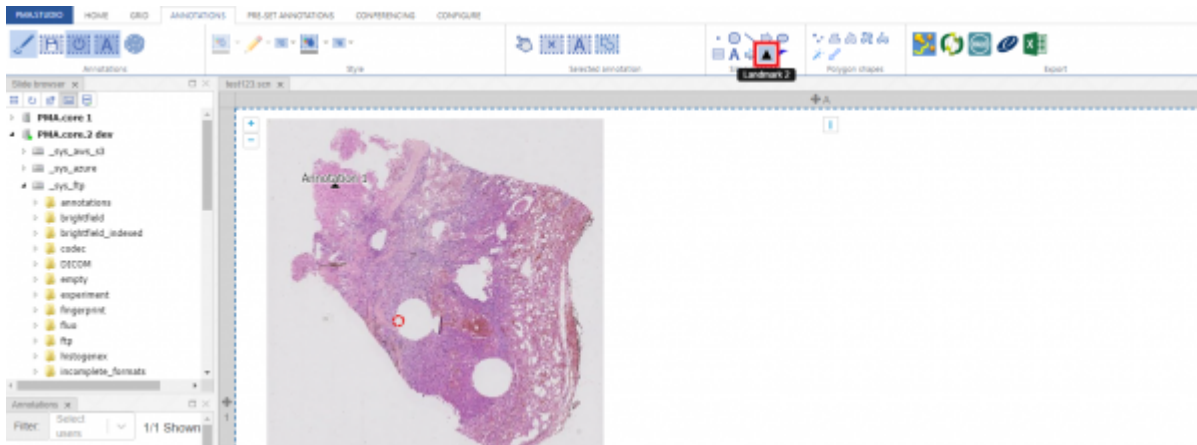

Landmark 3 Tool

Next is "Landmark 3" tool that allows you to add last of the 3 landmark annotation on a slide.

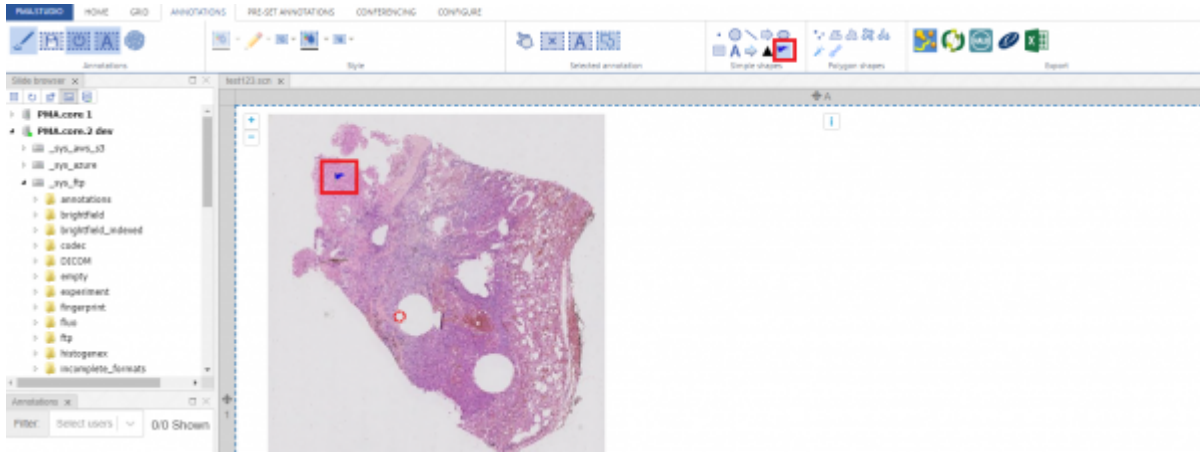

From:
<https://docs.pathomation.com/pma.studio/2.1.0/> - **PMA.studio 2.1.0**

Permanent link:
https://docs.pathomation.com/pma.studio/2.1.0/doku.php?id=annotations_simple&rev=1650563821

Last update: **2022/04/21 20:57**

