

# Links and sharing

## Auto-navigate

In some instances, you want to point people to PMA.studio, log in, and navigate directly to a particular slide or folder.

This use is mostly suited when everybody in your organization has access to PMA.studio, and you want people to use their own credentials.

In order for this to work, people must

- Have access to PMA.studio
- Have access to the PMA.core tile server on the slide/folder resides
- Have access to the slide/folder that you're referring them to

=== To a particular slide

=== To a particular folder

## Automate the share dialog

The content here only describes the syntax to automate share link creation. For a discussion and comprehensive user story, see [our blog article on the subject](#).

Note that when creating share links in this manner, users that receive the link will

- not have access to all PMA.studio features (only the specific slide(s) that you refer to)
- impersonate the account used to create the link with in the first place (probably your own)
- log access to the shared content under the impersonated credentials

=== For a particular slide

When

- PMA.studio runs at <https://server/pma.studio>, and
- Your content resides on PMA.core <https://server/pma.core>, and
- Your slide is called `slides/case/my-scan.svs`

Then your direct link to that slide becomes

<https://server/pma.studio/?serverUrl=https%3A%2F%2Fserver%2Fpma.core%2F&path=slides%2Fcase%2Fmy-scan.svs>

=== For a particular folder

From:

<https://docs.pathomation.com/pma.studio/2.1.0/> - **PMA.studio 2.1.0**

Permanent link:

[https://docs.pathomation.com/pma.studio/2.1.0/doku.php?id=auto\\_slides&rev=1643021029](https://docs.pathomation.com/pma.studio/2.1.0/doku.php?id=auto_slides&rev=1643021029)

Last update: **2022/01/24 13:43**

