


# Operating modes

PMA.studio can be used in different ways. The basic mode of operandi is **Full mode**, which allows one to view and annotate whole slide images. In addition to full mode, however, the viewer can also be run in **Embedded mode** (within an html iframe). Finally, a **separate administrative interface** is available to configure the interaction with other components of the platform, as well as the behavior of the various in the various operating modes.

## Full mode

The default mode of the view if “full mode”.

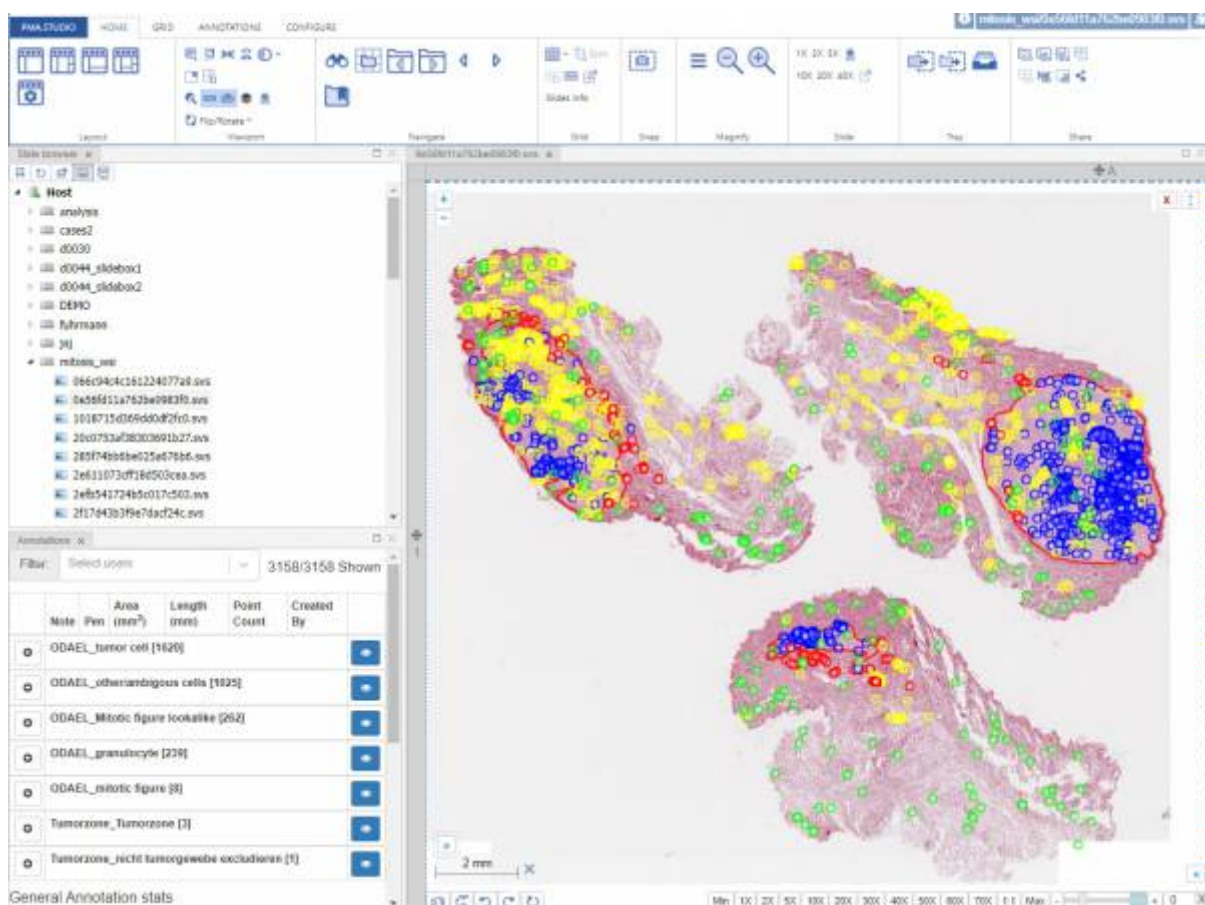

The loading sequence of the application is typically as follows:

- Display a splash screen
- If this is your first time pulling up the application in a particular browser, you may need to accept the End-User License Agreement. It is very important to read through this one, as it may vary from one organization to another.
- A basic framework is rendered, as illustrated in the [Navigation](#) section.
- Select an instance of PMA.core to connect to, fill out your credentials, and you're ready to browse whole slide images.

A word about the startup sequence: PMA.studio runs in a webbrowser. It is an understatement however to consider it a website. PMA.studio is a highly advanced web-accessible application that

makes use of the latest techniques in HTTP communication, HTML 5, and browser technology. Nevertheless no external plugins (Java, Flash, Silverlight...) are needed.

## Embedded mode

From:  
<https://docs.pathomation.com/pma.studio/2.1.0/> - **PMA.studio 2.1.0**

Permanent link:  
[https://docs.pathomation.com/pma.studio/2.1.0/doku.php?id=operating\\_modes&rev=1670341064](https://docs.pathomation.com/pma.studio/2.1.0/doku.php?id=operating_modes&rev=1670341064)

Last update: **2022/12/06 18:37**

