



# Slide information panel

The Info panel displays available slide information for currently selected slide.

Info	
File name	_sys_ref/brightfield/Grundium/hp1 (1).svs
Last Modified	Mar 01 2021 18:11
Width	131,072 pixels / 32.840 mm
Height	73,728 pixels / 18.473 mm
Max Magnification	40X
Micrometres Per Pixel X	0.250552
Micrometres Per Pixel Y	0.250552
Physical Size	328.53 MB
Tile Size	512 x 512 pixels
Barcode	
Barcode text	
Overview	
Macro	N/A
Layers	1
Channels	1

Currently selected slide is highlighted with blue dashed border.

Columns are pretty self-descriptive.

The barcode, macro and overview thumbnails are clickable. Clicking on the images will open then in a new browser tab.

From:  
<https://docs.pathomation.com/pma.studio/2.1.0/> - **PMA.studio 2.1.0**

Permanent link:  
[https://docs.pathomation.com/pma.studio/2.1.0/doku.php?id=panel\\_info&rev=1677836884](https://docs.pathomation.com/pma.studio/2.1.0/doku.php?id=panel_info&rev=1677836884)

Last update: **2023/03/03 12:48**

