

Available ribbon commands

See also the [tab inventory page](#).

| Ribbon section | Command name | Parameters |
|----------------|--------------------------------|--------------------|
| File menu | custom-panel-algo-ichi | props= |
| | custom-panel-algo-ni | props= |
| | custom-panel-ai-prov-mindpeak | props= |
| | custom-panel-ai-prov-spintellx | props= |
| | file | |
| | about-us | |
| | show-eula | |
| | manual | |
| | sign-out | |
| Ribbon section | Command name | Parameters |
| Home tab | simple-layout | |
| | detailed-layout | |
| | detailed-layout | |
| | viewport-toggle-overview | |
| | viewport-toggle-slide-path | |
| | viewport-toggle-barcode | |
| | add-tab | |
| | reset-layout | |
| | viewport-toggle-zoom | |
| | viewport-toggle-scalebar | |
| | viewport-toggle-label | |
| | panel-toggle-color | |
| | panel-toggle-search | |
| | open-folder-in-grid | |
| | prev-directory | |
| | next-directory | |
| | prev-image | |
| | next-image | |
| | add-folder-bookmark | |
| | reset-grid | |
| | clear-cell | |
| | open-in-tab | |
| | popout-info | |
| | snapshot | |
| | quick-snapshot | |
| | toggle-toolbar | |
| | increase-zoom | |
| | decrease-zoom | |
| | custom-zoom-1x | pma-magnification= |
| | custom-zoom-2x | pma-magnification= |
| custom-zoom-5x | pma-magnification= | |

| Ribbon section | Command name | Parameters |
|---------------------------------|---|--------------------|
| | custom-zoom-10x | pma-magnification= |
| | custom-zoom-20x | pma-magnification= |
| | custom-zoom-40x | pma-magnification= |
| | panel-toggle-color | |
| | send-slide-to-tray | |
| | send-to-tray | |
| | panel-toggle-tray | |
| | share-slide-tray | |
| | quick-share-folder | |
| | quick-share-slide | |
| | quick-share-slide-roi | |
| | quick-share-grid | |
| | quick-share-grid-rois | |
| | quick-share-thumbnail | |
| | share-slide | |
| Ribbon section | Command name | Parameters |
| Grid tab | toggle-1x1-grid | |
| | toggle-2x2-grid | |
| | toggle-3x3-grid | |
| | sync-views-toggle | |
| | reset-grid | |
| | clear-cell | |
| | open-in-tab | |
| | viewport-add-row | |
| | viewport-remove-row | |
| | viewport-remove-row | |
| | viewport-remove-column | |
| | flip-rotate-menu | |
| | reset-flip | |
| | flip-horizontally | |
| | flip-vertically | |
| | reset-rotation | |
| | rotate-clockwise-90 | |
| | rotate-counter-clockwise-90 | |
| | rotate-180 | |
| | zoom-in | |
| | panel-toggle-annotations | |
| | save-annotations | |
| | toggle-annotations | |
| | toggle-annotation-text | |
| | toggle-fingerprint | |
| Ribbon section | Command name | Parameters |
| Annotations tab | Cursor | |
| | delete-annotation | |
| | delete-all-annotations | |

| Ribbon section | Command name | Parameters |
|----------------------------------|--|--|
| | edit-annotation-text | |
| | merge-selected-annotations | |
| | modify-selected-annotation | |
| | Point | |
| | Circle | |
| | Line | |
| | Arrow | |
| | Ellipse | |
| | Rectangle | |
| | Landmark1 | |
| | MultiPoint | |
| | Polygon | |
| | ClosedFreehand | |
| | CompoundFreehand | |
| | Freehand | |
| | Wand | |
| | Brush | |
| | export-to-wkt | |
| | export-to-mld | |
| | export-to-annotations | |
| | export-to-aperio | |
| | export-to-csv | |
| | import-to-wkt | |
| | import-to-mld | |
| | import-to-halo | |
| | import-to-csv | |
| Ribbon section | Command name | Parameters |
| Pre-set tab | preset-Circle0237mm2 | pma-shape, pma-size-a, pma-size-b |
| | preset-Circle1mm | pma-shape, pma-size-a, pma-size-b |
| | preset-Circle2mm | pma-shape, pma-size-a, pma-size-b |
| | preset-Circle1cm | pma-shape, pma-size-a, pma-size-b |
| | preset-Square5mm | pma-shape, pma-size-a, pma-size-b |
| | preset-Square1cm | pma-shape, pma-size-a, pma-size-b |
| | preset-Tumor | pma-classification, pma-shape, pma-color |
| | preset-Macro | pma-classification, pma-shape, pma-color |
| Ribbon section | Command name | Parameters |
| Conferencing tab | create-conference | |
| | join-conference | |
| | start-conference | |
| | pause-conference | |
| | stop-conference | |
| | toggle-everyone-control-conference | |
| | broadcast-conference | |
| | panel-toggle-conference | |
| | panel-toggle-forms | |

| Ribbon section | Command name | Parameters |
|----------------|---|------------|
| | panel-toggle-links | |
| | panel-toggle-navigation | |
| | panel-toggle-bookmarks | |

From:
<https://docs.pathomation.com/pma.studio/2.1.0/> - **PMA.studio 2.1.0**

Permanent link:
https://docs.pathomation.com/pma.studio/2.1.0/doku.php?id=ribbon_commands&rev=1676035708

Last update: **2023/02/10 16:28**

