

## Available ribbon commands

See also the [tab inventory page](#).

| Ribbon section | Command name                   | Parameters |
|----------------|--------------------------------|------------|
| File menu      | custom-panel-algo-ichi         | props=     |
|                | custom-panel-algo-ni           | props=     |
|                | custom-panel-ai-prov-mindpeak  | props=     |
|                | custom-panel-ai-prov-spintellx | props=     |
|                | file                           |            |
|                | about-us                       |            |
|                | show-eula                      |            |
|                | manual                         |            |
|                | sign-out                       |            |
| Ribbon section | Command name                   | Parameters |
| Home tab       | simple-layout                  |            |
|                | detailed-layout                |            |
|                | detailed-layout                |            |
|                | viewport-toggle-overview       |            |
|                | viewport-toggle-slide-path     |            |
|                | viewport-toggle-barcode        |            |
|                | viewport-toggle-rotation-flip  |            |
|                | add-tab                        |            |
|                | reset-layout                   |            |
|                | viewport-toggle-zoom           |            |
|                | viewport-toggle-scalebar       |            |
|                | viewport-toggle-label          |            |
|                | panel-toggle-color             |            |
|                | panel-toggle-search            |            |
|                | open-folder-in-grid            |            |
|                | prev-directory                 |            |
|                | next-directory                 |            |
|                | prev-image                     |            |
|                | next-image                     |            |
|                | add-folder-bookmark            |            |
|                | reset-grid                     |            |
|                | clear-cell                     |            |
|                | open-in-tab                    |            |
|                | popout-info                    |            |
|                | snapshot                       |            |
|                | quick-snapshot                 |            |
|                | toggle-toolbar                 |            |
| increase-zoom  |                                |            |
| decrease-zoom  |                                |            |
| custom-zoom-1x | pma-magnification=             |            |
| custom-zoom-2x | pma-magnification=             |            |

| Ribbon section                  | Command name                                | Parameters         |
|---------------------------------|---|--------------------|
|                                 | <a href="#">custom-zoom-5x</a>              | pma-magnification= |
|                                 | <a href="#">custom-zoom-10x</a>             | pma-magnification= |
|                                 | <a href="#">custom-zoom-20x</a>             | pma-magnification= |
|                                 | <a href="#">custom-zoom-40x</a>             | pma-magnification= |
|                                 | <a href="#">panel-toggle-color</a>          |                    |
|                                 | <a href="#">send-slide-to-tray</a>          |                    |
|                                 | <a href="#">send-to-tray</a>                |                    |
|                                 | <a href="#">panel-toggle-tray</a>           |                    |
|                                 | <a href="#">share-slide-tray</a>            |                    |
|                                 | <a href="#">quick-share-folder</a>          |                    |
|                                 | <a href="#">quick-share-slide</a>           |                    |
|                                 | <a href="#">quick-share-slide-roi</a>       |                    |
|                                 | <a href="#">quick-share-grid</a>            |                    |
|                                 | <a href="#">quick-share-grid-rois</a>       |                    |
|                                 | <a href="#">quick-share-thumbnail</a>       |                    |
|                                 | <a href="#">share-slide</a>                 |                    |
| Ribbon section                  | Command name                                | Parameters         |
| <a href="#">Grid tab</a>        | <a href="#">toggle-1x1-grid</a>             |                    |
|                                 | <a href="#">toggle-2x2-grid</a>             |                    |
|                                 | <a href="#">toggle-3x3-grid</a>             |                    |
|                                 | <a href="#">sync-views-toggle</a>           |                    |
|                                 | <a href="#">reset-grid</a>                  |                    |
|                                 | <a href="#">clear-cell</a>                  |                    |
|                                 | <a href="#">open-in-tab</a>                 |                    |
|                                 | <a href="#">viewport-add-row</a>            |                    |
|                                 | <a href="#">viewport-remove-row</a>         |                    |
|                                 | <a href="#">viewport-remove-row</a>         |                    |
|                                 | <a href="#">viewport-remove-column</a>      |                    |
|                                 | <a href="#">flip-rotate-menu</a>            |                    |
|                                 | <a href="#">reset-flip</a>                  |                    |
|                                 | <a href="#">flip-horizontally</a>           |                    |
|                                 | <a href="#">flip-vertically</a>             |                    |
|                                 | <a href="#">reset-rotation</a>              |                    |
|                                 | <a href="#">rotate-clockwise-90</a>         |                    |
|                                 | <a href="#">rotate-counter-clockwise-90</a> |                    |
|                                 | <a href="#">rotate-180</a>                  |                    |
|                                 | <a href="#">zoom-in</a>                     |                    |
|                                 | <a href="#">panel-toggle-annotations</a>    |                    |
|                                 | <a href="#">save-annotations</a>            |                    |
|                                 | <a href="#">toggle-annotations</a>          |                    |
|                                 | <a href="#">toggle-annotation-text</a>      |                    |
|                                 | <a href="#">toggle-fingerprint</a>          |                    |
| Ribbon section                  | Command name                                | Parameters         |
| <a href="#">Annotations tab</a> | <a href="#">Cursor</a>                      |                    |
|                                 | <a href="#">delete-annotation</a>           |                    |

| Ribbon section                   | Command name                                       | Parameters                               |
|----------------------------------|--|--|
|                                  | <a href="#">delete-all-annotations</a>             |  |
|                                  | <a href="#">edit-annotation-text</a>               |  |
|                                  | <a href="#">merge-selected-annotations</a>         |  |
|                                  | <a href="#">union-selected-annotations</a>         |  |
|                                  | <a href="#">modify-selected-annotation</a>         |  |
|                                  | <a href="#">Point</a>                              |  |
|                                  | <a href="#">Circle</a>                             |  |
|                                  | <a href="#">Line</a>                               |  |
|                                  | <a href="#">Arrow</a>                              |  |
|                                  | <a href="#">Ellipse</a>                            |  |
|                                  | <a href="#">Rectangle</a>                          |  |
|                                  | <a href="#">Landmark1</a>                          |  |
|                                  | <a href="#">MultiPoint</a>                         |  |
|                                  | <a href="#">Polygon</a>                            |  |
|                                  | <a href="#">ClosedFreehand</a>                     |  |
|                                  | <a href="#">CompoundFreehand</a>                   |  |
|                                  | <a href="#">Freehand</a>                           |  |
|                                  | <a href="#">Wand</a>                               |  |
|                                  | <a href="#">Brush</a>                              |  |
|                                  | <a href="#">export-to-wkt</a>                      |  |
|                                  | <a href="#">export-to-mld</a>                      |  |
|                                  | <a href="#">export-to-annotations</a>              |  |
|                                  | <a href="#">export-to-aperio</a>                   |  |
|                                  | <a href="#">export-to-csv</a>                      |  |
|                                  | <a href="#">import-to-wkt</a>                      |  |
|                                  | <a href="#">import-to-mld</a>                      |  |
|                                  | <a href="#">import-to-halo</a>                     |  |
|                                  | <a href="#">import-to-csv</a>                      |  |
| Ribbon section                   | Command name                                       | Parameters                               |
| <a href="#">Pre-set tab</a>      | <a href="#">preset-Circle0237mm2</a>               | pma-shape, pma-size-a, pma-size-b        |
|                                  | <a href="#">preset-Circle1mm</a>                   | pma-shape, pma-size-a, pma-size-b        |
|                                  | <a href="#">preset-Circle2mm</a>                   | pma-shape, pma-size-a, pma-size-b        |
|                                  | <a href="#">preset-Circle1cm</a>                   | pma-shape, pma-size-a, pma-size-b        |
|                                  | <a href="#">preset-Square5mm</a>                   | pma-shape, pma-size-a, pma-size-b        |
|                                  | <a href="#">preset-Square1cm</a>                   | pma-shape, pma-size-a, pma-size-b        |
|                                  | <a href="#">preset-Tumor</a>                       | pma-classification, pma-shape, pma-color |
|                                  | <a href="#">preset-Macro</a>                       | pma-classification, pma-shape, pma-color |
| Ribbon section                   | Command name                                       | Parameters                               |
| <a href="#">Conferencing tab</a> | <a href="#">create-conference</a>                  |  |
|                                  | <a href="#">join-conference</a>                    |  |
|                                  | <a href="#">start-conference</a>                   |  |
|                                  | <a href="#">pause-conference</a>                   |  |
|                                  | <a href="#">stop-conference</a>                    |  |
|                                  | <a href="#">toggle-everyone-control-conference</a> |  |
|                                  | <a href="#">broadcast-conference</a>               |  |

| Ribbon section | Command name                            | Parameters |
|----------------|---|------------|
|                | <a href="#">panel-toggle-conference</a> |            |
|                | <a href="#">panel-toggle-forms</a>      |            |
|                | <a href="#">panel-toggle-links</a>      |            |
|                | <a href="#">panel-toggle-navigation</a> |            |
|                | <a href="#">panel-toggle-bookmarks</a>  |            |

From:  
<https://docs.pathomation.com/pma.studio/2.1.0/> - **PMA.studio 2.1.0**

Permanent link:  
[https://docs.pathomation.com/pma.studio/2.1.0/doku.php?id=ribbon\\_commands&rev=1680715201](https://docs.pathomation.com/pma.studio/2.1.0/doku.php?id=ribbon_commands&rev=1680715201)

Last update: **2023/04/05 20:20**

